

# ONETOUCH® *Diabetes Management Software*



## User Manual

See inside for a complete description of program functions

>> Link to the [Table of Contents](#)

>> Link to the [Index](#)

Information in this document is subject to change without notice. The software described in this document is furnished under a licensing agreement and may be used only in accordance with terms and conditions of the agreement. It is illegal to copy the software on any medium except as specifically allowed in the agreement.

© 2007-2014 LifeScan, Inc. a Johnson & Johnson company. All rights reserved.  
Microsoft, Windows and MS-DOS are registered trademarks of Microsoft Corporation.  
All other products are trademarked by their respective companies.

ONETOUCH®

# Table of Contents

<b>Part 1: Before You Begin</b>	<b>6</b>
WELCOME	7
ABOUT THIS MANUAL	9
GETTING TECHNICAL SUPPORT	12
CHANGES TO THE SOFTWARE:	12
NAVIGATING ONETOUCH® DMS	13
<i>Keyboard and Mouse Pointer Conventions</i>	13
<i>OneTouch® DMS Button Behavior</i>	14
<b>Part 2: Getting Started</b>	<b>18</b>
INSTALLING ONETOUCH® DMS ON YOUR PC	19
STARTING UP ONETOUCH® DMS	24
CUSTOMIZING ONETOUCH® DMS FOR PERSONAL USE	26
<i>Patient Setup</i>	28
General Tab	30
Schedule Tab	33
Medications Tab	38
Regimen Tab	40
Complications Tab	42
Find a Patient Tab	43
<i>Preferences</i>	44
General Tab	45
<i>Displaying Units of Measure</i>	47
<i>Displaying Results as Plasma Values or Whole Blood Values</i>	48
Report Options Tab	49
Global Targets Tab	52

## Table of Contents (cont.)

<a href="#">Hypoglycemic Tab</a>	55
<a href="#">Modify Meter Settings</a>	57
<a href="#">Clock Tab</a>	58
<a href="#">General Tab</a>	60
<a href="#">Schedule Tab</a>	62
<a href="#">Insulin Tab</a>	64
<a href="#">Oral Meds Tab</a>	65
<a href="#">Targets &amp; Hypoglycemic Tab</a>	67
<b>Part 3: Diabetes Management</b>	<b>68</b>
<a href="#">DOWNLOAD METER READINGS</a>	69
<a href="#">Connecting a Meter to Your PC</a>	70
<a href="#">Transferring Meter Readings</a>	71
<a href="#">ASSIGNING METER READINGS TO THE DATABASE</a>	72
<a href="#">CLEAR METER READINGS</a>	75
<a href="#">Completing the Clear Meter Readings Process</a>	76
<a href="#">VIEW AND MANAGE REPORTS AND DATA RECORDS</a>	77
<a href="#">General Report Information</a>	77
<a href="#">Choose Your Report</a>	85
<a href="#">Logbook Report</a>	85
<a href="#">The Day View screen of the Logbook Report</a>	90
<a href="#">Glucose Trend Report</a>	92
<a href="#">Pie Chart Report</a>	93
<a href="#">Summary Report</a>	94
<a href="#">Standard Day Report</a>	95
<a href="#">Insulin Report</a>	96

## Table of Contents (cont.)

<a href="#">Health Checks Report</a>	98
<a href="#">Average Readings Report</a>	100
<a href="#">Exception Report</a>	102
<a href="#">Data List Report</a>	104
<a href="#">Histogram Report</a>	106
<a href="#">Printing, Faxing and E-Mailing OneTouch® DMS Reports</a>	107
<a href="#">Modify Data Records (Edit, Omit, Restore, and Delete)</a>	110
<a href="#">Editing Records</a>	111
<a href="#">Omitting Records</a>	112
<a href="#">Restoring Records</a>	112
<a href="#">Deleting Records</a>	112
<a href="#">Manually Enter Data</a>	113
<a href="#">The Glucose Tab</a>	117
<a href="#">The Insulin Tab</a>	120
<a href="#">The Oral Meds Tab</a>	122
<a href="#">The Exercise Tab</a>	124
<a href="#">The Food Tab</a>	126
<a href="#">The Health Records Tab</a>	128
<b><a href="#">Part 4: OneTouch® DMS Database Management</a></b>	<b>132</b>
<a href="#">ARCHIVING/RETRIEVING DATA RECORDS</a>	133
<a href="#">EXPORTING/IMPORTING DATA RECORDS</a>	137
<a href="#">BACKING UP DATA</a>	140

## Table of Contents (cont.)

<b>Part 5: Appendices</b>	<b>141</b>
APPENDIX A: MENU BAR AND TOOL BAR COMMANDS	142
Menu Bar Commands	142
Tool Bar Commands	147
APPENDIX B: GLOSSARY	148
APPENDIX C: TROUBLESHOOTING	154
APPENDIX D: TECHNICAL SUPPORT	156
APPENDIX E: INDEX	157

# Part 1: Before You Begin

- Welcome
- About This Manual
- Getting Technical Support
- Changes to the Software
- Navigating OneTouch® DMS
  - Keyboard and Mouse Pointer Conventions
  - OneTouch® DMS Button Behavior

# Part 1: Before You Begin


## Welcome

Welcome to OneTouch® Diabetes Management Software (DMS). This Program is designed to help healthcare professionals and people with diabetes, track and monitor blood glucose levels. OneTouch® DMS can also be used to help plan meals, exercise, and medications. So whether you are a person with diabetes using OneTouch® Software at home, or a healthcare professional using it in your office, OneTouch® DMS can help you manage the information that can help you improve glycemic control.

OneTouch® DMS enables you to do the following:

- Transfer glucose readings and other data from a meter to your PC
- Choose from 11 Report formats to view and print data
- Calculate glucose averages and standard deviations
- Highlight glucose readings that are within, above, and below the target range
- Customize dates, meal slots, target glucose ranges, etc.
- Add data such as insulin doses, exercise duration, carb intake, and health comments
- Print Reports for long-term record keeping

**Figure 1.0**

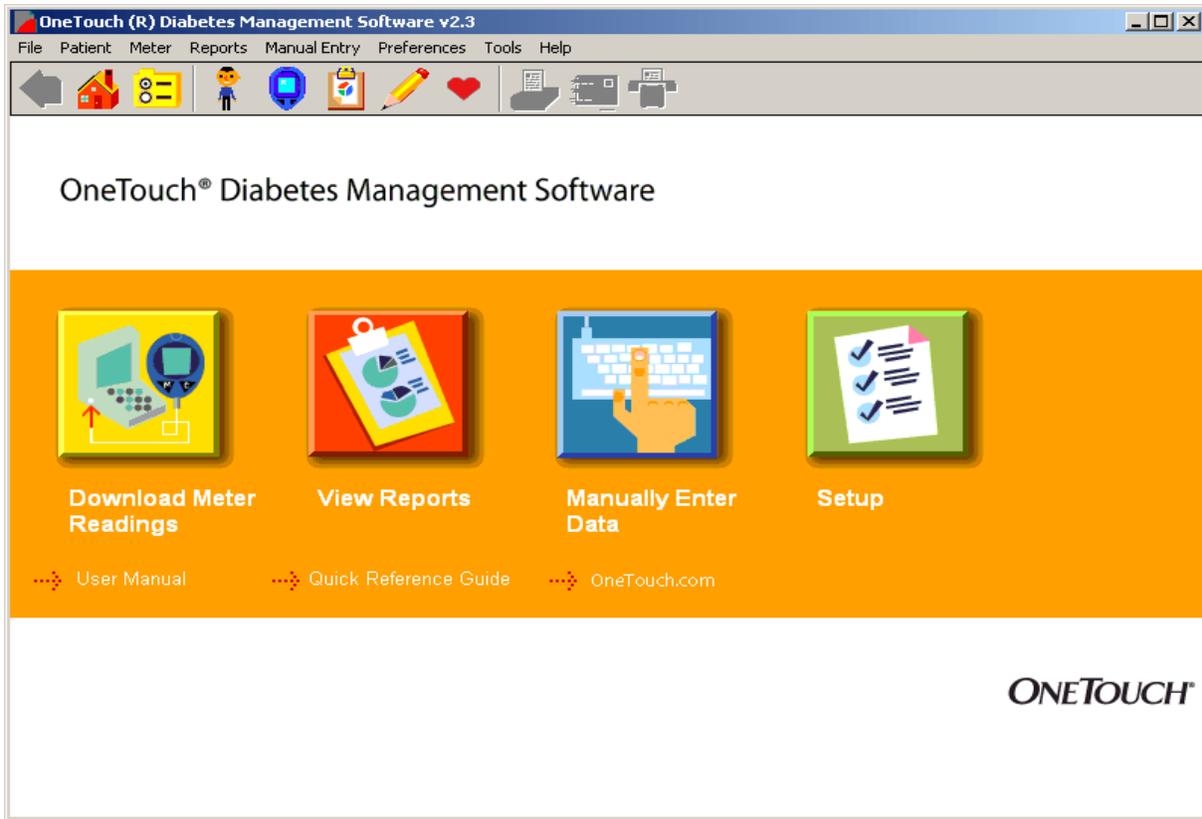

OneTouch® Diabetes Management Software (DMS) is designed for use with OneTouch® Blood Glucose Meters. Please refer to [OneTouch.com](http://OneTouch.com) for a complete list of currently supported devices.

It is assumed that OneTouch® DMS users have a basic understanding of Microsoft® Windows®. Please refer to the Windows® documentation that accompanies your PC for technical assistance.

## About This Manual

The OneTouch® DMS User Manual will explain and guide you in using the various OneTouch® DMS features. A Table of Contents and Index are provided to help you locate topics of interest.

Many sections of the User Manual are formatted in a style similar to the following:

When you see this:	You can do this:
Part 1: Before You Begin	Get a brief introduction to OneTouch® DMS before you start using it.
Part 2: Getting Started	Install OneTouch® DMS and customize it to meet your personal needs.
Part 3: Diabetes Management	Transfer meter readings, manage data, and generate Reports.
Part 4: OneTouch® DMS Database Management	Archive data for storage and export data for use with other applications.
Part 5: Appendices	Learn the OneTouch® DMS Menu, Tool Bar Commands, and terminology. Also, find solutions to problems you may encounter when using OneTouch® DMS.

While using OneTouch® DMS, you can access the User Manual in the following ways:

- From the **User Manual** hyperlink on the Software home screen

or

- From the computer desktop, click on the **START** button (on the lower left of the screen), then select **Programs\OneTouch Software** and click **User Manual**

The User Manual is designed as an electronic PDF file. To view or print a file in PDF format, you need Adobe® Acrobat® Reader 6.0 (or higher), a free application distributed by Adobe® Systems. PDF formats make it easy to display and search for content in an interactive fashion while using OneTouch® DMS. You may also print the entire User Manual or selected sections for future reference. Once the PDF file is opened, the Adobe® Acrobat® Reader screen provides control over the viewing and printing of the User Manual.

PDF files follow a set of standard procedures for navigating through documents. Several options are available for each desired navigational task.

### Searching for keywords or phrases:

- Click on **Edit** from the Menu Bar at the top of the screen. Then select **Search** from the drop-down menu. A pop-up field will appear where you may enter the keyword or phrase you are trying to locate. After searching, the pop-up field will indicate the number of times the keyword or phrase has been found. It will also highlight every found reference in the User Manual. You may search for another keyword or phrase by clicking on **New Search**.

or

- Click on the **Binoculars** icon on the Tool Bar. A pop-up field will appear where you may enter the keyword you are trying to locate. Once the keyword is located, you may search for other mentions by clicking on **New Search**.

## Paging through documents:

You have several options for paging through the User Manual:

- Using the scroll bar on the right
  - Scroll down the document to the desired page.
- Using **View** from the Menu Bar
  - Click on **Go To**. Then click on **First Page/Last Page/Previous Page** or **Next Page** to get to one of these pages, or **Page** to get to a particular page number.
- Using the navigational controls at the bottom of the screen.
  - Click on the left arrows to go to the **First Page** and **Previous Page**. Click on the right arrows to go to the **Next Page** and **Last Page**. To jump to a particular page number, enter that page number where the current page number is displayed and press the **Enter** key.
- Using the **Page Up**, **Page Down**, and **Arrow keys** on your keyboard
  - Press the **Page Up** and **Page Down** keys to scroll one page at a time.
  - Press the **Home** and **End** keys to go the first and last page.
  - Press the up/down **Arrow** keys to scroll one line at a time.

## Getting Technical Support

Technical support is available from LifeScan for registered users of OneTouch® DMS. Contact your local LifeScan office, or visit the Around the World section of the LifeScan website at LifeScan.com for the Technical Support telephone number in your area. Technical Support is able to provide services as long as you do not make any changes to the PC system on which you successfully installed the Software.

## Changes to the Software:

### Features that were added:

- Up to five fax numbers may be stored in OneTouch® DMS for easy retrieval when faxing reports.
- Out-of-target results on black and white printed reports are now easier to read
- Date range options that make it easier to display meter data that may have been stored with an incorrect date (e.g., a date in the future).
- The ability to transfer and display data from new devices, including non-OneTouch® branded meter devices (once communication has been established with the new device).
- The option to display glucose results in certain Data List Reports in the order that tests were taken, rather than by the meter calendar date that was automatically saved with each result.
- All records in the Before-Breakfast through Night meal slot series will be displayed with the date of the Before-Breakfast start time, including any records that may occur after midnight.

### Features that were eliminated:

- The option to indicate if the Software is used in a Clinic or Home setting.
- The ability to change the meal slot tag on glucose records that were transferred from meter.

## Navigating OneTouch® DMS

### Keyboard and Mouse Pointer Conventions

OneTouch® DMS is designed for users who know how to use a PC mouse for “point-and-click” operation. Your mouse and keyboard are all you need for OneTouch® DMS navigation.

OneTouch® DMS uses standard Microsoft® Windows® keyboard, mouse pointer, and drop-down menu conventions:

If you do this with your mouse and keyboard:	You can do this:
Point to a <b>OneTouch</b> screen icon with the mouse	Get more information about a particular function.
Point to data values with the mouse while viewing Reports	Get more information that has been tagged to a data value.
Click the mouse after pointing	Make a selection.
Double-click the mouse after pointing	Highlight a word.
Click the mouse while pressing the Shift key	Make multiple selections that are next to one another.
Click the mouse while pressing the CTRL key	Make multiple selections that are not next to one another.

For many OneTouch® DMS tasks, you have several options to achieve the same result. For example, to transfer meter readings you can:





- Click on the **Download Meter Readings** icon on the home screen, or
- Select **Get Meter Readings** from the Meter menu, or
- Click on the **Meter** icon on the Tool Bar.

## OneTouch® DMS Button Behavior





The following Command buttons appear on several OneTouch® DMS screens. Their actions are common to all screens unless otherwise noted. A button is enabled (active) if the label on the button is in **black font**. A button is disabled (not active) if the label on the button is in gray font.

When you click on this:	You can do this:
<p style="text-align: center;">Add new patient</p>	<p>Add a new patient name and profile to OneTouch® DMS database. After clicking on <b>Add New Patient</b>, you will go to the General Tab of the Patient Setup screen to begin entering information. If you just created a new database, only the <i>Guest</i> patient record exists. Before you can transfer readings from a meter to a patient other than <i>Guest</i>, you will have to add that patient name to the OneTouch® DMS database.</p>
<p style="text-align: center;">Add new record</p>	<p>Manually add a new data record to the database. After clicking on <b>Add New Record</b>, the Software will remain at the current screen where you can continue to add data records.</p> <p>If you are modifying a data record via the Report screens, you may add other types of data records by clicking on the <b>Add New Record</b> button. This will activate the other Tab selections on the Manual Entry-A screen.</p>
<p style="text-align: center;">Apply</p>	<p>Display a new report after updating patient name, date range, or Report format.</p>
<p style="text-align: center;">Archive</p>	<p>Copy one or more database records to a separate file for storage. Archiving lets you manage the size of the OneTouch® DMS database by removing older data that you may no longer need to review. This way you can keep the database size at a minimum so the OneTouch® DMS works most efficiently.</p>

**OneTouch® DMS Button Behavior (Cont.)**

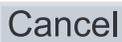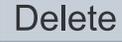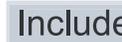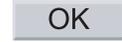When you click on this:	You can do this:
	Close the current screen without saving your data entries. After clicking on <b>Cancel</b> , OneTouch® DMS will then take you back to the previous screen. <b>Cancel</b> is always enabled (active).
	Delete a data record for a particular patient. <b>Only manually entered data records can be deleted from the OneTouch® DMS database.</b> The record will be permanently deleted from the database. <b>Delete</b> can also be used to delete a patient (and all of the patient's records) from the OneTouch® DMS database.
	Save your data entries for the current screen. After you click on <b>Done</b> , the screen will close and OneTouch® DMS will take you back to the previous screen.
	Copy one or more database records to a comma separated value (CSV) file, a common Excel file format, for use with other PC applications.
	Complete the desired Program function and continue to the next screen.
	Place exported data back in the OneTouch® DMS database.
	"Restore" an omitted record to the database.
	Complete the desired Program function.

**OneTouch® DMS Button Behavior (Cont.)**

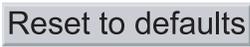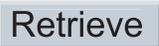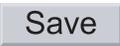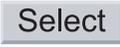When you click on this:	You can do this:
	Mark a data record for “omission” from calculations and Reports. The record remains in the database and may be re-included for future analyses with the <b>Include</b> button.
	Cancel any data entries from the current screen and restore the settings to the default values.
	Restore one or more archived data records to the database.
	Save data entries for the current screen. After you click on <b>Save</b> , the Software will remain at the current screen. This button lets you make content changes on several tabs and save them all at once. <b>Save</b> is only enabled (active) once you begin to add or modify data. To close the current screen, you must click on <b>Done</b> or <b>Cancel</b> .
	Choose the patient whose name is highlighted on the <b>Find a Patient</b> Tab of the Patient Setup screen. If you click on <b>Select</b> , that patient name will replace the Name in the field.

## OneTouch® DMS Links

Text hyperlinks appear on many OneTouch® DMS screens. Hyperlinks provide a fast and easy way to go to other functional areas of the OneTouch® DMS. Text hyperlinks (also known simply as links) are visible on OneTouch® DMS screens as dashed arrows followed by text. By clicking on the text portion, you will go to that functional part of the Software.

Some example text links include:

- > Add a Patient
- > Add Insulin
- > Add Medication
- > View/edit targets
- > Find a Patient
- > View Footnotes
- > View Statistics and Targets
- > Tell me more

OneTouch® DMS also utilizes certain data values (numbers, dates, times, etc.) and icons as links. They appear when viewing Reports. You may click on these links to go to a greater level of detail in the Report, or to go to the Manual Entry screens where you can modify or add data records. You can easily identify a data value link as any number or text that is underlined.

## Part 2: Getting Started

- Installing/Uninstalling/Upgrading OneTouch® DMS on your PC
- Starting Up OneTouch® DMS
- Customizing OneTouch® DMS for Personal Use
  - Patient Setup
    - General Tab
    - Schedule Tab
    - Medications Tab
    - Regimen Tab
    - Complications Tab
    - Find a Patient Tab
  - Preferences
    - General Tab
      - Displaying Units of Measure
      - Displaying Results as Plasma Values or Whole Blood Values
    - Report Options Tab
    - Global Targets Tab
    - Hypoglycemic Tab
  - Modify Meter Settings
    - General Tab
    - Clock Tab
    - Schedule Tab
    - Insulin Tab
    - Oral Meds Tab
    - Targets and Hypoglycemic Tab

## Part 2: Getting Started

### Installing OneTouch® DMS on Your PC

You can use these installation instructions if you:

- Are a new OneTouch® DMS user, or
- Are upgrading from InTouch® Software, or a previous version of OneTouch® DMS in English to this version of OneTouch® DMS, also in English, or
- Wish to uninstall OneTouch® DMS

#### Notes:

- **Do not use these instructions to upgrade to OneTouch® DMS in a new language.** For example, if you have OneTouch® DMS v2.1 in Spanish already installed on your system and wish to install this version of OneTouch® DMS in English, be sure to archive your Spanish database (OneTouch.mdb from C:\Program Files\OneTouch\Data) in a new folder, uninstall the Spanish v2.1 and delete the C:\Program Files\OneTouch folder before installing this version in English. Refer to the Archive topic in the User Manual for information on archiving your data.
- Do not use these instructions to upgrade from OneTouch® DMS Pro. OneTouch® DMS is not compatible with databases created with OneTouch® DMS Pro.

There are two steps you must complete before OneTouch® DMS will work on your PC. The first step is loading the software onto the PC. The second step is linking the software to a patient database.

## 1. To install OneTouch® DMS.

a. Download the software from OneTouch.com and follow the InstallShield® Wizard instructions for installing the Software Program on your PC.

### Note:

At the end of the installation, you should notice a **OneTouch** icon on your desktop.

## b. Country Selection

- Select the country where you purchase your test strips from the drop-down menu. It is important to select the right country to ensure that your test results are presented correctly, because the calibration reference varies from one country to another.
- For users in countries not included on this list, OneTouch® DMS offers an “Other” installation option. This option requires you to select your glucose value reporting options.
- Different glucose value reporting options should be selected based on the OneTouch® Test Strips and meters used in your country. Select the reporting options that match your OneTouch® Test Strip product information.



The screenshot shows two columns of radio button options. The first column is labeled 'Glucose shown as:' and contains 'Whole Blood' and 'Plasma'. The second column is labeled 'Glucose unit:' and contains 'mg/dL' and 'mmol/L'. The 'Plasma' and 'mg/dL' options are selected.

Glucose shown as:	Glucose unit:
<input type="radio"/> Whole Blood	<input checked="" type="radio"/> mg/dL
<input checked="" type="radio"/> Plasma	<input type="radio"/> mmol/L

## 2. To connect to a database.


OneTouch® DMS uses a database file to manage information. If you are not upgrading from a previous version of OneTouch® DMS, in order to begin, you must identify the correct file to use. You have three choices:

- OPTION 1 - Start fresh by creating a new file to store data.
- OPTION 2 - Use data from an old version of InTouch® Software.

- OPTION 3 - Link to an existing OneTouch® DMS file.

**Notes:**

- Please choose the required option with care. Choosing the wrong option may lead to inappropriate results.
- You will not have to repeat these steps once you have either created a new database or linked to an existing database. OneTouch® DMS will automatically link to that database every time you use the Software. If you reinstall OneTouch® DMS, you will have to re-establish the correct database link.
- OneTouch® DMS is not compatible with databases created with OneTouch® DMS Pro. Do not use the instructions above to connect with a OneTouch® DMS Pro database.

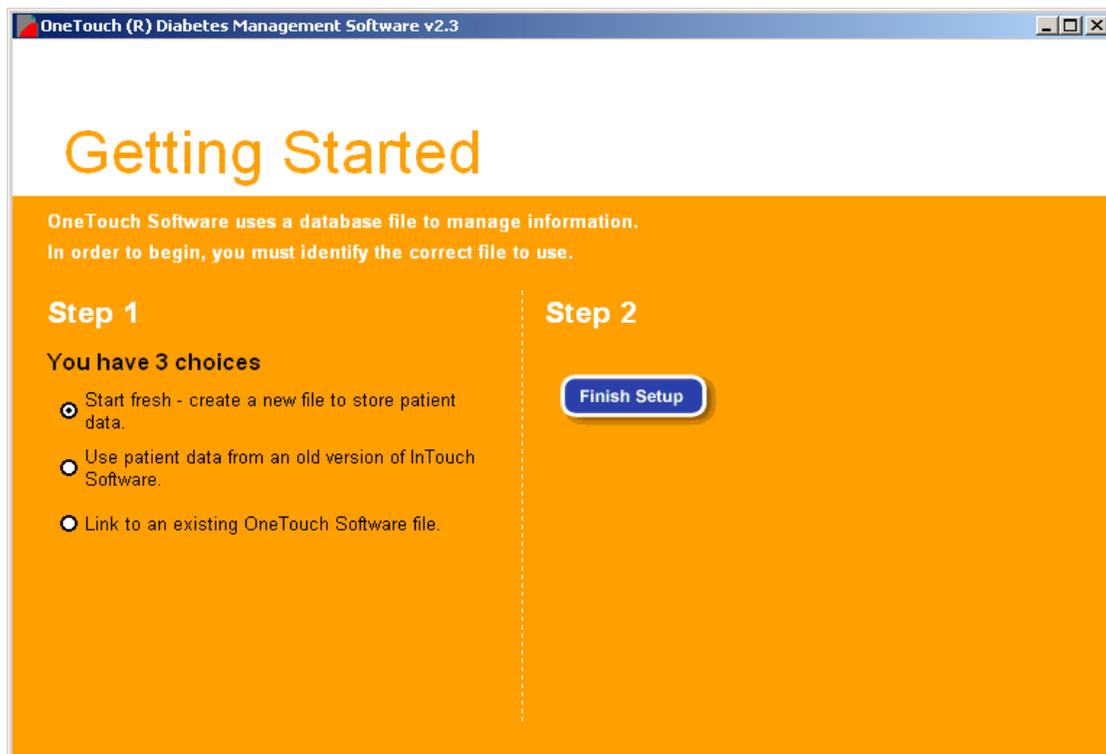**Figure 2.0**

If you:	And you want to:	You should do this:
Are using OneTouch® DMS for the first time	Create a new, blank database.	Select <b>Start fresh - create a new file to store patient data</b>  This is the default setting.
Are using OneTouch® DMS for the first time	Link to a database created with an earlier version of InTouch® Software	Select <b>Use patient data from an older version of InTouch Software</b>
Are reinstalling or upgrading OneTouch® DMS	Link to an existing database created with OneTouch® DMS.	Select <b>Link to an existing OneTouch Software file</b>

**Note:** Linking OneTouch® DMS to an existing database may take several minutes. This is due to the complexity of converting data to the new database format. It's also possible that there may not be enough disk space to perform a conversion from an earlier InTouch® Software database. In this case you will be notified and the conversion will not proceed. A message will prompt you to free up disk space and will indicate approximately how much space is needed.

## Uninstalling OneTouch® DMS from your PC

To uninstall OneTouch® DMS, you have 2 options. If you wish to save OneTouch® DMS database, be sure to move the database file to a safe storage area on your PC before you begin. The file is named **OneTouch.mdb** and it is stored in the **\LifeScan\OneTouch\Data** folder in the **C:\Program Files** directory (this is the default directory). Please also see the *OneTouch® DMS Database Management* section of the User Manual for information on archiving and exporting data.

1. Click on **Start\Programs\OneTouch Software**. Then select the **Uninstall** option. Follow the instructions for removing OneTouch® DMS from your PC.

or

2. Click on **Start\Settings\Control Panel\and Add/Remove Programs**. Follow the instructions for removing OneTouch® DMS from your PC.


## Starting Up OneTouch® DMS


Once OneTouch® DMS has been installed on your PC, and you have linked to a database, you are ready to begin. A **OneTouch** icon should appear on your PC Desktop.

If you:	You can do this:
See a <b>OneTouch</b> icon on your PC Desktop	Double-click on it.
Don't see a <b>OneTouch</b> icon on your PC Desktop	Click on <b>Start\Programs\OneTouch Software\OneTouch Software</b>

OneTouch® DMS opens up to the Program home screen (see [Figure 2.1](#)). The home screen contains the four main program functions. They are labeled **Download Meter Readings**, **View Reports**, **Manually Enter Data**, and **Setup**. Clicking on any of the four icons will activate the desired function. A description of each function follows:

**Figure 2.1**

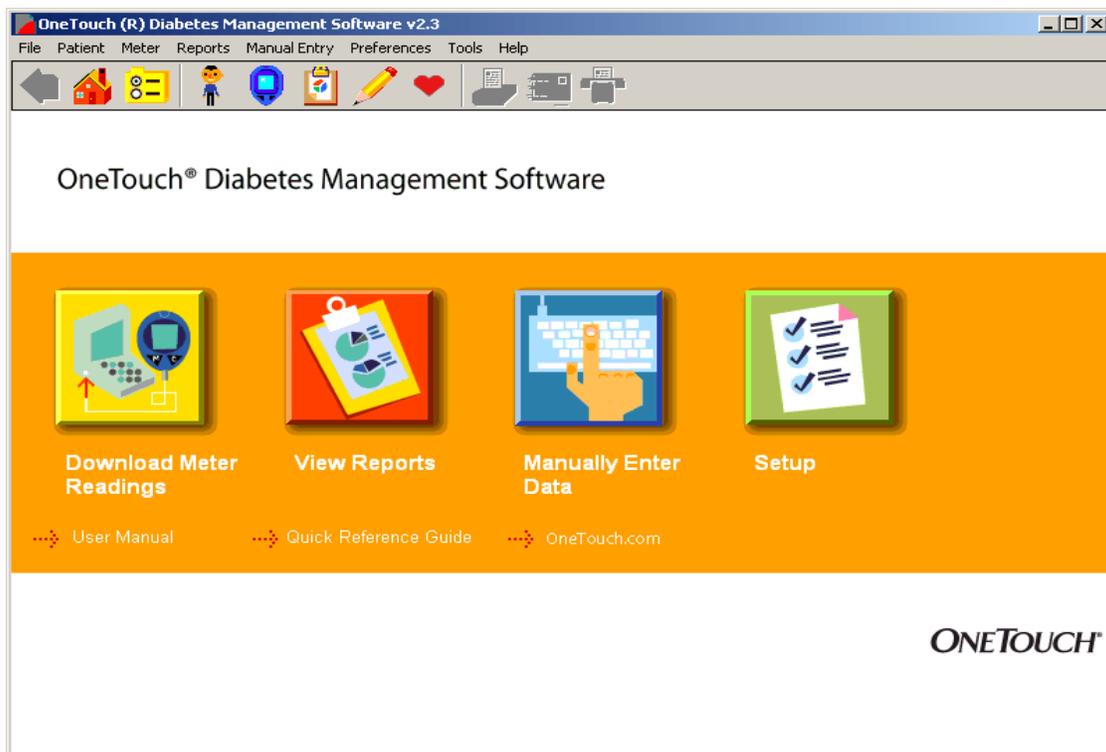

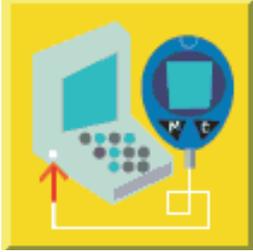When you see this:	You can do this:
<p>Download Meter Readings</p> 	<p>Transfer readings from a meter to OneTouch® DMS</p>
<p>View Reports</p> 	<p>View and print up to 11 different program Reports</p>
<p>Manually Enter Data</p> 	<p>Add data records or Comments to the OneTouch® DMS database. These records will supplement readings transferred from a meter.</p>
<p>Setup</p> 	<p>Customize OneTouch® DMS for personal use, create patient profiles, and modify meter settings</p>

The following links appear under the 4 icons:

→ **User Manual**

→ **Quick Reference Guide**

→ [OneTouch.com](#)


Clicking on any of these links will take you to that content area.

## Customizing OneTouch® DMS for Personal Use

The Setup main screen lets you customize OneTouch® DMS to meet your needs as a person with diabetes or a healthcare professional.

The Setup main screen contains three program functions ([see Figure 2.2](#)). They are labeled **Patient Setup**, **Preferences**, and **Meter Settings**. Clicking on any of the three icons will activate the desired function. A description of each function follows:

**Figure 2.2**

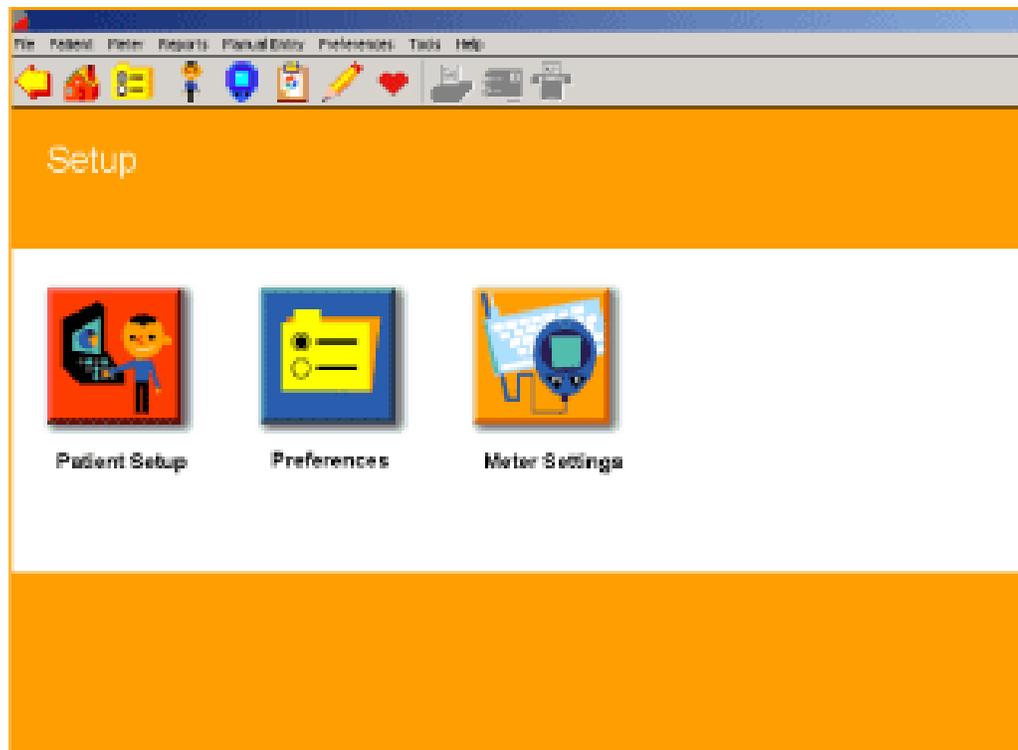

When you see this:	You can do this:
<p data-bbox="204 443 412 478">Patient Setup</p> 	<p data-bbox="683 443 1344 520">Set up diabetes profile data for you or your patients</p>
<p data-bbox="212 777 391 812">Preferences</p> 	<p data-bbox="683 777 1344 898">Customize the Software environment to meet your needs as a patient or healthcare professional</p>
<p data-bbox="204 1104 431 1140">Meter Settings</p> 	<p data-bbox="683 1104 1344 1182">Modify meter settings that will impact how data is stored and displayed in the meter</p>

## Patient Setup


The Patient Setup screen lets you create or delete diabetes profile data for yourself or your patients.

The Patient Setup screen contains six Tabs ([see Figure 2.3](#)). They are labeled **General**, **Schedule**, **Medications**, **Regimen**, **Complications**, and **Find a Patient**. The first 5 Tabs let you enter diabetes and other health-related profile data. **Find a Patient** lets you easily locate patient names in the database. Clicking on any of the six Tabs will activate the desired function.

A Patient Name will always appear in the Patient field. Most Program functions affect the Patient whose name appears in this box. If this is not the patient you want, you may select a different one by clicking on the down arrow. A drop-down menu of Patient Names will appear. Select the one you want. You may also click on the **Find a Patient** link. This will bring you to the **Find a Patient** Tab where you have several ways to locate a patient in the database.

If you are downloading data for a new patient you will need to set up a new patient profile. Click on the **Add a Patient** link to add a new patient to the database. A description of each Tab function follows below.

**Figure 2.3**

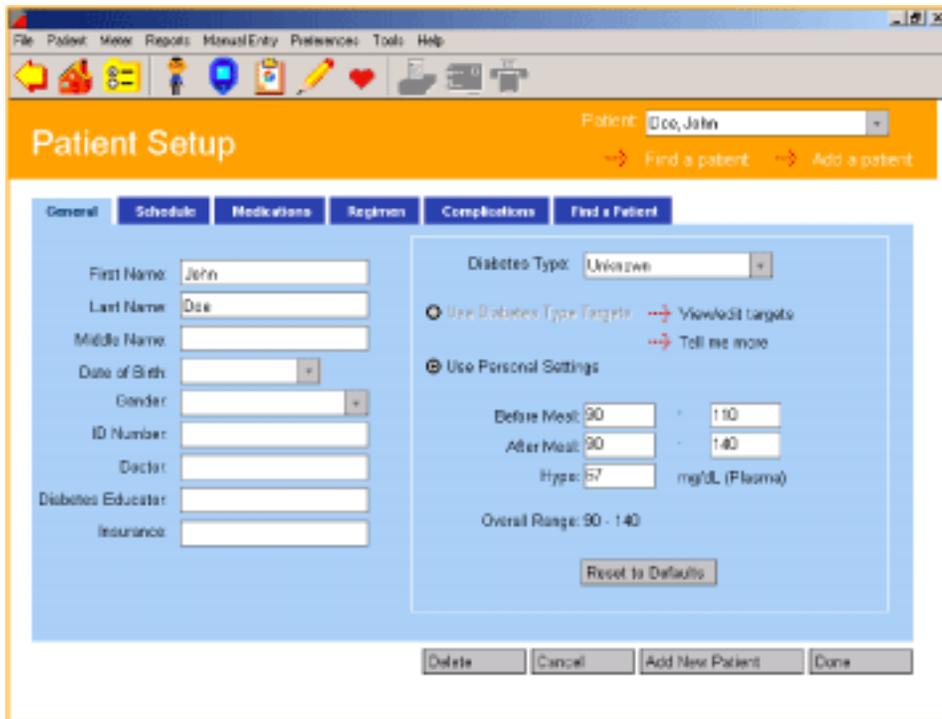


## General Tab

The General Tab lets you enter and view personal profile data ([see Figure 2.4](#)).

**Figure 2.4**

The screenshot displays the 'Patient Setup' window. At the top, the menu bar includes 'File', 'Patient', 'Menu', 'Reports', 'Manual Entry', 'Preferences', 'Tools', and 'Help'. Below the menu is a toolbar with various icons. The main area is titled 'Patient Setup' and shows a dropdown menu for 'Patient' with 'Doe, John' selected. Below this are buttons for 'Find a patient' and 'Add a patient'. The 'General' tab is active, showing fields for 'First Name' (John), 'Last Name' (Doe), 'Middle Name', 'Date of Birth', 'Gender', 'ID Number', 'Doctor', 'Diabetes Educator', and 'Insurance'. To the right, there are settings for 'Diabetes Type' (Unknown), 'Use Diabetes Type Targets' (checked), and 'Use Personal Settings' (checked). Below these are input fields for 'Before Meal' (90), 'After Meal' (90), and 'Fypc' (57) mg/dL (Plasma). The 'Overall Range' is shown as 90 - 140. A 'Reset to Defaults' button is located below the range. At the bottom of the window are buttons for 'Delete', 'Cancel', 'Add New Patient', and 'Done'.

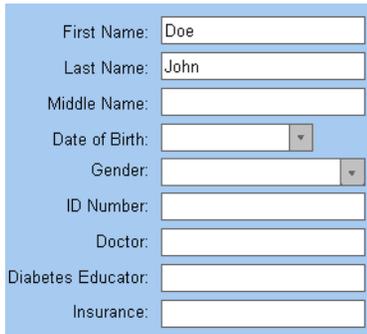**When you see this:****You can do this:****Patient Data Entry Fields**A screenshot of a patient data entry form. The form is light blue and contains several input fields. The fields are: First Name (text input with 'Doe'), Last Name (text input with 'John'), Middle Name (text input), Date of Birth (calendar icon), Gender (dropdown menu), ID Number (text input), Doctor (text input), Diabetes Educator (text input), and Insurance (text input).

Create a patient profile by entering Last, Middle, and First Names, Date of Birth, Gender, ID Number, Doctor and Diabetes Educator Names, and Insurance Carrier.

To specify Date of Birth, modify the information in the Date field or click on the down arrow. A pop-up calendar box will appear up with January 1, 1900 set as the default Date of Birth.

To change the year, click on the year and a set of up/down arrows will appear. Click on the up/down arrows until you find the desired year. To change the month, click on the left/right arrows that appear on the sides of the calendar box until you find the desired month. Then click on the desired day.

You may specify Diabetes Type by clicking on the down arrow in the Diabetes Type field. A drop-down menu of Diabetes Types will appear. Select the one you want. You may choose from the following Diabetes Type options:

- Unknown
- Type 1
- Type 2

“Unknown” is the default setting. You can also select Gestational, Custom 1, Custom 2 or Custom 3 if these target types are defined in the Preferences Global Targets screen.

When you see this:	You can do this:
Target Ranges	<p>Set a glucose target range for this patient for Before-Meal and After-Meal periods. Target ranges may be specified by Diabetes Type (select <b>Use Diabetes Type Targets</b>) or customized for each patient (select <b>Use Personal Settings</b>).</p> <p>If you specify <b>Use Diabetes Type Targets</b>, the patient will receive the target ranges defined in the Global Targets Tab of the Preferences screen. If you would like to view or modify the target settings that have been set for this Diabetes Type, click on the <b>View/edit targets</b> link. This will take you to the Global Targets Tab where you can make changes as needed. Default settings for Global Targets exist only for Type 1 and Type 2 Diabetes.</p> <p>If you have specified the Diabetes Type as “Unknown,” by default <b>Use Personal Settings</b> will be selected. Here you will need to set target ranges for this patient. Note that you can also select <b>Use Personal Settings</b> for Type 1, Type 2 or any other Diabetes Type.</p> <p>General rules for entering glucose target ranges are as follows:</p> <ul style="list-style-type: none"> <li>■ Enter the lower limit for each Meal Slot in the <b>From</b> field on the left. Enter the higher limit for each Meal Slot in the <b>To</b> field on the right.</li> <li>■ Each upper limit must be greater than or equal to its corresponding lower limit</li> <li>■ Before-Meal and After-Meal target ranges can overlap</li> <li>■ You must specify both Before-Meal and After-Meal target ranges</li> </ul> <p>OneTouch® DMS calculates an Overall (Target) Range that is used in OneTouch® DMS Reports. The Overall (Target) Range extends from the lowest range limit to the highest range limit. If you are using Personal Settings, you can customize the patient's hypoglycemic threshold value.</p>

## Schedule Tab


OneTouch® DMS uses meal slots to organize data for Reports. The Schedule tab of the Patient Setup screen lets you customize the start and end time for 7 pre-defined meal slots for both working and non-working days (see Figures [2.5a](#) and [2.5b](#)). This way you can track and monitor your glucose levels and medications for specific time periods throughout the day, customized to your daily routine.

Start and end times may be set for the following meal slots:


- Before-Breakfast
- After-Breakfast
- Before-Lunch
- After-Lunch
- Before-Dinner
- After-Dinner
- Night

**Note:** To ensure that glucose readings appear in OneTouch® DMS Reports in the same meal slot that was used to store the readings in the meter, it is important that the meal schedule you set up in OneTouch® DMS matches the meal schedule stored in the meter. If the schedules don't match, glucose readings that are transferred to OneTouch® DMS may be displayed in an inappropriate meal slot. If any out-of-target readings appear in the wrong meal slot, this may lead you to misinterpret information when tracking and trending data.

**Figure 2.5a**

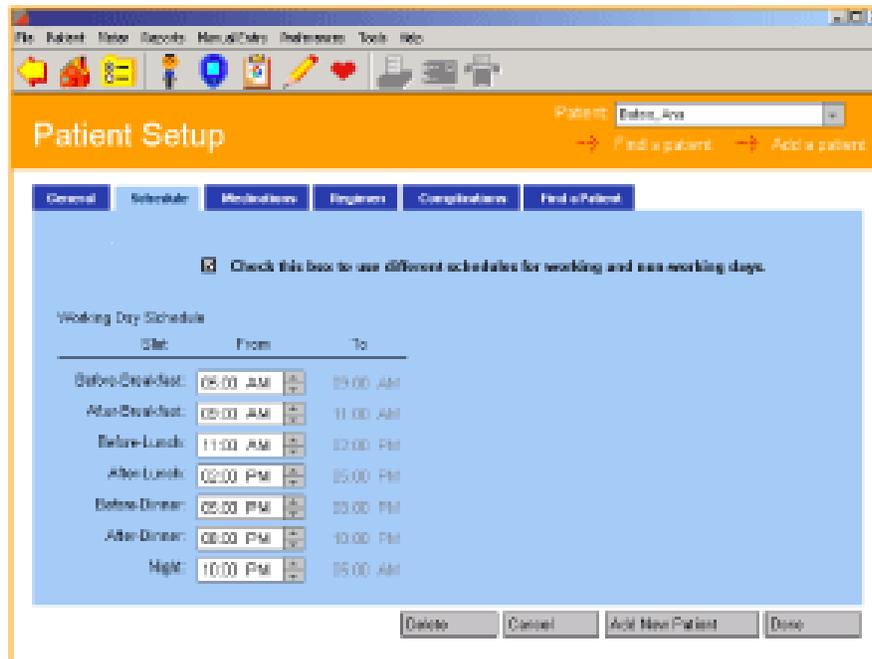

**Figure 2.5b**

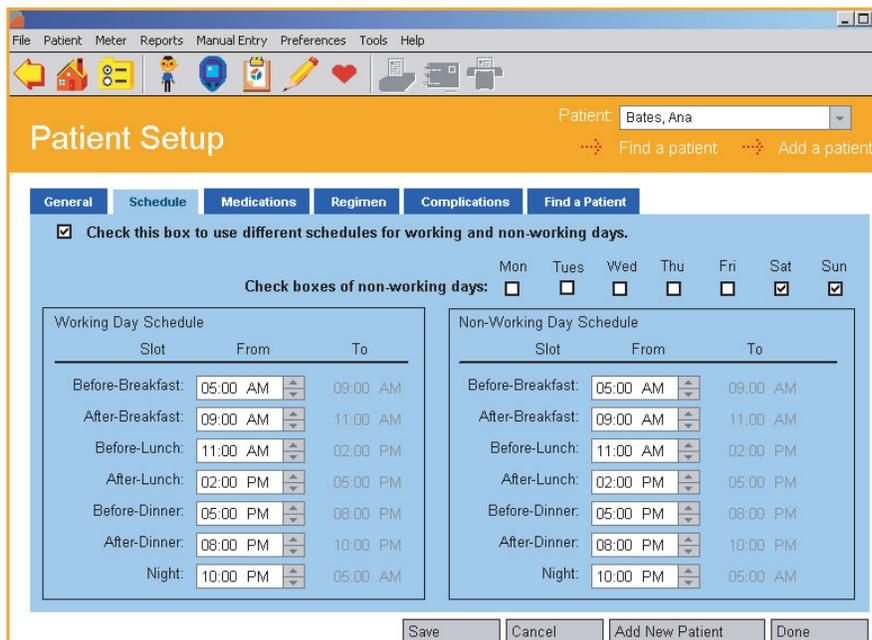

**Note:** All records in the Before-Breakfast through Night meal slot series will be displayed sequentially with the date of the Before-Breakfast start time. This includes records that may occur after midnight.

## To set meal slot times

### When you see this:

Slot	From	To
Before-Breakfast:	05:00 AM	09:00 AM
After-Breakfast:	09:00 AM	11:00 AM
Before-Lunch:	11:00 AM	02:00 PM
After-Lunch:	02:00 PM	05:00 PM
Before-Dinner:	05:00 PM	08:00 PM
After-Dinner:	08:00 PM	10:00 PM
Night:	10:00 PM	05:00 AM

### You can do this:

**Set meal slot times.** Review the meal slot times and decide if they match your schedule. OneTouch® DMS uses these meal slot times to organize data for Reports.

**Each meal slot start time begins when the previous one ends.**

**Change the From** time of one meal slot, and OneTouch® DMS automatically changes the **To** time of the previous meal slot.

**Change the hour** for any meal slot, by selecting the hour value that appears in the **From** field.

**Click** on the up/down arrows until you find the desired hour.

**Change the minutes** for any meal slot, by selecting the minutes value that appears in the **From** field.

**Click** on the up/down arrows until you find the desired minutes. All start times should be rounded to the nearest quarter hour.



## To specify different meal slots for working and non-working days:

OneTouch® DMS lets you set different meal slot schedules for working and non-working days. To use this option:

1. Select **Check this box to use different schedules for working and non-working days** at the top of the application screen.

**Check this box to use different schedules for working and non-working days.**

2. When the above check box is selected, a second option will appear: **Check boxes of non-working days**.

Check boxes of non-working days: Mon  Tues  Wed  Thu  Fri  Sat  Sun

3. A separate set of **From** and **To** time boxes will appear for each meal slot for the non-working days. Follow the same steps as above for changing meal slot times.

The screenshot shows the 'Patient Setup' window for 'Bates, Ana'. The 'Schedule' tab is selected. A checkbox is checked: 'Check this box to use different schedules for working and non-working days.' Below this, 'Check boxes of non-working days' are shown for Mon, Tues, Wed, Thu, Fri, Sat, and Sun, with Sat and Sun checked. Two tables are visible: 'Working Day Schedule' and 'Non-Working Day Schedule', each with columns for Slot, From, and To times.

Slot	From	To
Before-Breakfast:	05:00 AM	09:00 AM
After-Breakfast:	09:00 AM	11:00 AM
Before-Lunch:	11:00 AM	02:00 PM
After-Lunch:	02:00 PM	05:00 PM
Before-Dinner:	05:00 PM	08:00 PM
After-Dinner:	08:00 PM	10:00 PM
Night:	10:00 PM	05:00 AM

Slot	From	To
Before-Breakfast:	05:00 AM	09:00 AM
After-Breakfast:	09:00 AM	11:00 AM
Before-Lunch:	11:00 AM	02:00 PM
After-Lunch:	02:00 PM	05:00 PM
Before-Dinner:	05:00 PM	08:00 PM
After-Dinner:	08:00 PM	10:00 PM
Night:	10:00 PM	05:00 AM


## Using the schedule to assign meter readings to time slots:

When readings are transferred from the meter to OneTouch® DMS, they are tagged with a time stamp and are automatically assigned time slots. If you decide to modify meal slot times, you will impact only those readings that have not yet been transferred. Readings already in OneTouch® DMS will maintain their existing meal slot designation. For this reason, it is important to note if schedule times are appropriate before you transfer readings. It is also important to synchronize your meter clock with that of the PC (assuming the PC clock is correct), to ensure that future readings are slotted correctly. OneTouch® DMS will notify you if your PC and meter clocks are out of sync by more than 15 minutes.



## Using comments/events to assign meter readings to time slots:

Depending on meter type, some OneTouch® Meters allow you to tag a reading with a meal slot comment. Such readings will be assigned to the same meal slot by the OneTouch® DMS application, even if they are taken at a time outside the meal slot range. For example, a reading with a "Before- Breakfast" comment/event would be assigned to the Before-Breakfast time slot even if it were taken outside that time slot. This feature applies only to the OneTouch® UltraSmart® and OneTouch® Profile® Meters. For all other OneTouch® Meters, readings are assigned to time slots using the actual time of the reading.

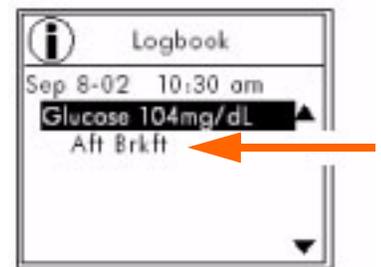


## Medications Tab

The Medications Tab lets you specify any medications (insulin or other) that are being used to manage diabetes ([see Figure 2.6](#)).

You may select any of the following categories:

- Insulin
- Oral Meds
- Insulin Pump
- Diet and Exercise

**Figure 2.6**

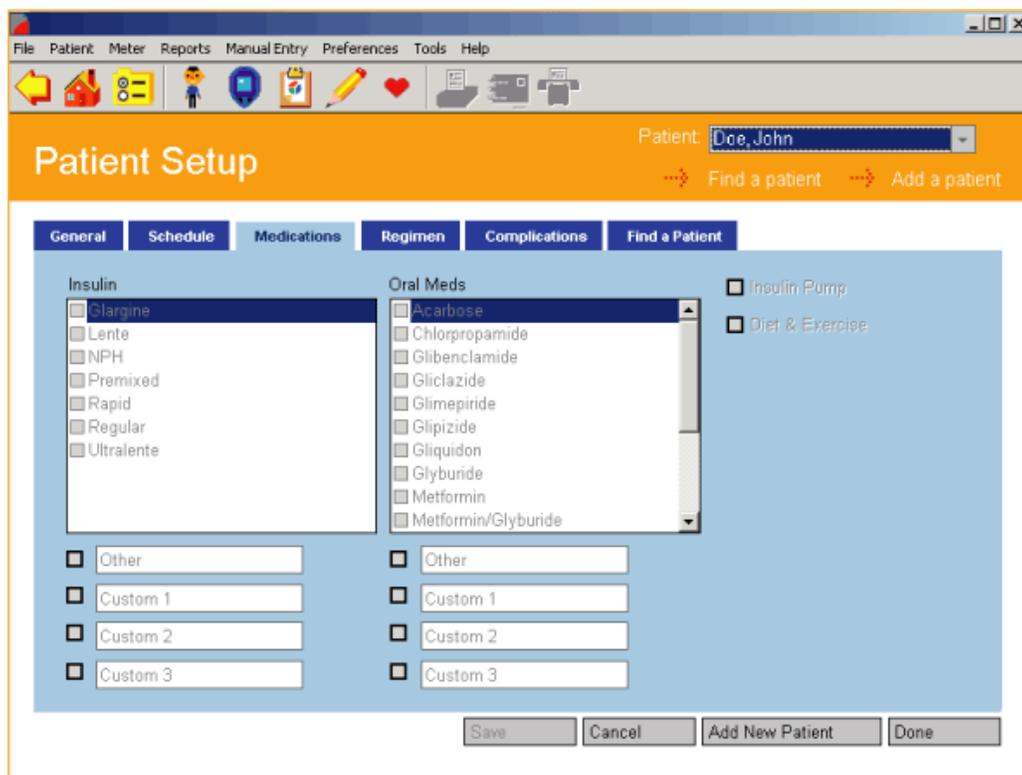

**Medications Tab (Cont.)**


<b>If you see this:</b>	<b>You can do this:</b>
Insulin	<p>Specify Insulin Types from the list that appears on screen. Check each of the Insulin Types that apply. You may also specify three custom Insulin Types by selecting any of the Custom selections below the list. You may create new names for any of the default Custom names.</p> <p>You may also specify one other Insulin Type by clicking on <b>Other</b> and entering a name.</p>
Oral Meds	<p>Specify Oral Meds from the list that appears on screen. You may also specify 3 custom Medications by clicking on any of the Custom selection boxes below the list. You may create new names for any of the default Custom names.</p> <p>You may also specify one other Medication by clicking on <b>Other</b> and entering a name.</p>
Diet and Exercise	Indicate that the patient is using Diet and Exercise to manage diabetes.
Insulin Pump	Indicate that the patient is using an Insulin Pump.

## Regimen Tab

The Regimen Tab lets you specify a daily schedule (Regimen) for insulin injections ([see Figure 2.7](#)).

Up to 5 different Regimens may be created and stored in the database.

**Figure 2.7**

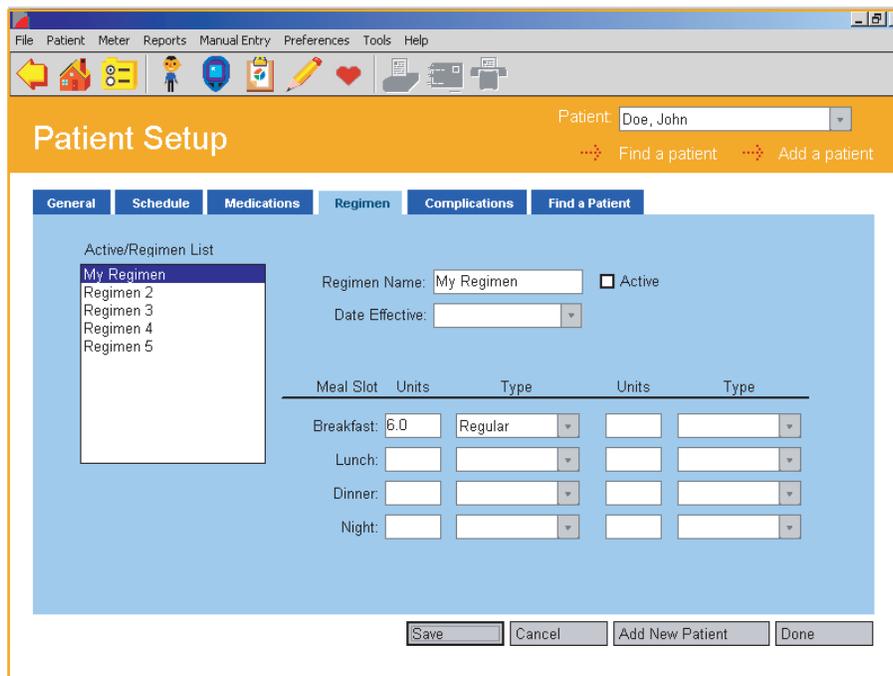


When you see this:	You can do this:
Active/Regimen List	Create up to 5 Regimens for any patient. Regimens will be stored in the database. To define a Regimen, select one of the 5 default Regimen names from the list on the left side of the screen. This Regimen name will replace the one in the Regimen Name field on the right side of the screen.
Regimen Name/Active	<p>Create new names for any of the five default Regimen names.</p> <p>To make any of the five Regimens the Active Regimen, click on <b>Active</b>. The word (Active) will then appear next to that Regimen in the list to the left.</p>
Date Effective	<p>Enter an effective start date and daily schedule for insulin injections. To specify the date, modify the information in the date field or click on the down arrow. A pop-up calendar box will appear with the current date set as the default Date.</p> <p>To change the year, click on the year and a set of up/down arrows will appear. Click on the up/down arrows until you find the desired year. To change the month, click on the left/right arrows that appear on the sides of the calendar box until you find the desired month. Then click on the desired day.</p>
Units and Type	For each Active Regimen, at least 1 insulin type and dose must be specified for at least 1 Meal Slot. You may enter up to 2 insulin doses for each Meal Slot.

## Complications Tab

The Complications Tab lets you specify medical complications related to diabetes (see [Figure 2.8](#)).

**Figure 2.8**

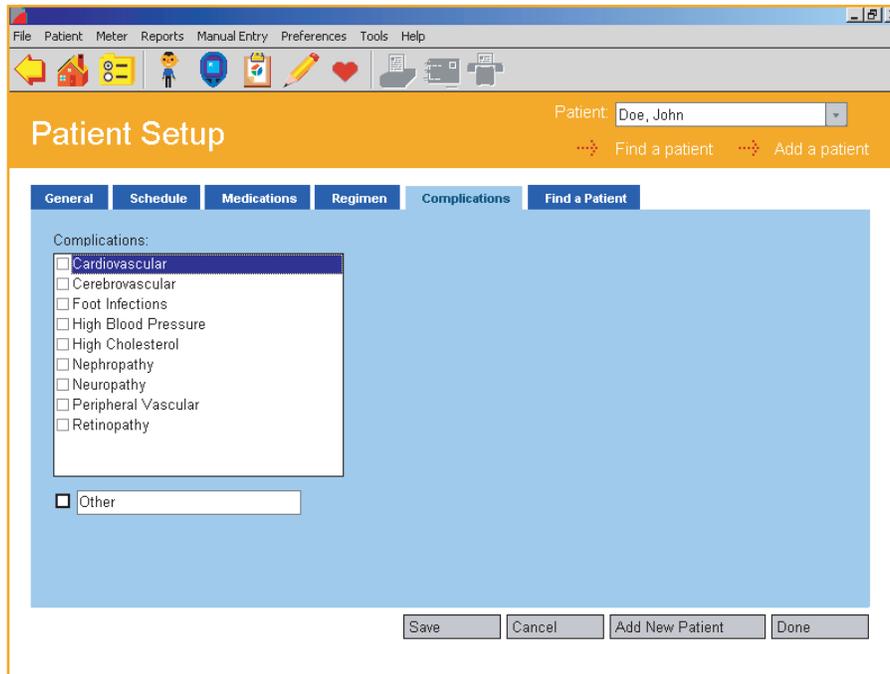


When you see this:	You can do this:
Complications	<p>Specify all diabetes-related Complication Types from the list that appears on screen. Click on all Complication Types that apply.</p> <p>You may also specify one other Complication Type by clicking on <b>Other</b> and entering a name.</p>

## Find a Patient Tab

The Find a Patient Tab lets you locate a Patient Name in the database (see Figure 2.9). This Patient Name will appear on all screens and Reports.

**Figure 2.9**

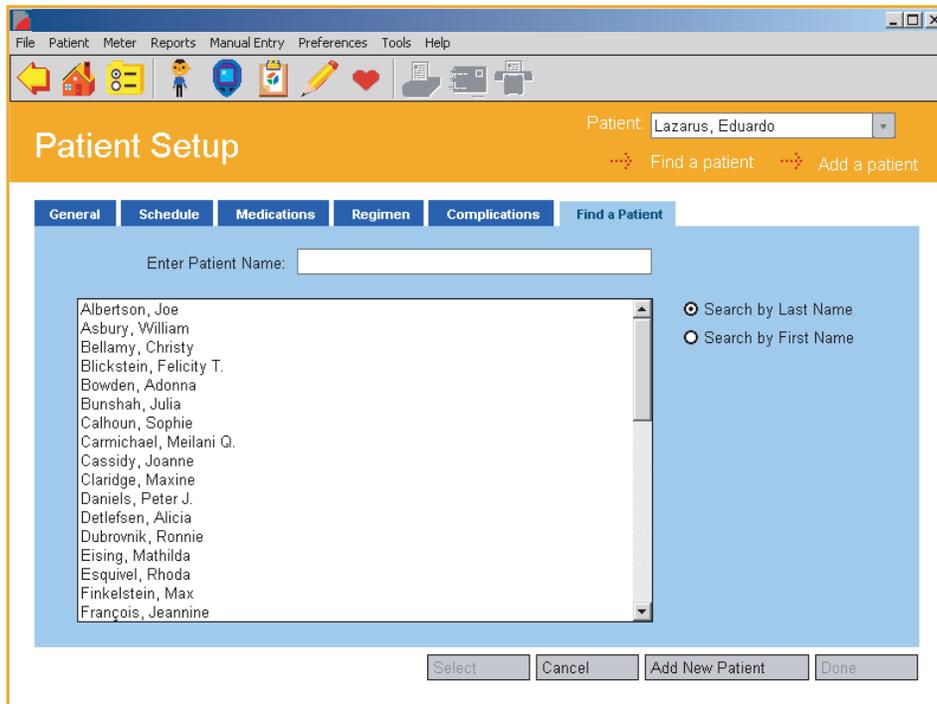

If the Patient Name you are looking for is not displayed in the Patient field, you have 3 options to find it:


1. Click on the down arrow in the Patient field. A drop-down menu of Patient Names will appear. Select the desired name. This name will then appear in the Patient field.
2. Scroll down the list of names in the large field. When you find the name you are looking for, select it. This name will then appear in the Enter Patient Name field.
3. Search for a Patient by typing in the Patient Name. First, select the option to search by First Name or Last Name. The default option is to search by Last Name. Begin entering the First or Last Name in the Enter Patient Name field. As you enter characters, OneTouch® DMS will search the database until it finds an exact or close match. When it does, the Patient Name will be highlighted in the large field. If this is the correct name, click on the **Select** button.

## Preferences

The Preferences screen lets you customize the OneTouch® DMS environment to meet your needs as a person with diabetes or a healthcare professional.

The Preferences screen contains four Tabs ([see Figure 2.10](#)). They are labeled **General**, **Report Options**, **Targets** and **Hypoglycemic**. Clicking on any of the four Tabs will activate the desired function. A description of each Tab function follows:

**Figure 2.10**

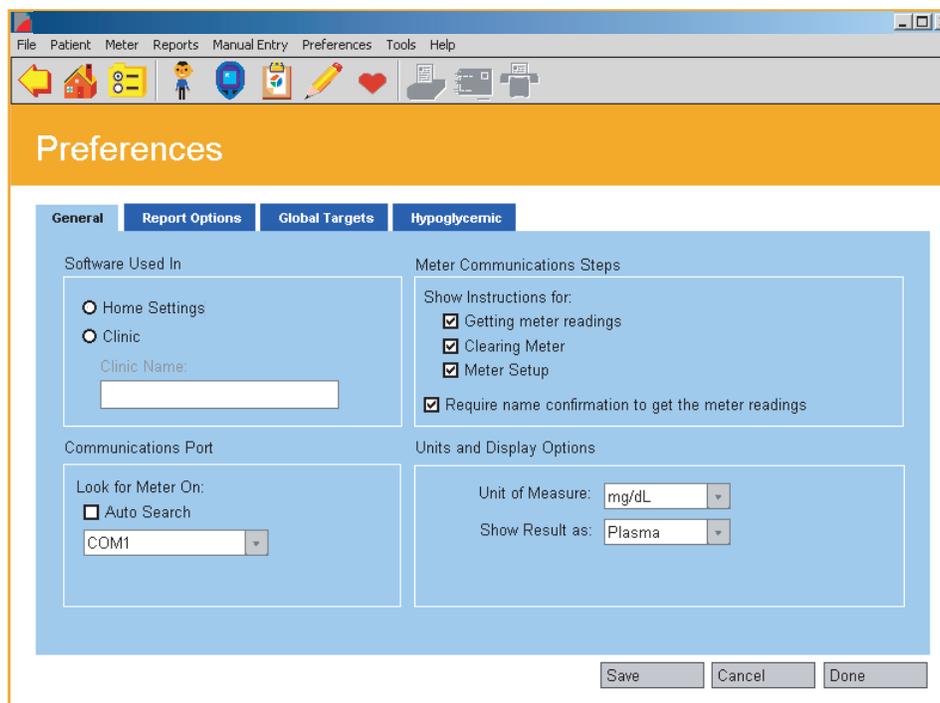

## General Tab

The General Tab lets you specify the settings that determine how you will use and interact with OneTouch® DMS ([see Figure 2.10](#)).

When you see this:	You can do this:
Look for Meter On	<p>Specify the serial port on your PC that you will use to connect to the meter. If you do not know the COM port, select <b>Autosearch</b>. OneTouch® DMS will attempt to locate the correct COM port automatically.</p> <p>If you know the correct COM port, uncheck Autosearch and select the port (COM1-COM256).</p> <p>“Autosearch” is the default setting.</p> <p><b>Note:</b> Many PCs only have 2 COM ports and your mouse or internal modem is usually connected to one of them. Refer to documentation for your PC for more information on available COM ports.</p>
Show Instructions for	<p>Specify if you want OneTouch® DMS to display instructions for these 3 functions that apply to readings in the meter memory. Select the options you want to display.</p>
Require name confirmation	<p>Specify if you want to confirm the patient name is correct each time you transfer meter readings to the OneTouch® DMS database.</p> <p>If you have already transferred readings from a meter to a patient in the database, the Software assumes you want to transfer the new readings in the same meter to the same patient. By requiring name confirmation, you will have the option of transferring readings to the same patient, a different patient, or a <i>Guest</i> patient.</p>


**General Tab (cont.)**

<b>When you see this:</b>	<b>You can do this:</b>
Units of Measure	<p>Specify whether you want readings displayed in OneTouch® DMS Reports as mg/dL (milligrams per deciliter) or mmol/L (millimoles per liter) units. Click on the down arrow in the <b>Unit of Measure</b> field. Then select the desired option from the drop-down menu. The unit of measure selected here will be used for glucose, cholesterol and triglycerides data.</p> <p>If the meter is set to mmol/L units, and you select mg/dL as your display option, OneTouch® DMS will automatically convert the readings for display purposes. The units setting on the meter will not be affected.</p>
Show Result As	<p>Specify whether glucose readings will be displayed in OneTouch® DMS Reports as either Whole Blood readings or Plasma readings. Click on the down arrow in the <b>Show Result as</b> field. Then select the desired option from the drop-down menu.</p> <p><b>Note:</b> To view results the same way they appear in the meter, refer to your meter display where the glucose unit of measure appears with the test result, and set the <b>Show Result as</b> option to display the same units.</p>

## Displaying Units of Measure

### IMPORTANT INFORMATION ABOUT UNITS OF MEASURE

- Different units of measure are standard in different countries or regions. Milligrams per deciliter (mg/dL) is the standard unit used in the United States. The unit of measurement used in Canada, some European countries and other parts of the world is mmol/L. This unit includes a decimal point but mg/dL results do not.
- There is a significant difference in the glucose reading depending on the units of measurement, mg/dL or mmol/L.
- To view results in OneTouch® DMS the same way they are displayed in your meter, refer to your meter display where the glucose unit of measure appears with the test result. If you wish to view results differently from the meter, OneTouch® DMS can automatically convert readings for display without affecting the setting in the meter.

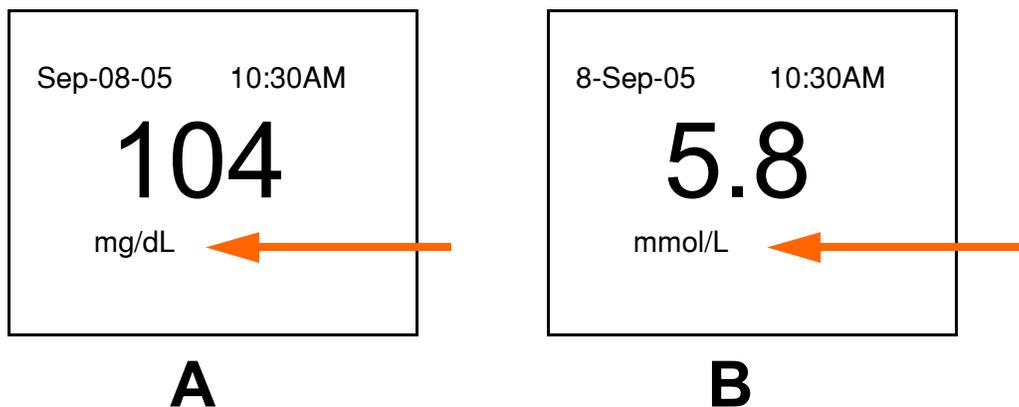

- A. Meter display showing mg/dL for the unit of measure.  
B. Meter display showing mmol/L for the unit of measure.

## Displaying Results as Plasma Values or Whole Blood Values

### **IMPORTANT INFORMATION ABOUT PLASMA AND WHOLE BLOOD READINGS:**

All OneTouch® meters require a sample of whole blood from your finger or arm to give you a blood glucose reading. But there is a difference in how certain OneTouch® Meters measure the amount of glucose in the blood.

A few older OneTouch® meters measure glucose based on the entire whole blood sample. Others measure glucose based on only the plasma portion of the whole blood sample. Plasma is the liquid portion of the whole blood sample after the red blood cells are removed. Most clinical laboratories measure plasma glucose. There is a difference in the glucose concentration in whole blood and plasma. A glucose reading from a plasma sample is about 12% higher than a reading from a whole blood sample. So if you are using a OneTouch® meter that measures plasma glucose, readings will be about 12% higher than those taken from a OneTouch® meter that measures glucose from the entire whole blood portion.

It's important to remember this difference when comparing glucose readings from OneTouch® Meters with measurements taken at the lab. Several OneTouch® Meters automatically convert readings to plasma values. Others may not, depending on the country or locale where the meter was purchased.

**CAUTION:** Due to the difference in glucose readings related to plasma or whole blood reporting, it is important to use the right test strips with OneTouch® Meters. That's because test strips are designed to work hand in hand with the way OneTouch® Meters measure glucose in the blood. For this reason, you should only use test strips purchased in the same country or locale where you purchased your meter. This will ensure that you are using the right test strips. As a result, your glucose readings will be as consistent and accurate as possible. Please refer to the documentation that comes with the test strips.


**Note:** When communicating with devices other than OneTouch® Meters, the Software assumes that these devices are plasma calibrated unless otherwise specified.

If you want to do this:	Then do this:
Compare meter readings with those from a lab	Set the OneTouch® DMS display option to plasma units.
Display meter readings in OneTouch® DMS Reports exactly as they are reported on the meter	First check how the meter reports readings. The glucose unit of measure appears with the test result on the meter display. Then set the OneTouch® DMS display option to match the way it's done with the meter display.

## Report Options Tab

The Report Options Tab lets you select the default settings for routine Report viewing and printing (see [Figure 2.11](#)).

**Figure 2.11**

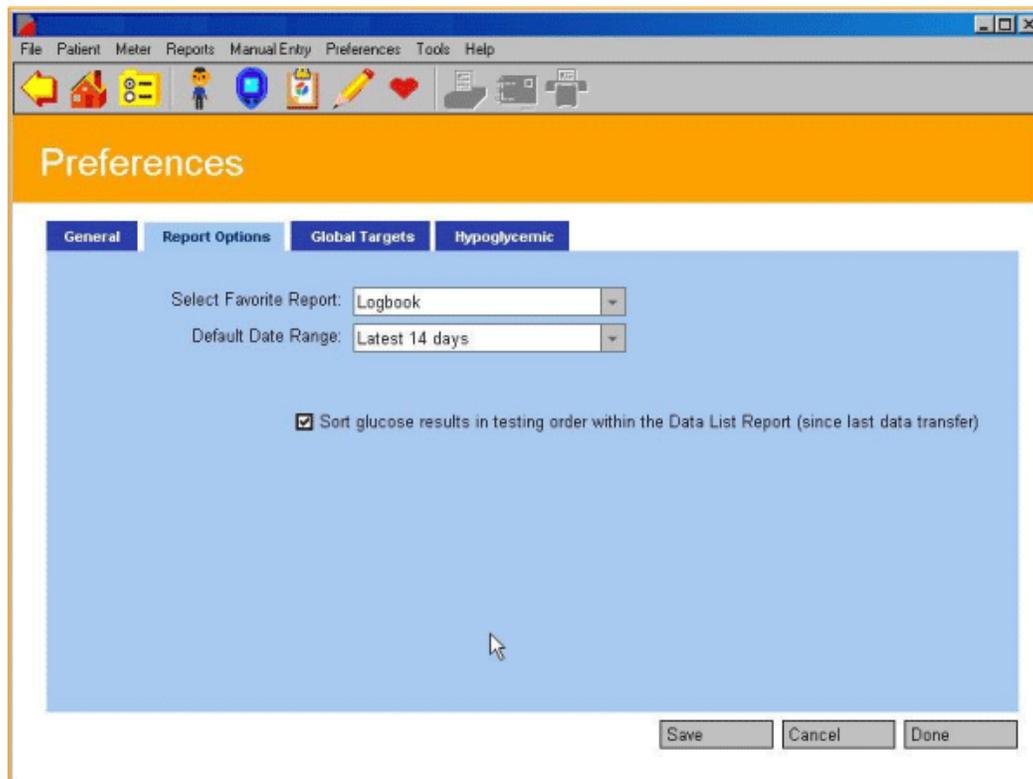

When you see this:	You can do this:
<p><b>Select Favorite Report</b></p>	<p>Specify the Report displayed directly after transferring readings from a meter. Click on the down arrow in the <b>Select Favorite Report</b> field. Select the Report you want from the drop-down menu. You may select any of the following Reports as the Favorite Report:</p> <ul style="list-style-type: none"> <li>■ Logbook</li> <li>■ Glucose Trend</li> <li>■ Pie Chart</li> <li>■ Standard Day</li> <li>■ Average Readings</li> <li>■ Insulin</li> <li>■ Exception</li> <li>■ Data List</li> <li>■ Histogram</li> <li>■ Health Checks</li> <li>■ Summary</li> </ul>
<p><b>Default Date Range</b></p>	<p>Specify one of several pre-set time periods for viewing and printing Program Reports. Click on the down arrow in the <b>Default Date Range</b> field. A drop-down menu of pre-set time periods will appear. Select the one you want. You may select from any of the following Date Ranges as the default:</p> <ul style="list-style-type: none"> <li>■ Since Last Transfer</li> <li>■ Latest 7 Days</li> <li>■ Latest 14 Days</li> <li>■ Latest 30 Days</li> <li>■ Latest 60 Days</li> <li>■ Latest 90 Days</li> </ul> <p>“Latest 14 days” is the default Date Range.</p> <p>OneTouch® DMS will display the corresponding calendar start and end dates (in Program Reports) based on the option chosen.</p>

When you see this:	You can do this:
<b>Sort glucose results in testing order within the Data List Report (since last data transfer)</b>	<p>Specify if you would like to see glucose results in certain Data List Reports in the order that tests were taken, rather than by the calendar date that was automatically saved with each result.</p> <p>This option is available only when viewing or printing the Data List Report and specifying the “Since last transfer” Date Range. This option is not available if you have transferred results from a OneTouch® UltraSmart® Meter. To display glucose results in chronological order as defined by the dates that were tagged to the meter result, the default selection must be deselected (unchecked).</p>

## Global Targets Tab

The Global Targets Tab lets you create glucose target ranges that can be applied to all patients in the database ([see Figure 2.12](#)). Templates may be set for each Diabetes Type (including three Custom types) and for both before-meal and after-meal slots.

The following leading health organization[s] make[s] target range recommendations in their clinical practice guidelines:

### [A.] American Diabetes Association

#### For mg/dL locales:

Pre-Meal Targets*	90-130 mg/dL plasma	or	80-116 mg/dL whole blood
Post-Meal Targets*	90-180 mg/dL plasma	or	80-161 mg/dL whole blood

#### For mmol/L locales:

Pre-Meal Targets*	5.0-7.2 mmol/L plasma	or	4.5-6.4 mmol/L whole blood
Post-Meal Targets*	5.0-10.0 mmol/L plasma	or	4.5-8.9 mmol/L whole blood

### [B.] American Association of Clinical Endocrinologists guideline for intensive diabetes self-management.

#### For mg/dL locales:

Pre-Meal Targets*	90-110 mg/dL plasma	or	80-98 mg/dL whole blood
Post-Meal Targets*	90-140 mg/dL plasma	or	80-125 mg/dL whole blood

#### For mmol/L locales:

Pre-Meal Targets*	5.0-6.1 mmol/L plasma	or	4.5-5.4 mmol/L whole blood
Post-Meal Targets*	5.0-7.8 mmol/L plasma	or	4.5-6.9 mmol/L whole blood

General rules for creating glucose target ranges are as follows:

- The **From** and **To** values represent the lower and upper limits for each Meal Slot target range
- Each **To** value must be greater than or equal to its corresponding **From** value
- Before-Meal and After-Meal target ranges can overlap
- You must specify both the Before-Meal and After-Meal target ranges for each Diabetes Type
- Default values for glucose target ranges are provided for Type 1 and Type 2 Diabetes Types only
- Global target ranges can be applied to every patient with that Diabetes Type, as specified in the General Tab of the Patient Setup screen

**Figure 2.12**

The screenshot shows the 'Preferences' dialog box with the 'Hypoglycemic' tab selected. The window title is 'Preferences' and the menu bar includes 'File', 'Patient', 'Meter', 'Reports', 'Manual Entry', 'Preferences', 'Tools', and 'Help'. The toolbar contains icons for navigation and actions. The main content area is titled 'mg/dL (Plasma)' and is divided into three sections: 'Type 1', 'Type 2', and 'Gestational'. Each section has a 'From' and 'To' column for 'Before Meal' and 'After Meal' targets. There are also three 'Custom' target ranges (Custom 1, Custom 2, Custom 3) with their own 'From' and 'To' columns. The 'Save', 'Cancel', and 'Done' buttons are at the bottom right.

Diabetes Type	Meal Slot	From	To	Custom	From	To
Type 1	Before Meal	90	110	Custom 1		
		90	140	Before Meal		
	After Meal	90	140	After Meal		
Type 2	Before Meal	90	110	Custom 2		
		90	140	Before Meal		
	After Meal	90	140	After Meal		
Gestational	Before Meal			Custom 3		
				Before Meal		
	After Meal			After Meal		


**Notes:**

- Default before-meal and after-meal target ranges are provided for Type 1 and Type 2 diabetes. There are no default target ranges for Gestational diabetes or the three Custom diabetes types.
- As you set or change target ranges for each Diabetes Type, you are impacting the targets that are applied to all patients with that Diabetes Type who use Diabetes Type targets. This means you are setting the target ranges for viewing and printing Reports for each individual patient. It also means you are affecting the target ranges that are applied in the General Tab of the Patient Setup screen.
- Targets will shift if you switch between Plasma and Whole Blood.

## Hypoglycemic Tab

The Hypoglycemic Tab lets you set a Hypoglycemic threshold (see [Figure 2.13](#)). This is the baseline at or below which glucose readings in Program Reports are designated “hypoglycemic”.

**Figure 2.13**

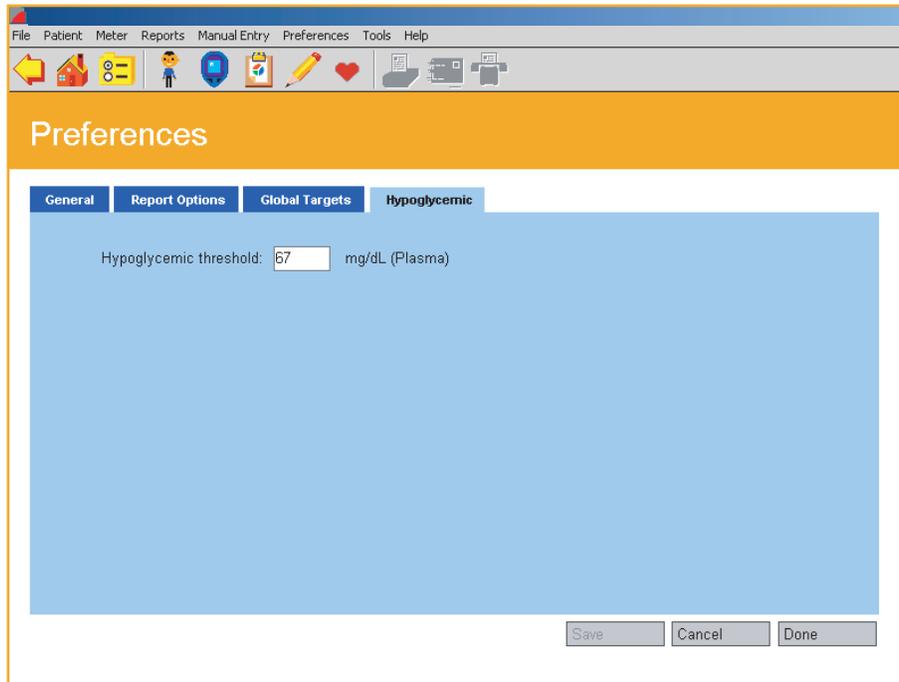

**When you see this: You can do this:****Hypoglycemic threshold**

Modify the **Hypoglycemic threshold** by entering the desired number. Please note whether the Hypoglycemic threshold is displayed in mg/dL (or mmol/L) units and as a Plasma (or Whole Blood) value.

The Hypoglycemic threshold must be set below the lower limits of all Before-Meal and After-Meal target ranges.

Depending on locale one of the following default hypoglycemic threshold settings is provided:

- 67 mg/dL (3.7 mmol/L) when using plasma values, and 60mg/dL (3.3 mmol/L) when using whole blood values

or

- 70 mg/dL (3.9 mmol/L) when using plasma values, and 63 mg/dL (3.5 mmol/L) when using whole blood values.

To change the Hypoglycemic threshold, enter the desired threshold value in the field. The Hypoglycemic threshold must be from:

- 50 to 90 mg/dL (2.8 to 5 mmol/L) when using plasma values

or

- 45 to 80 mg/dL (2.5 to 4.4 mmol/L) when using whole blood values.

**Notes:**

- The Hypoglycemic threshold will change whenever you shift between Whole Blood and Plasma display options. You may switch display options in the General Tab of the Preferences screen. OneTouch® DMS will convert the value automatically.
- You can customize the hypoglycemic value for individual patients in the Patient Setup General screen.

## Modify Meter Settings

The Meter Settings main screen lets you modify meter settings that will impact how data is stored in the meter and displayed in the OneTouch® DMS Reports.

The main screen contains up to six Tabs. They are labeled **Clock**, **General**, **Schedule**, **Insulin**, **Oral Meds** and **Targets & Hypoglycemic**. Clicking on any of the six Tabs will activate the desired function.


**Note:** Some Tabs (or Tab options) may not be available for your meter. Only those that are available will appear on the OneTouch® DMS screens. When you connect the meter to the PC, OneTouch® DMS determines which Tab options are available for your meter. Only those options will be displayed.

## Clock Tab

The Clock Tab lets you set and match the time and date for the meter and your PC (see [Figure 2.14](#)).


### Figure 2.14

You may select only one of the three Clock Tab options. If you select the third option, you must enter a Date and Time for the meter.

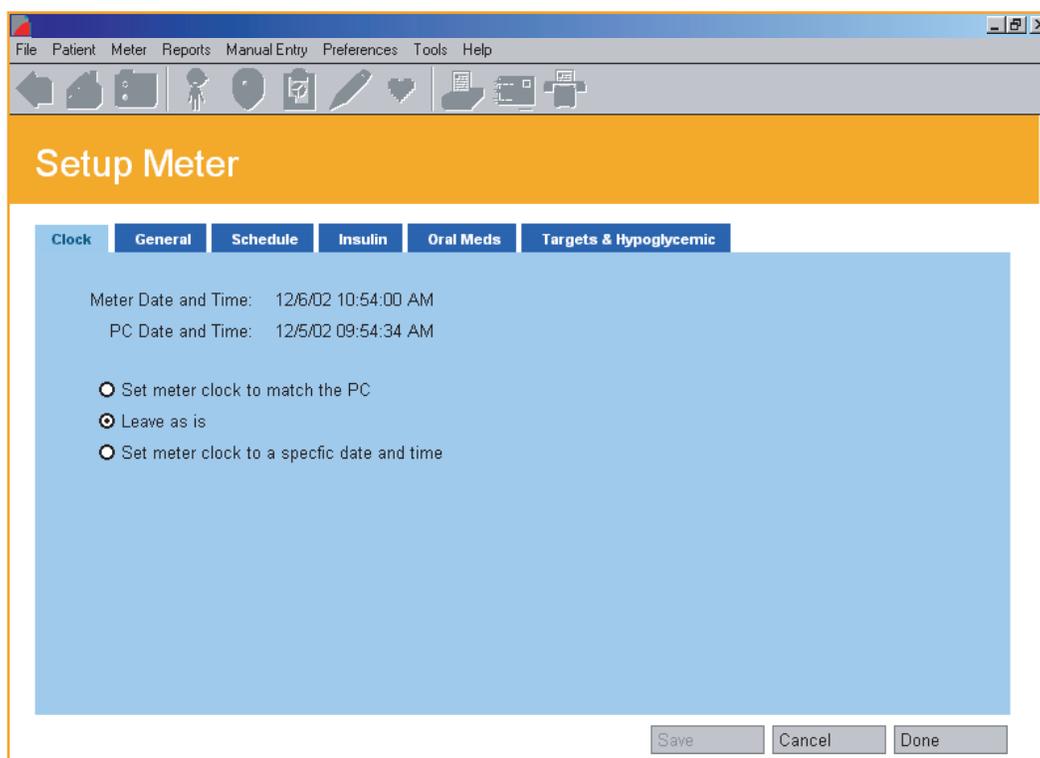

**Note:** To ensure that glucose readings appear in OneTouch® DMS Reports in the same meal slot that was used to store the readings in the meter, it is important that the PC clock time matches the meter clock time.

If the clock times differ, glucose readings that are transferred to OneTouch® DMS may be displayed in an inappropriate meal slot. If any out-of-target readings appear in the wrong meal slot, this may lead you to misinterpret the information when tracking and trending data.

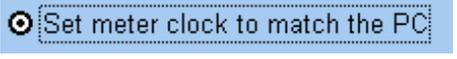
When you see this:	You can do this:
	Match the meter clock time with the PC clock time.
	Leave the meter clock and PC clock times as they are.
	<p>Set the date and time of the meter clock so that it's different from the time of the PC clock.</p> <p>To specify the date, modify the information in the Date and Time fields that appear when this option is selected, or click on the down arrow. A pop-up calendar box will appear.</p> <p>To change the year, click on the year and a set of up/down arrows will appear. Click on the up/down arrows until you find the desired year. To change the month, click on the left/right arrows that appear on the sides of the calendar box until you find the desired month. Then click on the desired day.</p> <p>You may modify the Time field by entering the desired Time, or by clicking on the hour or minute values that appear. Then click on the up/down arrows until you find the desired hour and minute.</p> <p><b>Note:</b> If you change the meter clock time, only those glucose readings taken after you have changed the time will be affected. Results taken prior to the change are not affected.</p>

## General Tab

The General Tab lets you specify many of the settings that control how meter readings are displayed on the meter screen ([see Figure 2.15](#)). Different setup options are available for different meters.

For each General Tab option, select the desired entry from each of the drop-down menus in the displayed fields.

**Figure 2.15**

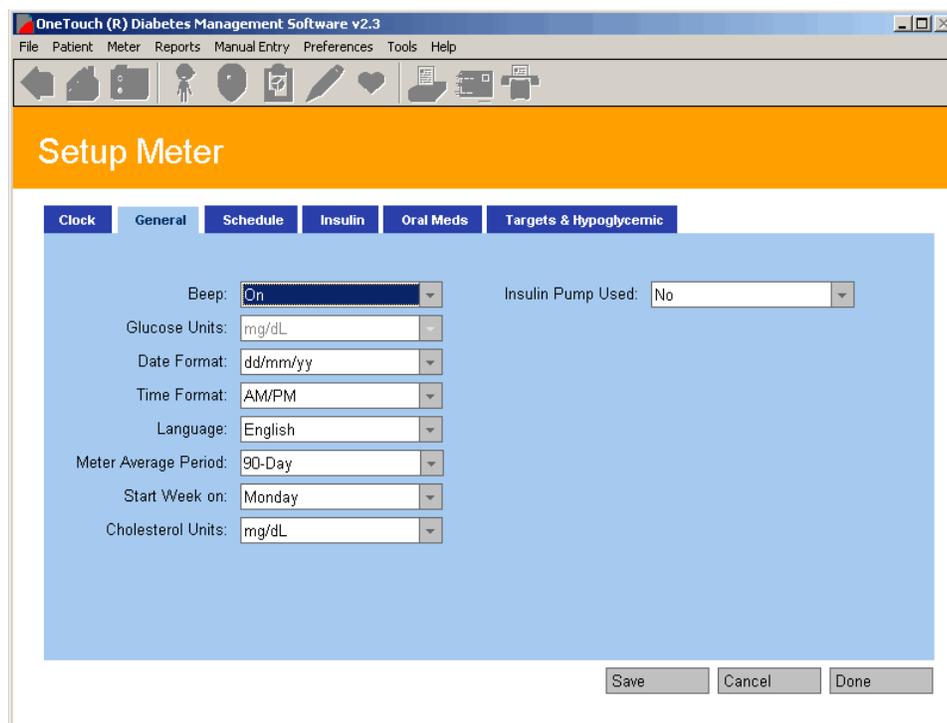

### Notes:

- Figure 2.15 is a view of a OneTouch® UltraSmart® Meter setup screen
- Not all options are available for all meters


<b>When you are using a OneTouch® UltraSmart® Meter and you see this:</b>	<b>You can do this:</b>
Language	Specify the language the meter uses to display messages.
Beep	Turn meter audio cues ON or OFF. Audio cues are used to confirm the meter has performed a function or to alert you to a message.
Cholesterol Units	Display cholesterol, triglycerides, and glucose readings as either mg/dL (milligrams per deciliter) or mmol/L (millimoles per liter).
Time Format	Display times either in AM/PM format or 24-hour format (for example, 6:00 PM is 18:00 in 24-hour format).
Date Format	Display dates either in mm/dd/yy format or dd/mm/yy format (mm is the month, dd is the day, and yy is the year).
Start Week on	Specify either Sunday or Monday as the first day in the week.
Meter Average Period	Modify the 14-day glucose readings average to 7, 30, 60 or 90 days. The average will appear next to the most recent reading.
Insulin Pump Used	Specify if an insulin pump is used.

## Schedule Tab

The Schedule Tab lets you customize the start and end times for the 7 pre-defined Meal Slots that are stored in the meter (see [Figure 2.16](#)). The meter will tag and record all meter readings with a Meal Slot designation. Each Meal Slot start time begins when the previous one ends. So when you change the **From** time of one Meal Slot, OneTouch® DMS automatically changes the **To** time of the previous Meal Slot.

**Note:** This Tab is only available when connecting a OneTouch® UltraSmart® Meter.

**Figure 2.16**

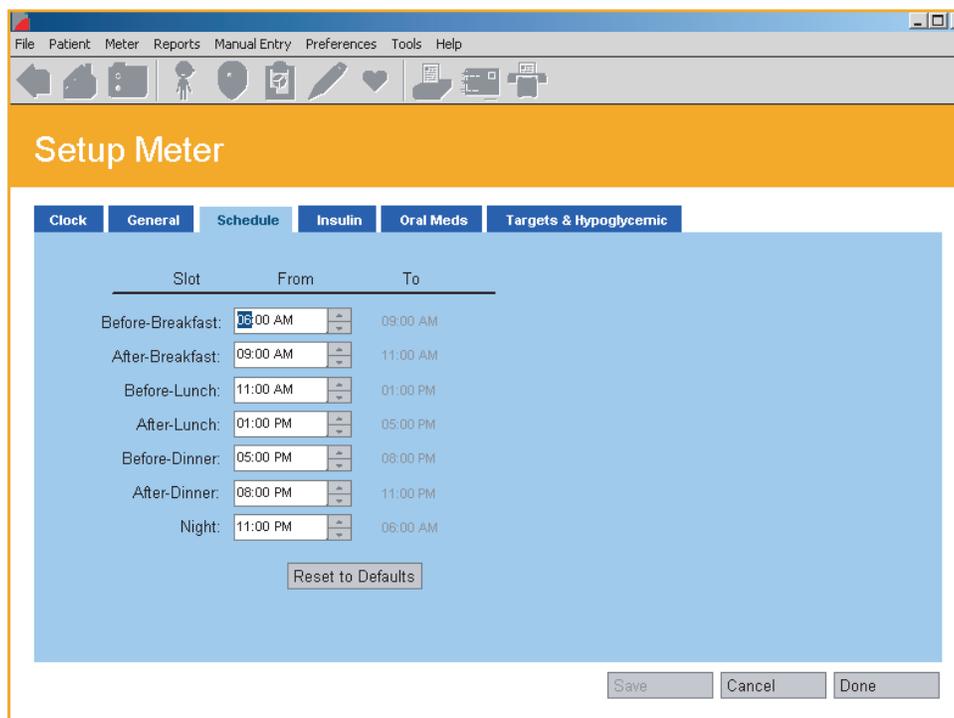

**Note:** To ensure that glucose readings appear in OneTouch® DMS Reports in the same meal slot that was used to store the readings in the meter, it is important that the meal schedule stored in the meter matches the schedule stored for that patient in OneTouch® DMS. If you change the schedule stored in the meter, ensure that the schedule for that patient in OneTouch® DMS is changed as well.

If the schedules don't match, glucose readings that are transferred to OneTouch® DMS may be displayed in an inappropriate meal slot. If any out-of-target readings appear in the wrong meal slot, this may lead you to misinterpret the information when tracking and trending data.

When you see this:	You can do this:
Before-Breakfast After breakfast Before-Lunch After-Lunch Before-Dinner After-Dinner Night	Specify the time periods for any Meal Slot. Click on the up/down arrows to get to the desired <b>From</b> time for each Meal Slot. The corresponding <b>To</b> time of the previous Meal Slot will be updated automatically.

## Insulin Tab

The Insulin Tab lets you add new Insulins to the Types that are already stored in the meter (see [Figure 2.17](#)).

**Note:** This Tab is only available with a OneTouch® UltraSmart® Meter.

**Figure 2.17**

Existing Insulins (A)	Add New Insulins (B)	Select Favorite Insulins From A and/or B
Rapid	Humalog	InsulinA
Regular		InsulinB
Lente		None
NPH		
UltraLen		
Premixed		
Other		
InsulinA		
InsulinB		
InsulinC		


When you see this:	You can do this:
Existing Insulin Types	Display/list the Insulin Types already available with the meter.
Additional Insulin Types	Add up to 11 additional Insulin Types.
Select Favorite Insulin Types	<p>Select 3 Insulin Types (from either of these lists) to be stored in the meter as <b>Favorite</b> Insulin Types. This will make it easier to enter and record Insulin units in the meter memory.</p> <p>Fields will appear for the 3 <b>Favorite</b> Insulin Types. Click on the down arrow in each field. A drop-down menu of Insulin Types will appear. Select the desired option for each Favorite Insulin Type.</p>

## Oral Meds Tab

The Oral Meds Tab lets you add new Medications to the Types that are already stored in the meter ([see Figure 2.18](#)).

**Note:** This Tab is only available with a OneTouch® UltraSmart® Meter.

**Figure 2.18**

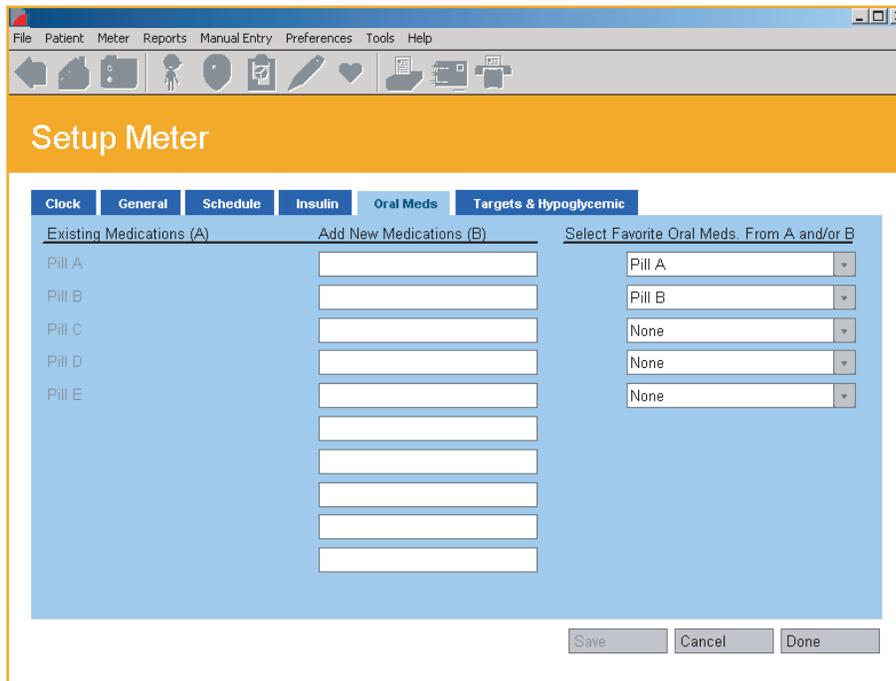

When you see this:	You can do this:
Existing Medications	Display/list the Oral Medication Types (Pills A through E) already available with the meter.
Additional Medications	Add up to 10 additional Oral Medication Types.
Select Favorite Medications	<p>Select up to five Oral Medication Types from either of these lists to be stored in the meter as <b>Favorite</b> Oral Medication Types. This will make it easier to enter and record medication doses in the meter memory.</p> <p>Fields will appear for the five <b>Favorite</b> Oral Medication Types. Click on the down arrow in each field and select the desired option from the drop down menu.</p>

## Targets & Hypoglycemic Tab

The Targets & Hypoglycemic Tab lets you set the Before-Meal and After-Meal glucose target ranges, and the Hypoglycemic threshold, that are stored in the meter ([see Figure 2.19](#)).

General rules for creating glucose target ranges are as follows:

- The **From** and **To** values represent the lower and upper limits for each Meal Slot target range
- Each **To** value must be greater than or equal to its corresponding **From** value
- Before-Meal and After-Meal target ranges can overlap
- The **From** values for any Before-Meal and After-Meal target ranges must be greater than the Hypoglycemic threshold

**Note:** This Tab is only available with a OneTouch® UltraSmart® Meter.

**Figure 2.19**

The screenshot shows the 'Setup Meter' application window. The title bar contains 'File Patient Meter Reports Manual Entry Preferences Tools Help'. Below the title bar is a toolbar with various icons. The main content area has a blue header with the text 'Setup Meter'. Below the header are several tabs: 'Clock', 'General', 'Schedule', 'Insulin', 'Oral Meds', and 'Targets & Hypoglycemic'. The 'Targets & Hypoglycemic' tab is selected. The content area is light blue and contains the following fields:

	Target	From	To	
Before Meal:	<input type="text" value="20"/>	<input type="text" value="130"/>	<input type="text" value="130"/>	mg/dL (Plasma)
After Meal:	<input type="text"/>	<input type="text"/>	<input type="text"/>	
Meter Hypoglycemic Level Setting:	<input type="text" value="70"/>			

At the bottom right of the content area are three buttons: 'Save', 'Cancel', and 'Done'.

## Part 3: Diabetes Management

- Download (Transfer) Meter Readings
  - Connecting a Meter to your PC
  - Transferring Meter Readings
- Assigning Meter Readings to the database
- Clear Meter Readings
- View and Manage Data Records and Reports
  - General Report Information
  - Report Formats
    - Logbook Report
      - Day View Screen
    - Glucose Trend Report
    - Pie Chart Report
    - Summary Report
    - Standard Day Report
    - Health Checks Report
      - Health Checks Detail Screen
    - Average Readings Report
    - Insulin Report
    - Exception Report
    - Data List Report
    - Histogram Report
- Printing, Faxing, and E-Mailing Reports
- Modifying Data Records
  - Rules for Modifying Data Records
  - Editing, Omitting, and Deleting Data Records
- Manually Enter Results
  - Rules for Adding Data Records Manually
    - Glucose
    - Insulin
    - Oral Meds
    - Exercise
    - Food
    - Health Records

## Download Meter Readings

Glucose readings transferred from the meter are saved in the database. Any Date and Time tags and any Comments or Event Markers associated with a reading are also transferred. Event Markers impact how glucose readings are organized and displayed in Program Reports. Once readings have been transferred, you may clear the readings from the meter memory.

Before you can transfer readings or clear readings from the meter memory, you must first establish communication between your PC and the meter. Communication steps are covered below. Please note the following as you establish communications between the meter and your PC:

- It makes no difference which language the meter is using to display readings. OneTouch® DMS is designed to work with any language you have selected for the display.
- Only one meter can communicate with OneTouch® DMS at any given time.
- You may click on the **Cancel** button at any time to stop the process.
- If the meter Clock time differs from the PC Clock time by more than 15 minutes, a message box will appear. It will instruct you to make sure the PC and meter Clock times are the same and accurate.

**Caution:** If you will be transferring readings from more than 1 meter (for example, one meter at home and one at work), make sure all meter clock times match the PC clock time. This will help ensure that your results can be compared to one another and are accurate.

## Connecting a Meter to Your PC

1. Connect the appropriate cable to the meter data port. Firmly push the cable until it clicks into place.
2. Connect the other end of the cable to one of the available USB ports on your PC.
3. Make sure the COM port number matches the selection you made in the General Tab on the Preferences screen. Or, select the **Autosearch** option in the General Tab. This is the default setting. With the Autosearch option, OneTouch® DMS will try to locate the correct COM port where you connected the meter.
4. To begin data transfer, most meters need to be turned OFF; some meters need to be turned ON. If communication fails when you have the meter OFF, turn the meter ON.


## Transferring Meter Readings

Once you have successfully connected the meter to your PC, you can complete the Download Meter Readings process ([see Figure 3.0](#)):

1. Click on the **Download Meter Readings** icon on the home screen.
2. Click on the **Download Meter Readings** button on the Instruction screen when you are ready.

**Note:** If you selected **Show Instructions for Getting meter readings** in the General Tab of the Preferences screen, instructions will appear to guide you through the Download Meter Readings process ([see Figure 3.0](#)). If you do not want to see these reminder steps in the future, select the **Don't show these instructions again** option on the Program screen. Or you may go to the General Tab of the Preferences screen and deselect the **Show Instructions for Getting meter readings**.


**Figure 3.0**

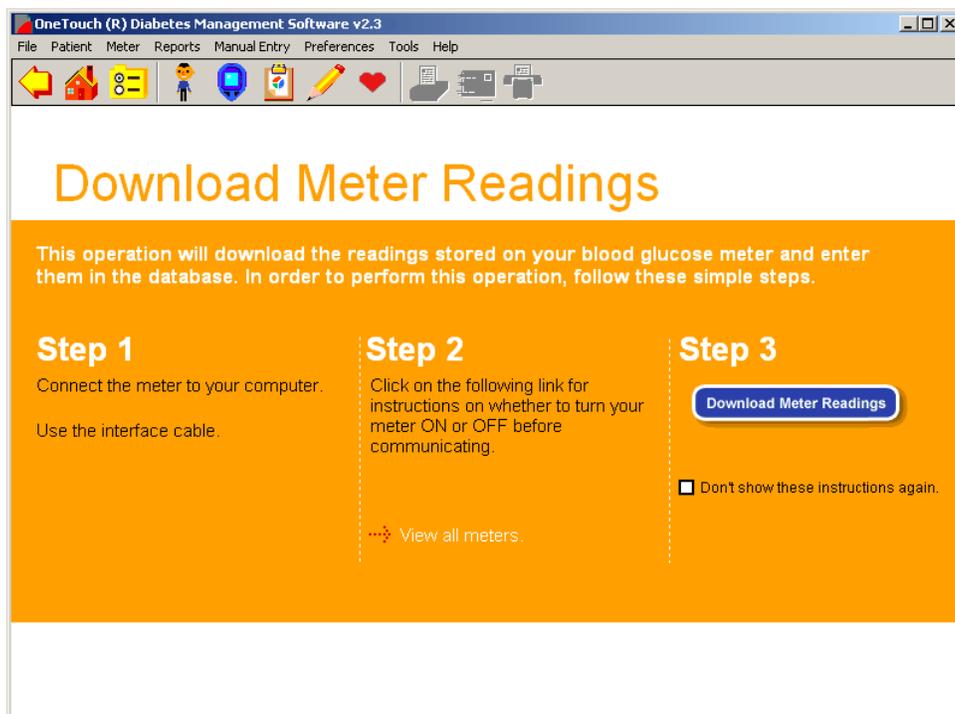

## Assigning Meter Readings to the Database

Readings transferred from a meter must always be linked to a patient name in the OneTouch® DMS database (see [Figure 3.1](#)). Unless you selected **Require name confirmation to get the meter readings** in the General Tab on the Preferences screen, OneTouch® DMS will automatically try to assign meter readings to the same patient you transferred its readings to the last time. OneTouch® DMS keeps track of meter-patient assignments for every transfer by checking a unique ID number that's stored in the meter memory.

**Figure 3.1**

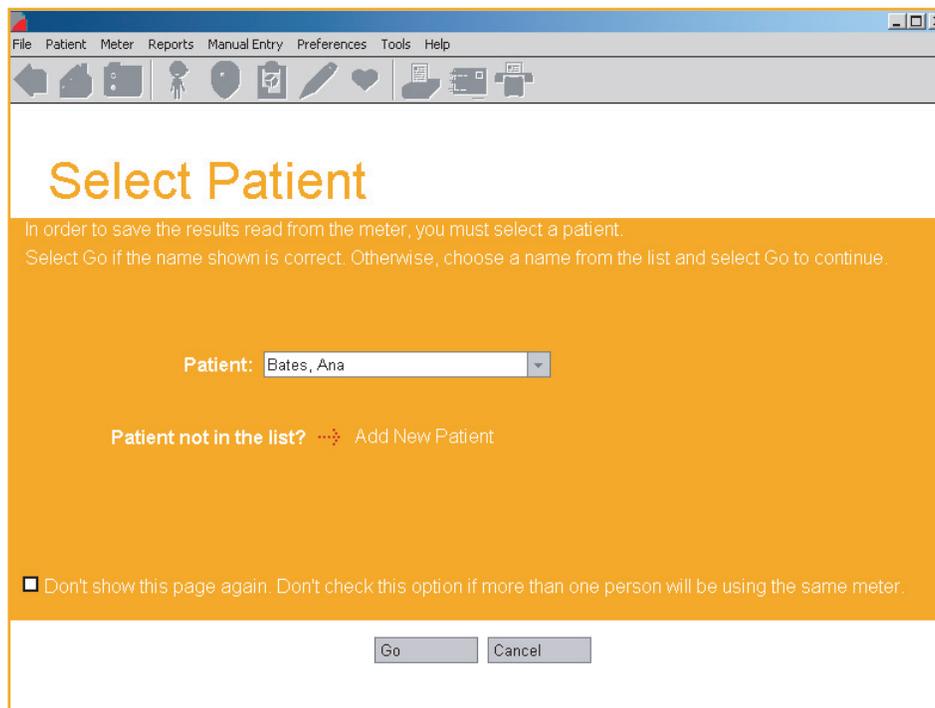

## The rules for assigning meter readings to the database are as follows:

- You may transfer readings from a particular meter to the same patient as the previous transfer, a new patient, or a Guest patient.
- If you want to transfer readings from a meter to a different patient than before, you must have selected **Require name confirmation to get the meter readings** in the General Tab of the Preferences screen. You will be prompted to select a new patient name. Scroll through the names in the Patient field and select the desired one. If you do not see the patient name you are looking for, you may click on the **Add New Patient** link to create a new name.
- A Guest patient is a permanent entry in the OneTouch® DMS database. It serves as a generic patient name. For example, you can transfer meter readings to the Guest patient name for anyone for whom you want to generate a report but for whom you don't care if their records are saved in the database. Previous Guest entries are deleted from the database every time you start up the Software and each time you transfer new readings to the Guest patient.
- You may stop the transfer process by clicking on the **Cancel** button.

## Notes:

- If you are a new OneTouch® DMS user with a blank (empty) database, only the Guest patient name exists in the database. This means you will have to click on the **Add New Patient** button to create a new patient name for the meter transfer.
- If meter readings are not successfully transferred, an on screen message box will prompt you to check the meter/cable connections. For other meter communications problems, see Appendix C: Troubleshooting.

After readings are transferred from a device and saved to a patient file, OneTouch® DMS will display the following information:

1. Total number of results transferred
2. Number of duplicate results
3. Number of *unsuitable* results. A result is reported as unsuitable when the record transferred from the device is invalid. Some examples of unsuitable results include:
  - The record is missing date or time information.
  - The record date is invalid (e.g. the 13th month in the year).
4. Number **unsupported** results. Unsupported results are valid result types that are either not currently shown in OneTouch® DMS reports or include more information than expected. Some examples of unsupported results are:
  - A new type of record, e.g. measuring lung capacity.
  - A temperature value that is tagged to a glucose meter reading.

Both of these examples are theoretical. The first describes a record type that is not currently shown in OneTouch® DMS reports. The second describes a result with too much information. In both examples, the data is stored in OneTouch® DMS, but not displayed in a current OneTouch® DMS report. Unsupported results may be viewed if a software driver has been installed on your PC that lets you display the data in a special report format. (See **Add-in Reports** under *View and Manage Reports and Data Records*).

**Caution:** OneTouch® DMS assumes that a meter is associated with only one patient at a time. So when a transfer takes place, all readings are assumed to belong to that one patient. If you plan to have more than one patient use the same meter, make sure to clear the meter readings between transfers. This will ensure that data for different patients are not intermingled. Refer to the *Clear Meter Readings* section for information about clearing meter readings.

## Clear Meter Readings

Once you transfer readings to the database, you can clear the readings from the meter memory. All readings will be permanently erased from the meter. Clearing meter readings follows a similar process to Download Meter Readings.


1. Connect the appropriate cable to the meter data port. Firmly push the cable until it clicks into place.
2. Connect the other end of the cable to one of the available USB ports on your PC.
3. Make sure the COM port number matches the selection you made on the General Tab of the Preferences screen. Or, select the **Autosearch** option on the General Tab. If the Autosearch option is selected, OneTouch® DMS will try to locate the correct COM port where you connected the meter.
4. To connect, most meters need to be turned OFF; some meters need to be turned ON. If communication fails when you have the meter OFF, turn the meter ON.

## Completing the Clear Meter Readings Process

Once you have successfully connected the meter to your PC, you can complete the Clear Meter Readings process ([see Figure 3.2](#)).

1. Click on **Meter** on the Menu Bar. Select Clear Meter Readings from the drop-down menu.
2. Click on the **Clear Meter Readings** button in Step 3 below when you are ready.

**Figure 3.2**

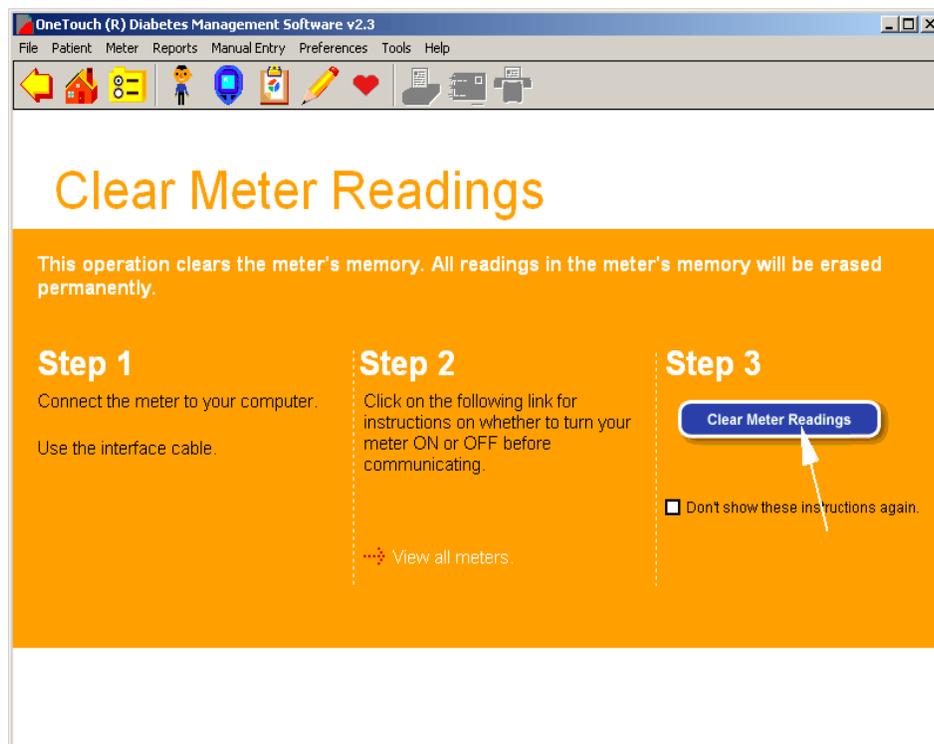

**Note:** If you selected **Show Instructions for Clearing Meter** in the General Tab on the Preferences screen, instructions will appear to guide you through the Clear Meter Readings process. If you do not want to see these reminder steps in the future, select **Don't show these instructions again** on the OneTouch® DMS screen. Or you may go to the General Tab of the Preferences screen and deselect **Show Instructions for Clearing Meter**.

## View and Manage Reports and Data Records


### General Report Information

OneTouch® DMS lets you view and analyze glucose readings over different time periods. You can choose from among 11 different Report formats ([see Figure 3.3](#)). Each one presents glucose readings and other health data in a different way.

Data may be viewed as plasma or whole blood values. Each Report clearly indicates which values are used in the display. Keep in mind that glucose values, target ranges, and the Hypoglycemic Value will be adjusted according to the display option you choose.

If you have more than one insulin Regimen, be sure you have the right one in place for the selected Date Range.

**Figure 3.3**

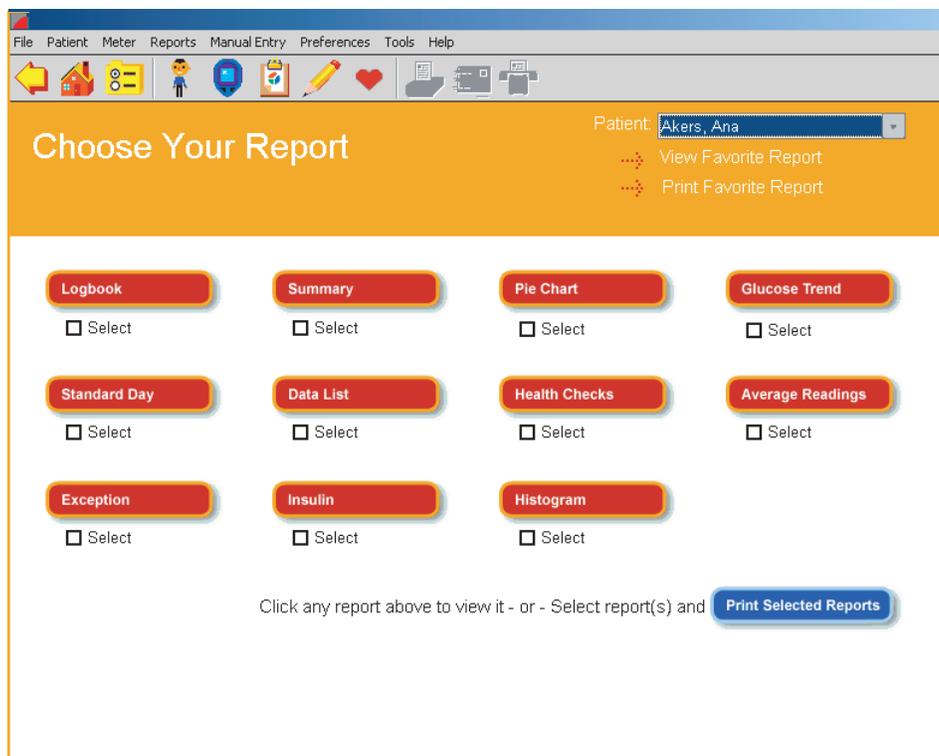

To view Program Reports you have several options:

- OneTouch® DMS will display a Favorite Report directly after readings are transferred from a meter. You can set the Favorite Report in the Report Options Tab on the Preferences screen.
- Click on the **View Reports** icon on the home screen. This will take you to the Choose Your Report main screen where an icon will appear for each of the 11 available Reports. Click on any of the buttons to display the desired Report. To view the Favorite Report, click on the **View Favorite Report** link. From the Choose Your Report main screen you can select one or more Reports for printing by clicking on any number of the **Select** boxes that appear under the Report buttons. Then click on the **Print Selected Reports** button.
- Click on the **Clipboard** icon in the Tool Bar. This will display the Favorite Report.
- Click on **Reports** in the Menu Bar. Then select the desired Report from the drop-down menu. This will display the desired Report.
- While displaying the current Report, you can always choose to view another Report, or change the Date Range for viewing the same Report.
  - To change the Date Range, you may either select one of the pre-defined Date Ranges from the drop-down menu, or customize your own Date Range. To customize the Date Range, modify the dates in the **From** and **To** boxes or click on either of the down arrows. A calendar box will appear where you can modify either date. You must click on **Apply** before the Report will appear for the new Date Range.
  - To change the Report, click on the down arrow in the Report field, and select a new Report format. You must click on **Apply** before the new Report will appear.

Some Reports may be larger than the current OneTouch® DMS screen. Use the scroll bar on the right to move down to the bottom part of the Report if necessary.

To enlarge your screen for viewing Reports you may click on the **Maximize** button on the top right of the OneTouch® DMS screen.

**Note:** Several Reports include the percentage of glucose readings that are outside and within the target ranges. You may note that some percentages do not add up to exactly

100%. This is due to “rounding error” and does not reflect on the accuracy of the calculations or the OneTouch® DMS.

## Viewing Additional Data

The Software database may include additional data records that cannot be displayed in existing OneTouch® DMS reports. These records may have been transferred from a newer OneTouch® Meter or a device from an authorized vendor. When additional records exist for a given date range and a software driver has been installed on your PC that lets you display the data in a special report format, OneTouch® DMS will provide the following cues:


- A hyperlink for an **Add-in Report** will appear at the bottom of relevant OneTouch® DMS reports. Click on the hyperlink to view these records.
- From the Reports command on the Menu bar, the drop down list of available reports may include an **Add-in Reports** option that will take you to the “Select A Report” screen. This screen will include links to any Add-in Reports that were detected by OneTouch® DMS during application startup, and where the user can select the date range and/or patient name.
- On the Choose Your Report screen, a hyperlink for **Add-in Reports** will be displayed if any **Add-in Reports** were detected by OneTouch® DMS during application start up.

### Rules for Viewing Additional Data

- The structure and content of **Add-in Reports** are defined by the software driver for the new device, and are not subject to the rules for displaying existing OneTouch® DMS reports.
- The Statistics Panel content will not include data that are shown in Add-in reports, unless the data is also included in existing OneTouch® DMS reports.
- **Add-in Reports** cannot be faxed or e-mailed through OneTouch® DMS.

## Using ToolTips (the “hover-over” feature) when viewing Reports

ToolTips provide an easy way to display information that's tagged to a particular data value on a Report screen. This OneTouch® DMS feature is also very useful when viewing results that are represented by symbols rather than numbers, or that are very close to one another on the screen. To activate ToolTips, move the mouse pointer to a particular data value. A message box will appear with Date, Time, Comments, and other information depending on the result type.

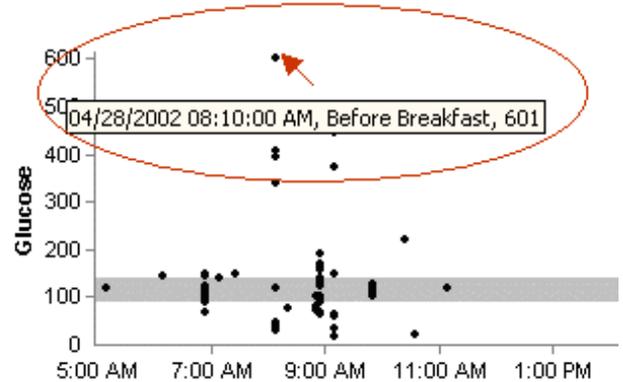

- Using ToolTips with Insulin Doses will also display the Insulin Type.
- Using ToolTips with **Exercise** icons will also display the duration and intensity of the exercise.

**Note:** Remember that selected Reports (Logbook, Summary, Standard Day, and Day View) will display records for meal slots that cross the midnight hour as falling in the same day as the Before-Breakfast slot that preceded it. However, ToolTips will show the actual date of the readings.

## Report Header

A Header appears at the top of every printed Report.

The Header includes:

- Report Name
- Patient Name
  - The name of the patient covered by the Report
- Date Range
- Report Date
  - The calendar date the report was generated
- Glucose Units
  - Specifies if mg/dL or mmol/L units were used in the Report
- Plasma or Whole Blood values
  - Specifies if plasma or whole blood values were used in the Report
- Patient's Age and Sex - if entered in Patient Setup
- Patient's Doctor

## Statistics Panel

A statistics panel is included in all reports except for the Insulin, Health Checks, Exception Reports, and any **Add-in Reports**. The Panel provides a quantitative summary of the data that is included in a particular Report.

When you see this in the Statistics Panel:	It means this:
Glucose Average	The average of all glucose readings in the Report.
% Within Target	The percentage of glucose readings in the Report that are within the Before-Meal and After-Meal target ranges.
# of Glucose Readings	The total number of glucose readings included in the Report. Omitted and Deleted readings are not included in the calculations.
Number of Hypo Readings	The total number of readings that were at or below the Hypoglycemic threshold.
Standard Deviation	A measure of the variability (spread) between individual glucose readings and the Glucose Average calculated above. The higher the variability, the higher the standard deviation.
Target Type	Specifies the Diabetes Type for the patient covered by the Report and whether the results are compared to global targets or personal targets.
Before-Meal and After-Meal Targets, and Hypoglycemic Settings	Specifies the glucose target ranges and Hypoglycemic threshold used for reporting purposes. You may click on the text links to go to the Global Targets Tab of the Preferences screen, or the Patient Setup if personal settings are in use, to modify the settings.

## Footnotes

A footnotes link will appear at the bottom of any Report when glucose readings covered in the Report Date Range were:

- Above the linear range of the meter that recorded the reading. These readings are designated as HIGH.
- Below the linear range of the meter that recorded the reading
- Modified from their original values
- Omitted from the Report

## Colors and Shading in Reports

Several Reports use color, shading and typefaces to display data or to differentiate data when viewing or printing Reports.

### Viewing Reports:

- The following reports use separate cell colors and **bold** type to highlight individual readings that are outside the target ranges: Pie Chart, Logbook Report, Data List Report, Summary Report, and the Day View screens.
- Different cell colors are used for above target readings, below target readings, and hypoglycemic readings. A legend that explains the meaning of the colors will appear on every Report that has out-of-target readings.
- Insulin doses that were transferred from a meter are highlighted in regular type. Doses that were taken according to a OneTouch® DMS Regimen will appear in **bold** type.
- Several Report graphs also show the Overall (Target) Range as a shaded gray band.

### Printing Reports:


- Reports may be printed in color or black and white.
  - **Color printing:** Identical color schemes are used for viewing and printing reports.
  - **Black and white printing:** In the Logbook Report, the Data List Report, the Summary Report, and Day View screens, out-of-target readings are highlighted as **bold italics**. The Pie Chart Report uses fill patterns to differentiate out-of-target and within-target breakdowns. A legend that explains the meaning of the different fill patterns will appear on every Report that has out-of-target readings.
- The default setting for printing is color. To change this setting, deselect color print from the File, Print Options menu.
- The Overall (Target) Range will appear as horizontal dashed lines.
- Insulin doses that were transferred from a meter are highlighted in regular type. Doses that were taken according to a OneTouch® DMS Regimen will appear in **bold** type.

## Choose Your Report

### Logbook Report

The Logbook Report displays daily glucose readings and other data in a logbook format (see Figure 3.4a). Readings are sorted in chronological order by date and time, and organized into Meal Slots. Insulin, carb, exercise, and Health Note data are also included. This Report helps you easily track glucose with health related data that can impact glucose readings. Out-of-target glucose readings are displayed by colored cells, helping to easily spot trends relative to a particular time of day.

**Figure 3.4a**

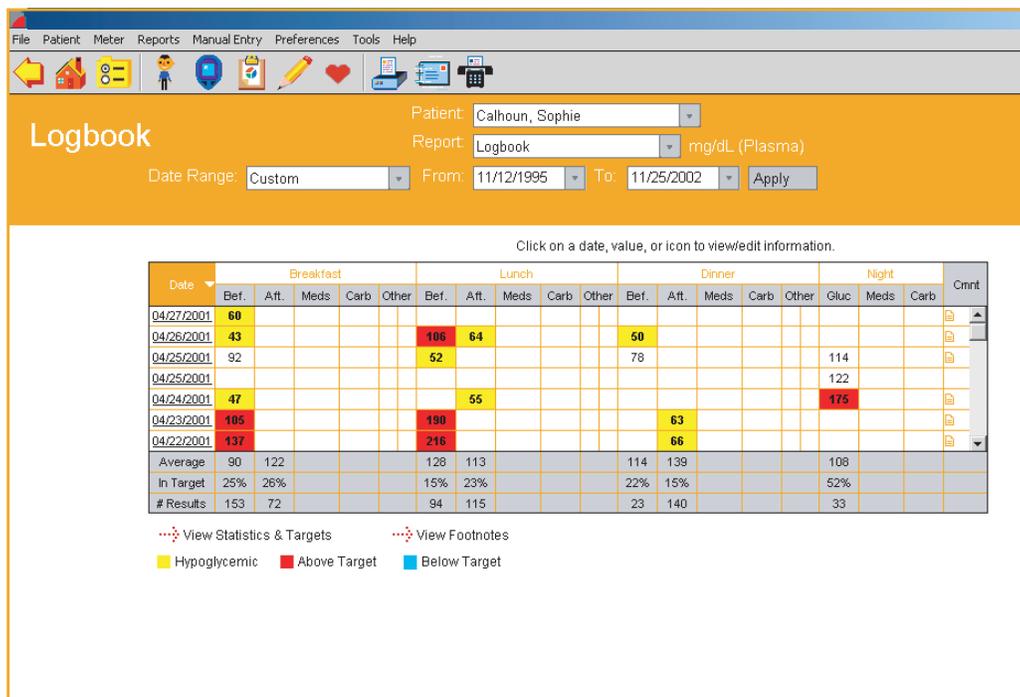

**Figure 3.4b**

The screenshot shows the 'Record Details' window in the OneTouch (R) Diabetes Management Software v2.3. The window title is 'OneTouch (R) Diabetes Management Software v2.3' and the menu bar includes 'File', 'Patient', 'Meter', 'Reports', 'Manual Entry', 'Preferences', 'Tools', and 'Help'. The patient name is 'Doe, John'. The 'Record Details' section has tabs for 'Glucose', 'Insulin', 'Oral Meds', 'Exercise', and 'Food'. The 'Glucose' tab is active, showing a date of '10/20/2001', time of '12:00 PM', slot of 'After Breakfast', and type of 'Finger Tip'. The glucose value is '53' mg/dL, with an option for 'HIGH' and radio buttons for 'Plasma' and 'Whole Blood'. The 'Glucose Comments' section includes checkboxes for 'Illness', 'Vacation', 'Stress', 'Feel Hypo', 'Menses', 'Food', 'Exercise', and 'Exercise type', each with sub-options. A 'Comments' text box is at the bottom, and buttons for 'Omit', 'Cancel', 'Add New Record', and 'Done' are at the bottom right.

## Sorting Data

The Logbook Report may be sorted by date from most recent to oldest, or vice versa. The default order is most recent to oldest. Click on the arrow next to the word Date on the top left of the Report to change the order.

## Modifying Data

You may modify data records while viewing the Logbook Report. Click on any glucose, insulin, carb, exercise, or Health Note value and you will go to the **Record Details** screen ([see Figure 3.4b](#)) where you can add comments or change the meal slot tagged to the reading. Please see the sections on *Modifying Data Records* and *Manually Enter Data* for more information on modifying or adding data records.

## Assigning Data to a Meal Slot

Data are grouped into Meal Slots. Times for each Meal Slot are set in the Schedule Tab of the Patient Setup screen. The Meal Slots are as follows:

- Before-Breakfast
- After-Breakfast
- Before-Lunch
- After-Lunch
- Before-Dinner
- After-Dinner
- Night

The Logbook Report groups the data further into “Breakfast”, “Lunch”, “Dinner”, and “Night” categories. Glucose readings will appear in their appropriate Meal Slot. Insulin, carb, exercise, and health note data will be grouped into the “Breakfast”, “Lunch”, “Dinner” and “Night” Slots only.

## Glucose Readings

- Glucose readings are organized by Meal Slot. Times for Meal Slots are set in the Schedule Tab of the Patient Setup screen.
  - Multiple glucose readings for the same Meal Slot in any given day will be displayed as sequential logbook entries within the same Date.
  - All records in the Before-Breakfast through Night meal slot series will be displayed sequentially with the date of the Before-Breakfast start time. This includes records that may occur after midnight.
- Readings higher than 600 mg/dL (or 500 mg/dL for the OneTouch® SureStep®/ GlucoTouch®/GlucoTouch® Plus Meters) are displayed as HIGH in the Report. HIGH values will be converted to 601 mg/dL and 501 mg/dL (for OneTouch® SureStep®/ GlucoTouch®/GlucoTouch® Plus Meters) when used in calculations. If the display units are in mmol/L, the corresponding converted values are 33.4 mmol/L and 27.8 mmol/L.
- Readings lower than 20 mg/dL (1.1 mmol/L) for OneTouch FastTake®/ PocketScan®/ EuroFlash®/SmartScan™/ OneTouch® Ultra®/ OneTouch® Ultra 2/ OneTouch® UltraSmart® and InDuo® Meters, are displayed as LOW in the Report. LOW values will be converted back to their recorded values when used in calculations.
- Readings outside the Before-Meal and After-Meal target ranges, or below the Hypoglycemic threshold are highlighted by **bold** type and differentiating cell colors. A dynamic legend that explains the meaning of the colors will appear on the bottom of the Report.
- The Total Number of glucose readings, the Average Glucose Level, and the Within Target % will appear at the bottom of each Meal Slot.

## Insulin Data

Insulin doses are organized into Breakfast, Lunch, Dinner and Night and will appear as abbreviations within the column labeled **Meds**. The abbreviations for the corresponding Insulin Types appear in a legend:

- Insulin doses that were transferred from a meter are highlighted in regular font. Insulin doses that were taken according to a OneTouch® DMS Regimen will appear in **bold** font.

## Carb Data

The total carb units for that Meal Slot are displayed within the column labeled **Carbs**.

## Exercise Data

An **Exercise** icon will appear in the column labeled **Other** if any Exercise results are recorded in that slot. A legend descriptor for the icon will appear at the bottom of the Report.

## Health Notes Data

A **Heart** icon will appear in the column labeled **Other** if there are Health Notes recorded for that Meal Slot. A legend descriptor for the icon will appear at the bottom of the Report.


## Comments

A **Note** icon will appear in the column labeled **Cmnt** if any Comments are tagged to any record appearing in that row of the Report.

## The Day View screen of the Logbook Report

The Logbook Report provides access to the Day View screen (see [Figure 3.5](#)). This screen lets you display detailed information for any day in the Report. Multiple results for any Meal Slot will be included, as well as other Food, Exercise, and Health Record data that does not appear in the Logbook Report.

**Figure 3.5**

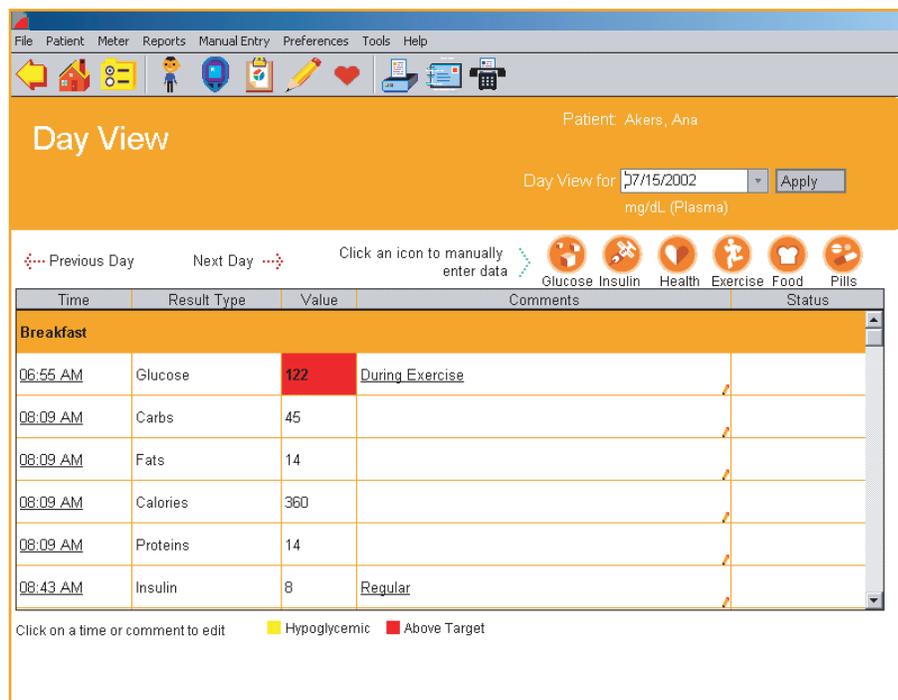

You may also modify data records by clicking on any of the Time or Comment links associated with any row of the Report. When you click on a link, OneTouch® DMS takes you to the Record Details screen where you can modify data records. Please see the sections on *Modifying Data Records and Manually Enter Data* for more information on modifying or adding data records.

- To access the Day View screen from the Logbook Report, click on any of the Date links. The Day View screen for that particular day will be displayed.
- Data records are grouped by the following Meal Slots:
  - Breakfast
  - Lunch
  - Dinner
  - Night

- Within each Meal Slot grouping, records are ordered chronologically from oldest to most recent. The order may not be changed in the Day View screen.
- Clicking on **Time** links associated with any data record will take you to the Record Details screen where you can modify data records. Data entry fields will be pre-filled with the existing data record. You may make modifications as desired.
- Similar to other OneTouch® DMS Reports, glucose readings outside the Before-Meal and After-Meal target ranges or below the Hypoglycemic Value will be highlighted in **bold** type and cell color.
- Comments will appear in the **Comments** column for any data record that has been tagged. Clicking on **Comment** links associated with any data record will take you to the Record Details screen where you can modify data records. Data entry fields will be pre-filled with the existing data record. You may make modifications as desired.
- New data records may be added from the Day View screen. Simply click on the **Glucose, Insulin, Medications, Exercise, Food, or Health** icons on the top right of the screen. This will take you to the appropriate Manual Entry screen.
- The Status of any special data records is displayed in the **Status** column as follows:
  - “Added” if the data record was entered manually
  - “Omitted” if the data record is marked for omission from Reports and calculations
  - “Edited” if the data record has been modified. (If a data record is “Edited” and then “Omitted”, the Status is “Omitted”)
- The Day View screen may be printed.

## Glucose Trend Report


The Glucose Trend Report displays daily glucose readings for a selected Date Range (see [Figure 3.6](#)). This Report helps you easily track changes in glucose readings from day to day.

- Glucose readings are displayed as dots on an X-Y graph. Each reading is connected to the next with a dashed line.

**Note:** The dashed line that connects blood sugar readings in this graph does not reflect actual glucose levels. Blood sugars can go high or low throughout the day especially around mealtimes, medication or exercise. Actual testing before and after these events is required to get a better picture of how they affect your sugar levels.

- Readings will appear in chronological order from oldest to most recent. The X-axis represents the date and time of the reading and the Y-axis represents the actual glucose level. All glucose readings for a particular day are displayed. When printing the Glucose Trend Report, readings are displayed as dots and are connected by solid lines.
- The shaded gray area on the graph represents the Overall (Target) Range. Any readings that fall above or below the shaded gray area are considered outside of this range. When printing the Glucose Trend Report, the Overall (Target) Range limits will appear as horizontal dashed lines.
- You may point to any dot to display the value, Date, Time, Slot and Comments tagged to that result.

**Figure 3.6**

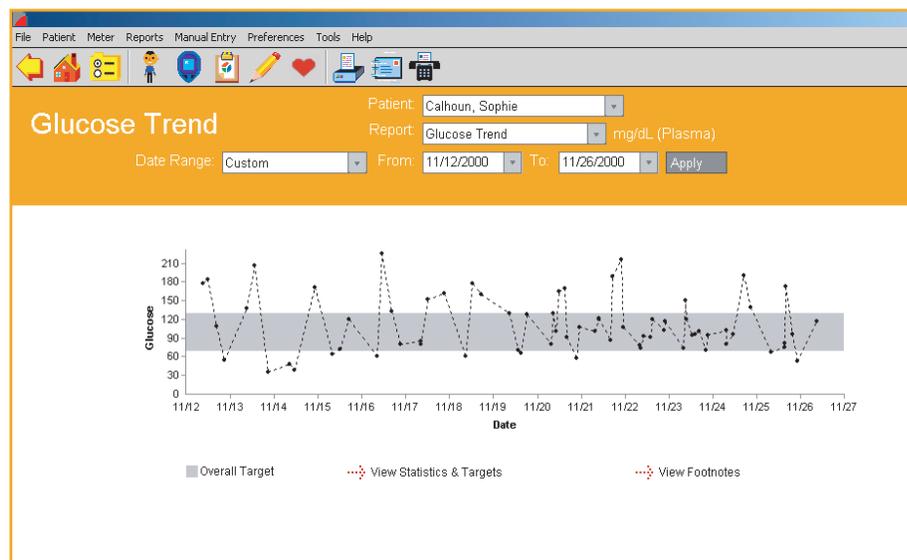


## Pie Chart Report

The Pie Chart Report displays the percentage of glucose readings that fall above, below and within each Before-Meal and After-Meal target range. This Report helps you easily see how glucose readings compare with target ranges for each Meal Slot ([see Figure 3.7](#)).

- A total of eight pie charts will appear in the Report. These represent the Before-Meal and After-Meal breakdowns for each Meal Slot, one for Night and one for Overall Target Range.
- Pie charts for each Before-Meal and After-Meal pairing will appear side by side. This helps you scan for trends within a particular Meal Slot.
- Chart pairings are also stacked vertically so you can compare Before-Meal percentages and After-Meal percentages across all Meal Slots.
- The Pie Chart Report uses applicable Before-Meal and After-Meal target ranges to display an Overall Total range. The Overall Total pie chart provides the percentage breakdowns using all glucose readings within and outside the Overall (Target) Range.


**Figure 3.7**

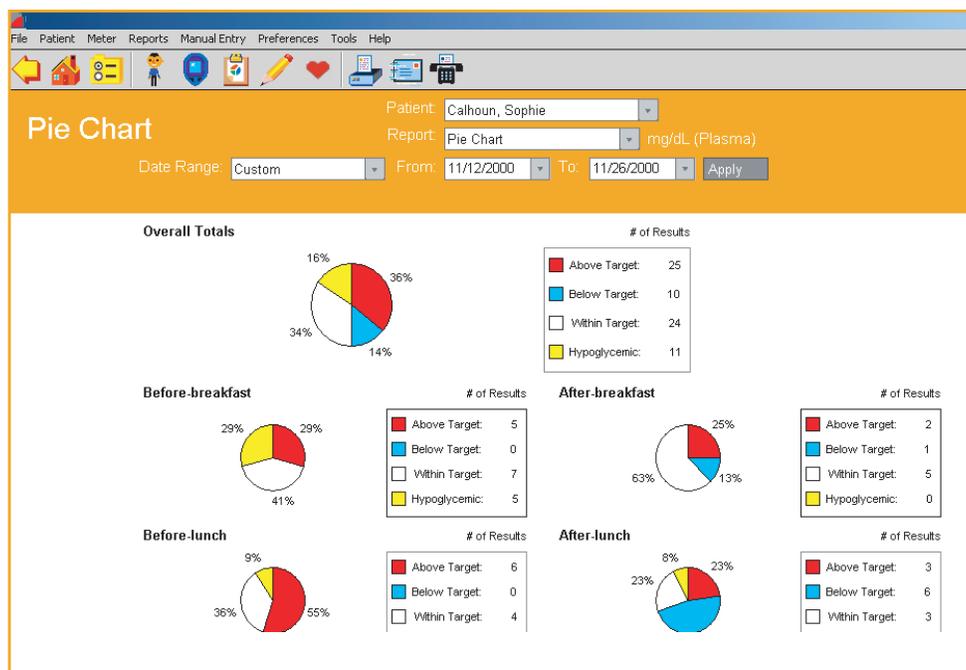

## Summary Report

The Summary Report provides an overview for quick analysis of glucose readings and related health data over the selected date range (see [Figure 3.8](#)). It includes the Logbook Report, Glucose Trend Report, and an overview of the Pie Chart Report. A link to a Statistics Panel is also included. Only the Logbook portion is shown below.

**Figure 3.8**

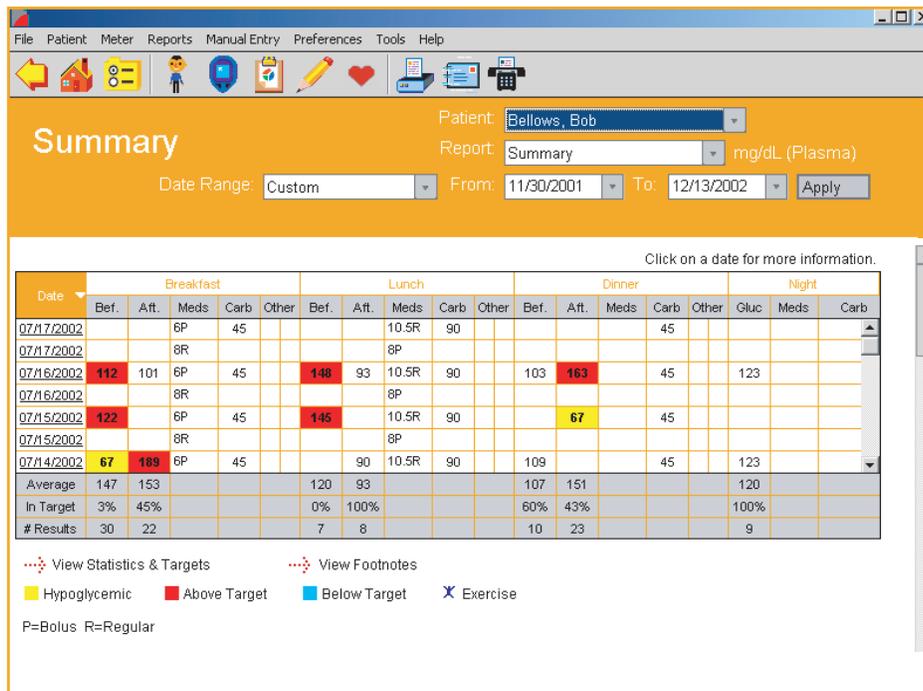


**Note:** All records in the Before-Breakfast through Night meal slot series will be displayed sequentially with the date of the Before-Breakfast start time. This includes records that may occur after midnight.

## Standard Day Report

The Standard Day Report displays glucose readings over a 24-hour period ([see Figure 3.9](#)). This Report helps you easily identify patterns in glucose readings by time of day.

- Glucose readings are displayed as dots on an X-Y graph. Readings from the entire date range will be plotted in one graph. The X-axis represents the time of the reading and the Y-axis represents the actual glucose level.
- The shaded gray area on the graph represents the Overall (Target) Range. Any readings that fall above or below the shaded gray are considered outside of this Overall (Target) Range. When printing the Standard Day Report, the Overall (Target) Range limits will appear as horizontal dashed lines.
- You may use ToolTips to display the value, Date, Time, Slot and Comments tagged to any result.

**Figure 3.9**

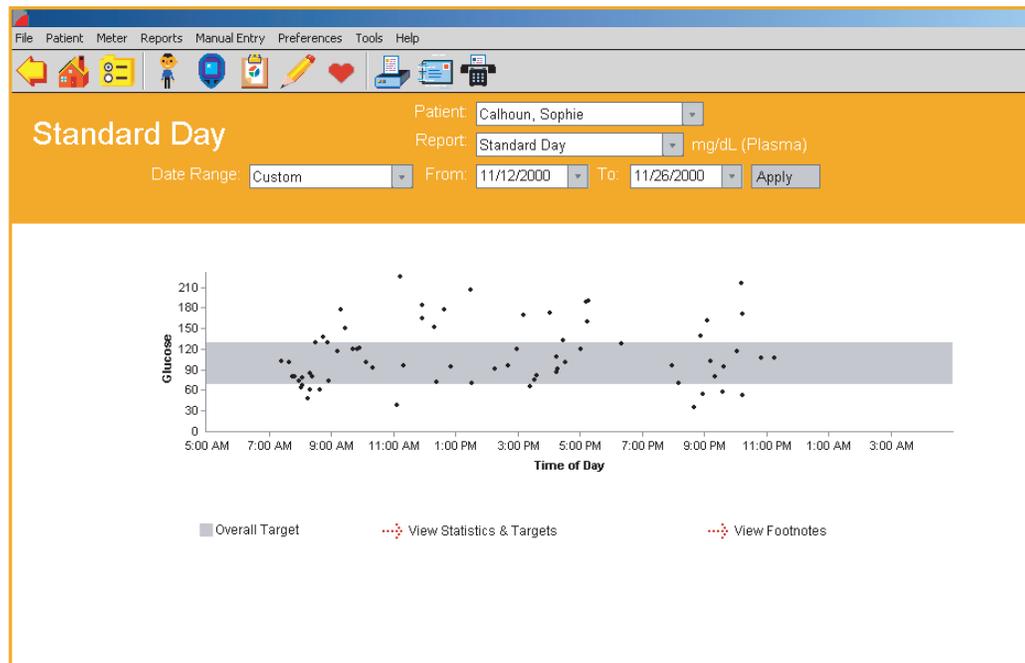


## Insulin Report

The Insulin Report displays average glucose readings, insulin units, and carbohydrates by Meal Slot ([see Figure 3.10](#)). This Report helps you easily analyze the relationship between glucose readings, insulin dosing, and carbohydrate intake.

The Insulin Report is composed of two graphs that are stacked on top of one another. The same X-axis (Breakfast, Lunch, Dinner and Night Meal Slots) is used for both graphs. The Y-axis represents the actual glucose, insulin or carb value.

- The top graph displays the average glucose reading by Meal Slot. Glucose averages are represented as dots and are connected by a dashed line. **Note:** The dashed line that connects average blood sugar readings in this graph does not reflect actual average glucose levels.
- The bottom graph displays average insulin units and carb intake by Meal Slot. Each insulin and carb average is represented as a separate bar. If there are multiple Insulin Types for any Meal Slot, the averages will appear as side-by-side bars. Only Insulin Injection and Bolus amounts are used in calculating the averages. Pump Daily Total amounts are not included. Insulin data transferred from a meter or entered manually for any Meal Slot will override any insulin units defined in the Regimen Tab of the Patient Setup screen. Colors will be used to differentiate insulin averages for each insulin type, and to differentiate insulin averages from carb averages.

**Figure 3.10**

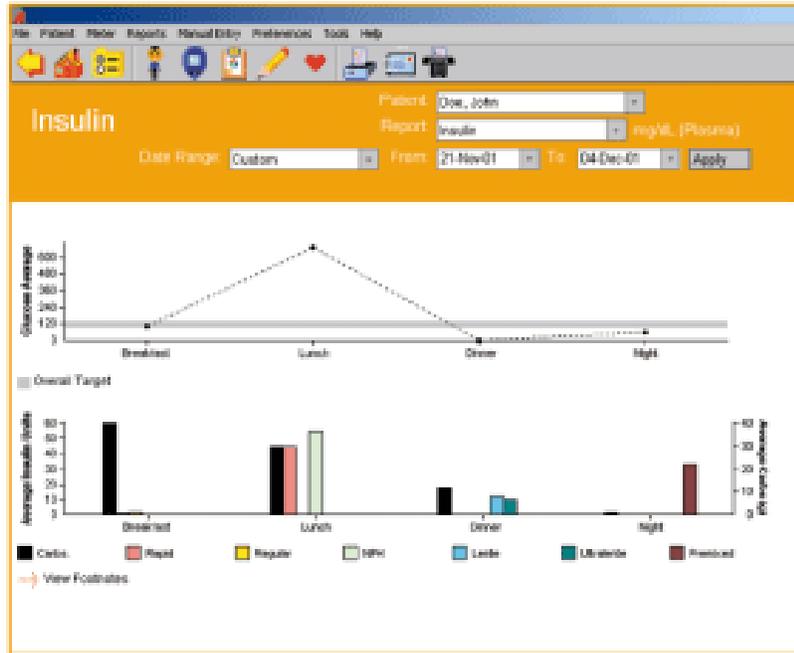


## Health Checks Report


The Health Checks Report displays results from diabetes-related tests and exams ([see Figure 3.11](#)). Results may be from doctor exams, lab tests, and other at-home tests. This Report helps you easily monitor how other health factors are impacted by diabetes.

- The Health Checks Report includes the following result types:
  - Cholesterol (HDL, LDL, and Total)
  - Triglycerides
  - Microalbumin
  - Ketones
  - HbA1c
  - Height
  - Weight
  - Blood Pressure
  - Dr. Visit (Office Visit, Eye Exam, Foot Exam)
- For each Health Checks Result Type (for example, Ketones), the corresponding Value and Date will appear.
- Multiple results may exist for any Result type but only the most recent result is displayed. However, the Health Checks Detail screen lets you view all results for a particular result type ([see Figure 3.12](#)). To access the Health Checks Details screen, click on any of the result type links. The screen will display all records for a given result type in both log and graph format.

**Figure 3.11**

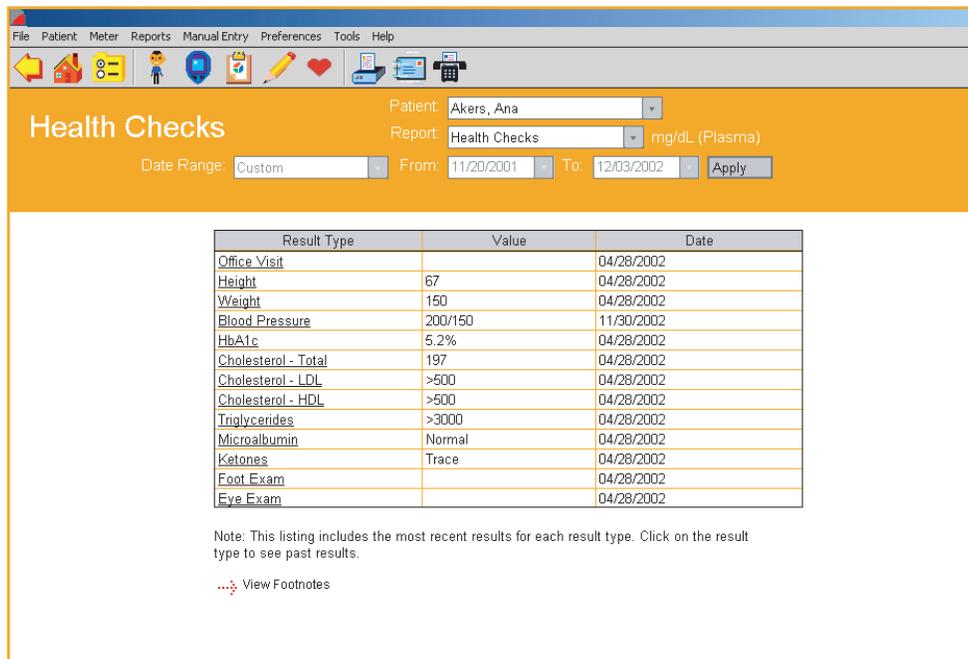

**Figure 3.12**

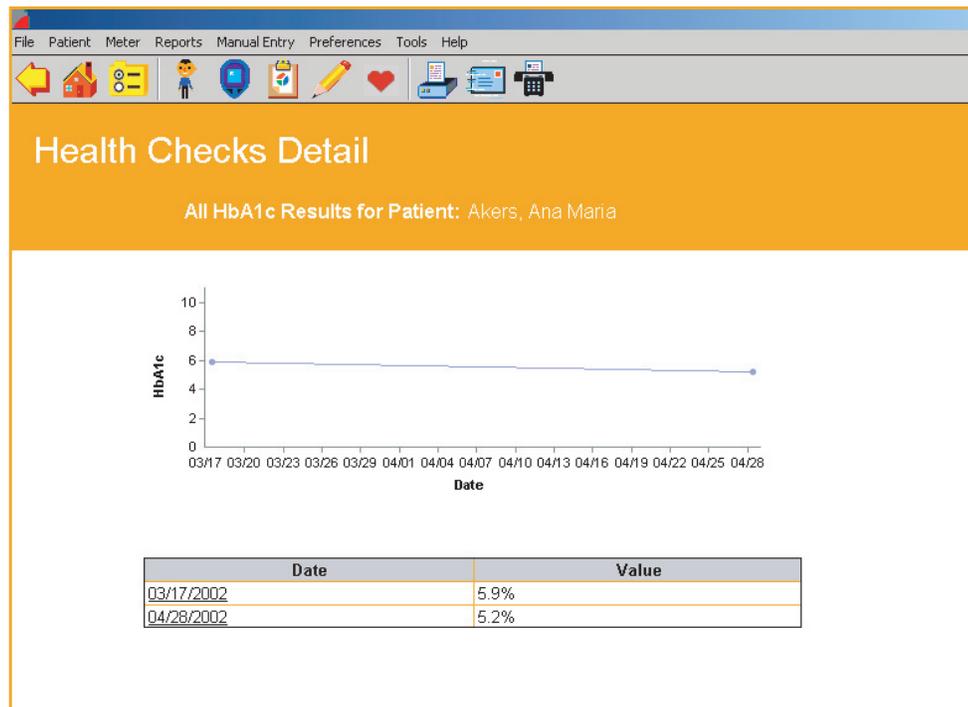


## Average Readings Report

The Average Readings Report displays glucose averages by Meal Slot, exercise period, and day of the week ([see Figure 3.13](#)). This Report helps you easily monitor how glucose readings are impacted by meals, exercise, and day.

Glucose averages are presented in 3 separate bar charts that are stacked vertically.

- The top chart displays glucose averages by Meal Slot for each day of the week. The X-axis represents the day of the week and the Y-axis represents the glucose average.
- The middle chart displays glucose averages by Meal Slot. The X-axis represents the Breakfast, Lunch, Dinner, and Night Meal Slots, and the Y-axis represents the glucose average. Before-Meal and After-Meal period averages are displayed as side-by-side bars for each Meal Slot.
- The bottom chart displays glucose averages by Exercise period for each day of the week. Readings you have tagged with exercise comments are displayed here. The X-axis represents the day of the week and the Y-axis represents the glucose average. Before Exercise, During Exercise, and After Exercise period averages are displayed as side-by-side bars for each day of the week.

**Figure 3.13**

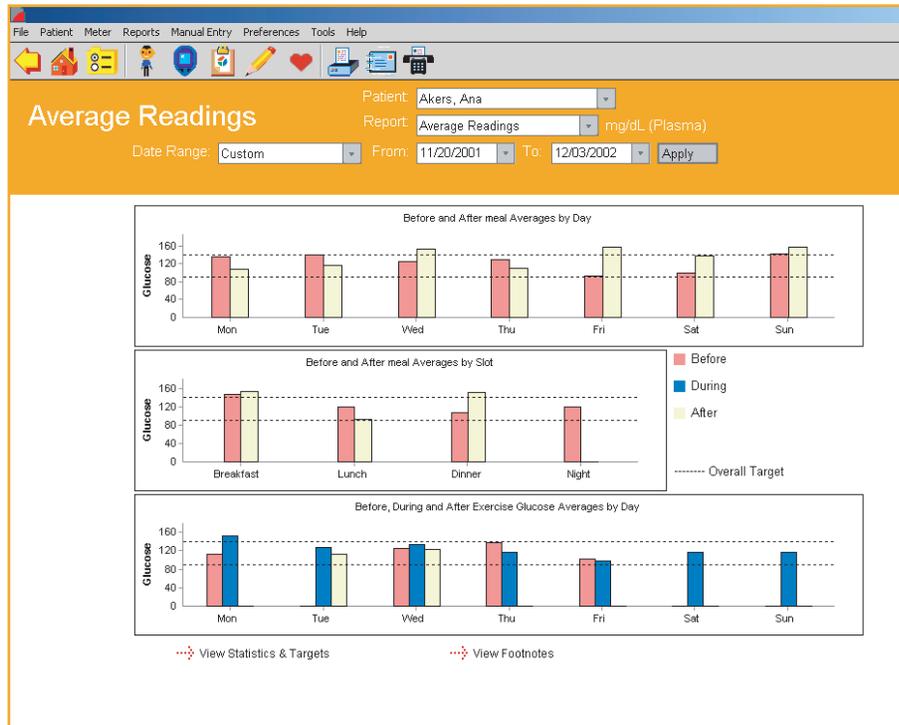


## Exception Report

The Exception Report displays key diabetes-related information for all patients in the OneTouch® DMS database (see [Figure 3.14](#)). This Report helps you easily flag patients who may need to be managed more closely. It can also be used to document improved standards of care in a practice. Finally, Reports can be generated for subsets of patients, i.e. those under a given insurance plan. If you enter insurance information for each patient, you can run a report including all patients with a given carrier.

Report data may be sorted by the following criteria by clicking on the sort arrow in the correct column:

- Patient Name
- Total number of glucose readings in the date range
- Percentage of glucose readings below the target ranges
- Percentage of glucose readings above the target ranges
- Total number of hypoglycemic readings
- HbA1c
  - By Value or Date
- Dr. Visit by Date
- Eye Exam
- Foot Exam

**Figure 3.14**

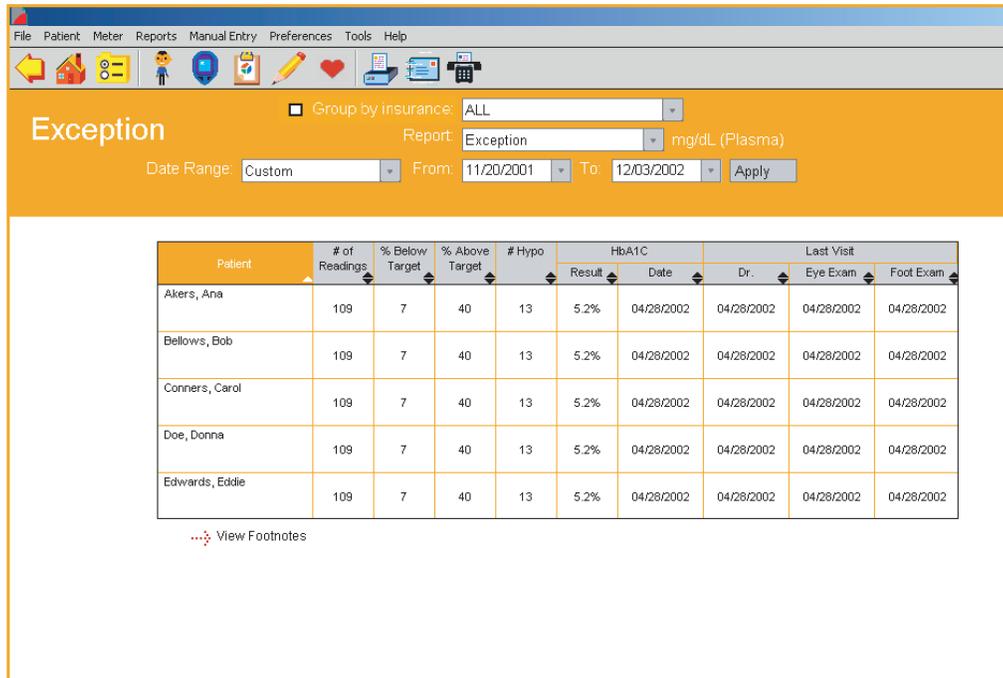

## Data List Report

The Data List Report displays a sequential list of all glucose, insulin, food, exercise, and other health data stored in the OneTouch® DMS database (see [Figure 3.15](#)). All Comments and any Edited, Omitted, or Added Status tags will be included. This Report helps you easily monitor all relevant data for any patient in the database.

**Note:** While insulin values entered manually or transferred from a meter are displayed in the Data List Report, insulin regimen values are not. Please refer to the following summary table to review how insulin values are displayed in the Data List Report.

Source of Insulin Data	Displayed in the Data List Report
Transferred from a meter	YES
Added manually in OneTouch® DMS	YES
Insulin Regimen	NO

■ Data may be sorted by the following:

- Date/Time
  - OneTouch® DMS recognizes the date and time of glucose results based on how they were tagged by the meter. In some cases, meter date and time may have been incorrectly set by the user so that the calendar date sequence of results may not match the actual order that the tests were taken.
  - In the Data List Report (“Since last transfer”), you have the option to display glucose results in the order that the tests were taken, rather than by calendar date. This feature is the default option in the Software. This feature is not available if you have transferred results from a OneTouch® UltraSmart® Meter.
  - To view and print Data List Reports in calendar date order, first deselect this option in the *Report Options* tab under *Preferences*. Then select the “Since last transfer” Data Range under Data List Reports.
- Result Type
- Meter Serial Number (if included in the Report parameters)

- You may also modify data records from within the Data List Report by clicking on any of the date or time links associated with any result type. When you click on a link, OneTouch® DMS takes you to the Record Details screen with the appropriate Tab already selected. Please see the sections on *Editing Data Records* and *Manually Enter Data* for more information on modifying or adding data records.
- Glucose readings outside the Before-Meal and After-Meal target ranges, or below the Hypoglycemic Value are highlighted by **bold** type and differentiating cell colors. A legend that explains the meaning of the colors will appear on the bottom of the Report.

**Figure 3.15**

Click on a Date or Time or Comment Column to view/edit information.

Date	Time	Slot	Result Type	Value	Comments	Status
11/30/2002	06:29 AM	Before Breakfast	Blood Pressure	200/150		Added
07/17/2002	05:09 PM	Before Dinner	Fats	14		
07/17/2002	05:09 PM	Before Dinner	Calories	360		
07/17/2002	05:09 PM	Before Dinner	Proteins	14		
07/17/2002	05:09 PM	Before Dinner	Carbs	45		
07/17/2002	12:37 PM	Before Lunch	Pump Daily Total	14		


■ Hypoglycemic   
 ■ Above Target   
 ■ Below Target   
 [View Statistics & Targets](#)   
 [View Footnotes](#)

## Histogram Report

The Histogram Report highlights before-meal and after-meal patterns together with glucose distribution for a given date range (see [Figure 3.16](#)). This Report helps you easily monitor the frequency of glucose readings that fall within and outside a set of ranges.

- Glucose readings from Before-Meal, After-Meal and Night slots are presented as stacked bars on an X-Y chart. The X-axis represents the pre-defined glucose ranges and the Y-axis represents the total number of glucose readings in a Meal Slot, for that particular range.

**Figure 3.16**

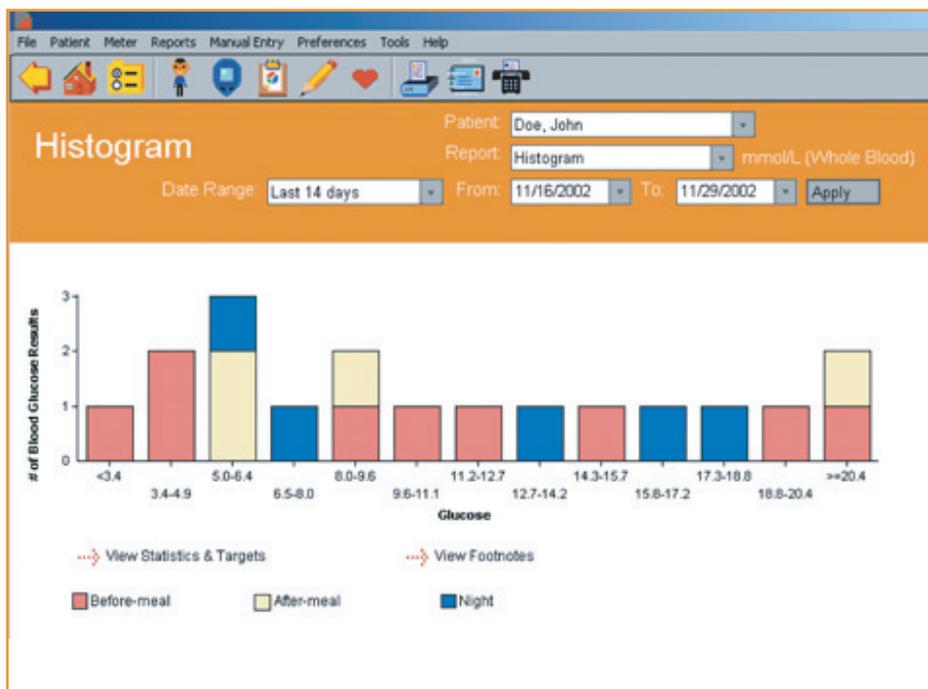

## Printing, Faxing and E-Mailing OneTouch® DMS Reports

OneTouch® DMS helps you print all Reports for long-term record keeping. This includes the ability to print the Day View screen and the Health Checks Drill-Down screen.

You can also fax and e-mail Reports to other people with whom you want to share the data.

### To print any Report:

You have several options for printing reports.

#### 1. Print directly from a report

- Display the Report, click on the **File** Menu Bar Command, and then select **Print Report** from the drop-down menu, or
- Simply click on the **Printer** icon on the Tool Bar.

#### 2. Print without viewing first

- From the **Choose Your Report** main screen, click on any number of the Select boxes that appear under the Report buttons.
- Click on the **Print Selected Reports** button to print the desired Reports.

The default setting for printing is color. To change this setting, deselect color print from the File, Print Options menu.

### Printing Add-in Reports

Add-in reports can be printed directly from the report through the browser printing feature. Add-in reports cannot be printed without viewing first.

**To fax any Report:**

1. Display the Report in the **Choose Your Report** main screen
2. Click on the **Telephone** icon on the Tool Bar. A Fax cover sheet will appear.
3. Follow the information in the table below to identify which fields require entries and which are optional

If you see this:	The information is:	Comments:
Date	Required	Pre-filled
Time	Required	Pre-filled
Recipient's Name	Optional	---
Recipient's Phone Number	Optional	---
Receiving Fax Number	Required	The Program automatically stores up to five fax numbers. Select a number from the drop-down menu.
Sender's Name	Required	---
Sender's Telephone Number	Required	---
Notes	Optional	The sender can add a text message of up to 100 characters.

4. Click Send

**Note:** You can only fax from OneTouch® DMS if your PC has a connected fax modem.

## To e-mail any Report:

1. Display it first in the **Choose Your Report** main screen
2. Click on the **Letter** icon on the Tool Bar. An e-mail message will appear with your report included as an attachment.
  - Enter the e-mail address, a short message, and then send.

## Notes:

- Guest patient data in any Report cannot be e-mailed.
- You can only e-mail reports directly from OneTouch® DMS if you have a MAPI-compliant e-mail client installed on your PC, and an internet connection. Check the Help file for details on how to mail reports with a non-MAPI e-mail system.

## Modify Data Records (Edit, Omit, Restore, and Delete)

OneTouch® DMS lets you make the following types of changes to records in the database:

- Edit records
- Omit records from Reports and calculations
- Restore records that have been previously marked as omitted
- Permanently delete records

Editing may be done whenever you are in the Logbook Report, the Data List Report, the Day View screen of the Logbook Report, or the Health Checks Detail screen. Click on any of the time, value, icons or comment links provided on screen. This will take you to the Record Details screen where you may modify the data record. Many of the data entry fields will be pre-filled with the existing values. Make your edits as necessary.

Omitting, restoring, and deleting data records may be done in the Record Details screen (similar to the Manual Entry screens).

**Note:** The buttons that will appear on the bottom of the Record Details screens may be different than the buttons that appear on the Manual Entry screens. That's because you can only add (not modify) records to the database if you go to the Manual Entry screens directly from the home screen.

## Editing Records

You may edit OneTouch® DMS data records, but there are some limitations. Your ability to edit is determined by the record type and the types of data included in the record.

If the data record :	Then you can edit these data values:
Was transferred from a meter	<ul style="list-style-type: none"> <li>■ Comments tagged to the record</li> </ul> <p>You may not edit any other data values.</p>
Was entered manually	<ul style="list-style-type: none"> <li>■ The Meal Slot tagged to the record</li> <li>■ Comments tagged to the record</li> <li>■ Date of record</li> <li>■ Time of record</li> <li>■ Record type</li> <li>■ Record value</li> </ul>
Unsupported results	These cannot be edited.

After you edit a data record, it is tagged as “edited” in the Status column that will appear in some OneTouch® DMS Reports. Footnotes at the bottom of the Report will indicate that the Report includes readings that have been edited.

## Omitting Records

You may want to omit a record that is misleading (for example, if a friend borrowed your meter to run a test). When you omit a record, it is marked in the software to be left out of calculations and reports.

**Note:** Records transferred from your meter cannot be deleted from the Software. Omitted records will always remain in the database. The Omit function simply excludes omitted records from calculations and reports (except the Data List Report, see below).

Footnotes at the bottom of reports will indicate if some readings that would have been included in the report have been omitted. If you want to view which records were Omitted from a report, go to the Data List Report. This Report shows all data records in the database, including those marked as omitted. If the word "Omitted" appears in the Status column, the record was marked to be left out of calculations and reports.

In contrast: any record that was manually entered using the computer can be deleted permanently.

## Restoring Records

You may re-include data records that have been tagged as omitted. When viewing the Day View screen for the date of the omitted record, click on either the comment icon or the time link on the row of the record with the "omitted" tag. Then click on the **Include** button in the Record Details screen. The omitted tag will be removed from the Status column of the OneTouch® DMS Reports.

## Deleting Records

You may permanently delete some data records from the OneTouch® DMS database. Only manually entered data records can be deleted. Records transferred from a meter cannot be deleted.

## Manually Enter Data

The Manual Entry screens let you add new data records to the OneTouch® DMS database. You may also modify data records through the Manual Entry screens when accessed via the OneTouch® DMS Reports. The editing function is described in the *Modifying Data Records* section of the User Manual.

There are two Manual Entry screens, Manual Entry-A and Manual Entry -B.

- The Manual Entry-A screen ([see Figure 3.17](#)) contains five Tabs labeled **Glucose**, **Insulin**, **Oral Meds**, **Exercise**, and **Food**.
- The Manual Entry-B screen contains only one Tab for **Health Records** ([see Figure 3.18](#)). On this screen, you may add diabetes-related health data. These data records can help provide a more complete picture of diabetes health status. Data entry will be similar to that of the other data records. Click on the **Go to Other Records** link to return to the Manual Entry-A screen.

Both Manual Entry screens include a link to the other.

### Whenever you manually add a record, the following rules apply:

- Records will be tagged as “added” in the OneTouch® DMS database. This will differentiate the record from others that have been transferred from a meter. That tag will appear in the Status column of the Day View screen in the Logbook and Data List Reports. A footnote will also appear at the bottom of each Report where “added” data records are included.
- Records will include a “Time” stamp that is accurate to the nearest second
- The Software restricts you from adding duplicate results

Date, Time and Result (value) are required data entry fields for adding any record type. Comments are optional.

**Figure 3.17**

OneTouch (R) Diabetes Management Software v2.3

File Patient Meter Reports Manual Entry Preferences Tools Help

Patient: Doe, John

**Manual Entry - A**

Glucose Insulin Oral Meds Exercise Food [Go to Health Records](#)

Date: 8/16/2006  
 Time: 11:56 AM  
 Slot: Before Lunch  
 Type: Finger Tip

Glucose:  Or:  HIGH mg/dL  Plasma  Whole Blood

Glucose Comments

<input type="checkbox"/> Illness	<input type="checkbox"/> Food	<input type="checkbox"/> Exercise	<input type="checkbox"/> Exercise type
<input type="checkbox"/> Vacation	<input type="radio"/> Different Food	<input type="radio"/> Before	<input type="radio"/> Mild
<input type="checkbox"/> Stress	<input type="radio"/> Fasting	<input type="radio"/> During	<input type="radio"/> Hard
<input type="checkbox"/> Feel Hypo	<input type="radio"/> Too much food	<input type="radio"/> After	<input type="checkbox"/> Medication
<input type="checkbox"/> Menses	<input type="radio"/> Not enough food		<input type="checkbox"/> Other

Comments:

Save Cancel Add New Record Done

**Figure 3.18**

OneTouch (R) Diabetes Management Software v2.3

File Patient Meter Reports Manual Entry Preferences Tools Help

Patient: Conners, Carol

**Manual Entry - B**

[Go to Other Records](#)

Type: HbA1c


Date: 12/04/2002  
 Time: 04:25 AM


Result:  (%)

Comments:

Save Cancel Add New Record Done

You have several options to get to the Manual Entry screens where you can add a record by clicking on the appropriate Tab:

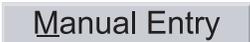When you see this:	You can do this:
<p>On the Home Screen</p> 	<ul style="list-style-type: none"> <li>■ Click on the icon and you will go directly to the Manual Entry-A screen. Click on any of the 5 Tabs labeled <b>Glucose, Insulin, Oral Meds, Exercise,</b> and <b>Food</b> to enter data.</li> <li>■ If you would like to add a Health Record, click on the <b>Go to Health Records</b> link. You will go to the Manual Entry-B screen.</li> <li>■ Click on the <b>Go to Other Records</b> link to return to the Manual Entry-A screen.</li> </ul>
<p>On the Menu Bar</p> 	<ul style="list-style-type: none"> <li>■ Click on <b>Manual Entry</b>. Then select <b>Glucose, Insulin, Oral Meds, Exercise, Food,</b> or <b>Health Records</b> from the drop-down menu to add a particular record type.</li> </ul>
<p>On the Tool Bar</p> 	<ul style="list-style-type: none"> <li>■ Click on the <b>Pencil</b> icon and you will go directly to the Manual Entry-A screen. Click on any of the five Tabs labeled <b>Glucose, Insulin, Oral Meds, Exercise,</b> and <b>Food</b> to enter data.</li> </ul>
<p>On the Tool Bar</p> 	<ul style="list-style-type: none"> <li>■ Click on the <b>Heart</b> icon and you will go directly to the Manual Entry-B screen. Here you can enter data for <b>Health Records</b>.</li> </ul>

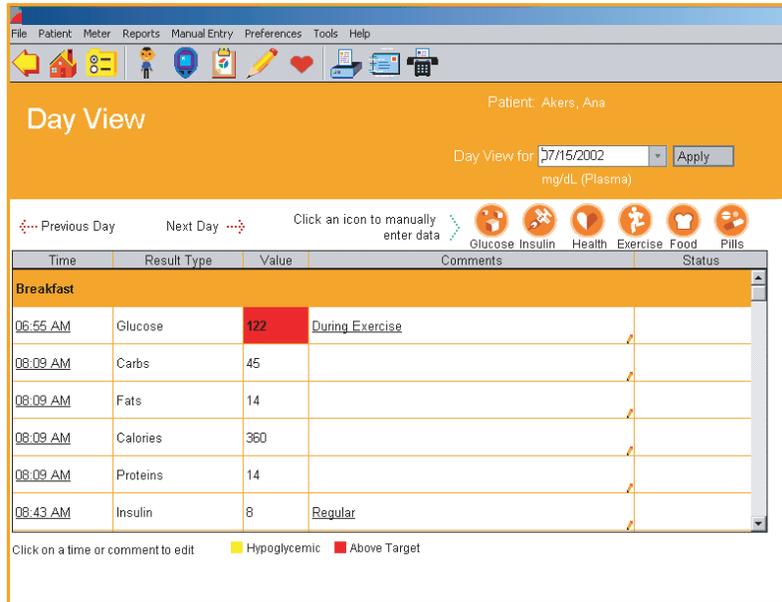**Figure 3.19**

## The Day View screen of the Logbook Report

You have 2 options for adding data records from the Day View screen ([see Figure 3.19](#)):

1. Click on any highlighted links for Glucose, Insulin, Pills, Exercise, Food, or Health record values. You will go either directly to the Manual Entry-A screen with that particular Tab selected, or to the Manual Entry-B screen.

If you clicked on any link other than Health Records, you will go to the Manual Entry-A screen with that Tab selected. This will let you modify that particular data record (please see the *Modifying Data Records* section). If you would like to add a data record, click on the **Add New Result** button. This will activate all Tab options where you may add Glucose, Insulin, Oral Meds, Exercise and Food records.

If you clicked on Health Records, you will go to the Manual Entry-B screen. Click on the **Go To Other Results** link to return to the Manual Entry-A screen.

2. Click on the **Glucose, Insulin, Health Record, Exercise, Food** or **Pills** icons on the top of the Day View screen. You will either go directly to the Manual Entry-A screen with that Tab already selected, or to the Manual Entry-B screen. Click on the **Add New Records** button to activate any other of the Tabs.

If you clicked on Health Records, you will go to the Manual Entry-B screen. Click on the **Go To Other Results** link to return to the Manual Entry-A screen.

## The Glucose Tab

The Glucose Tab lets you add Glucose Records to the OneTouch® DMS database (see [Figure 3.20](#)).

**Figure 3.20**

OneTouch (R) Diabetes Management Software v2.3

File Patient Meter Reports Manual Entry Preferences Tools Help

Manual Entry - A Patient: Doe, John

Glucose Insulin Oral Meds Exercise Food Go to Health Records

Date: 8/16/2006  
Time: 11:56 AM  
Slot: Before Lunch  
Type: Finger Tip

Glucose:  Or:  HIGH mg/dL  Plasma  Whole Blood

Glucose Comments

<input type="checkbox"/> Illness	<input type="checkbox"/> Food	<input type="checkbox"/> Exercise	<input type="checkbox"/> Exercise type
<input type="checkbox"/> Vacation	<input checked="" type="radio"/> Different Food	<input checked="" type="radio"/> Before	<input checked="" type="radio"/> Mild
<input type="checkbox"/> Stress	<input checked="" type="radio"/> Fasting	<input checked="" type="radio"/> During	<input checked="" type="radio"/> Hard
<input type="checkbox"/> Feel Hypo	<input checked="" type="radio"/> Too much food	<input checked="" type="radio"/> After	<input type="checkbox"/> Medication
<input type="checkbox"/> Menses	<input checked="" type="radio"/> Not enough food		<input type="checkbox"/> Other

Comments:

Save Cancel Add New Record Done

When you see this:	You can do this:
Date and Time	<p>Set the Date and Time for the new Glucose Record. Both are required data entry fields. You may modify the information in the date field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month and year.</p> <p>You may modify the information in the Time field by entering the desired time, or by clicking on the hour or minute values that appear. Then click on the up/down arrows until you find the desired hour and minute.</p>
Meal Slot	<p>Tag the new Glucose Record to a Meal Slot. Click on the down arrow where you can select any of the 7 pre-defined Meal Slots.</p>
Glucose Reading	<p>Specify the glucose reading for the new Glucose Record. The glucose reading is a required data entry field. You must enter a value or click on <b>HIGH</b> in the check box.</p> <ul style="list-style-type: none"> <li>■ Values may be in 1-unit increments for mg/dL and 0.1-unit increments for mmol/L.</li> <li>■ If you click on HIGH, it specifies that the glucose reading was above the linear range of the meter.</li> </ul> <p>You must specify Plasma or Whole Blood values for any glucose reading that is manually added.</p>
Type	<p>Specify the type of glucose reading. Click on the down arrow in the <b>Type</b> field. A drop-down menu of options will appear:</p> <ul style="list-style-type: none"> <li>■ Fingertip</li> <li>■ Alternate site</li> <li>■ Laboratory</li> <li>■ Control</li> </ul> <p><b>Click</b> on the desired option.</p>

When you see this:	You can do this:
Glucose Comments	<p>Assign a Comment to the Glucose Record. You may select from pre-defined Glucose Comments or create a new Comment in your own words.</p> <p><b>Click</b> on any pre-defined Glucose Comment that applies:</p> <ul style="list-style-type: none"> <li>■ Illness</li> <li>■ Vacation</li> <li>■ Stress</li> <li>■ Feel Hypo</li> <li>■ Menses</li> <li>■ Food (select only 1 below) <ul style="list-style-type: none"> <li>• Different Food</li> <li>• Fasting</li> <li>• Too much food</li> <li>• Not enough food</li> </ul> </li> <li>■ Exercise (select only 1 below) <ul style="list-style-type: none"> <li>• Before</li> <li>• During</li> <li>• After</li> </ul> </li> <li>■ Exercise type (select only 1 below) <ul style="list-style-type: none"> <li>• Mild</li> <li>• Hard</li> </ul> </li> <li>■ Medication</li> <li>■ Others</li> </ul> <p><b>Or, enter</b> new information about your glucose reading in the Comments field.</p>

**Note:** Additional Comment options may be displayed, depending on the meter you are using. Refer to the Owner's Booklet for that meter for information on comment options.

## The Insulin Tab

The Insulin Tab lets you add Insulin Records to the OneTouch® DMS database (see [Figure 3.21](#)).

**Figure 3.21**

The screenshot shows the 'Manual Entry - A' window in the OneTouch DMS software. The window title is 'Manual Entry - A' and the patient name is 'Conners, Carol'. The 'Insulin' tab is selected, and the 'Date' is set to 12/04/2002, 'Time' to 04:20 AM, and 'Slot' to Night. The 'Amount' field is empty, and the 'Type' is set to 'Injection'. The 'Bolus' radio button is selected. The 'Comments' field is empty. The 'Save', 'Cancel', 'Add New Record', and 'Done' buttons are visible at the bottom.

### When you see this:

### You can do this:

#### Date and Time

Set the Date and Time for the new Insulin Record. You may modify the information in the Date field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month and year.

You may modify the information in the Time field by entering the desired Time, or by clicking on the hour or minute values that appear. Then click on the up/down arrows until you find the desired hour and minute.

#### Meal Slot

Tag the new Insulin Record to a Meal Slot. Click on the down arrow where you can select any of the 7 pre-defined Meal Slots.

When you see this:	You can do this:
Injection/Bolus/Pump Daily Total	<p>Specify the insulin dosing method types and units. Click on the selection box that applies.</p> <p>“Injection” is the default selection.</p> <p>If you select “Injection”:</p> <ul style="list-style-type: none"> <li>■ Select an Insulin Type from the drop-down menu in the field. Insulin Types were created in the Medications Tab of the Patient Setup screen. If you want to add a new insulin type to the menu list, click on the <b>Add Insulin</b> link. This will take you to the Medications Tab of the Patient Setup screen where you can add new Insulin Types.</li> <li>■ Enter an amount from 0 to 250 in 0.1-unit increments.</li> </ul> <p>If you select “Bolus”:</p> <ul style="list-style-type: none"> <li>■ Enter an amount from 0 to 60 in 0.1-unit increments.</li> </ul> <p>If you select “Pump Daily Total”:</p> <ul style="list-style-type: none"> <li>■ Enter an amount from 0 to 100 in 0.1-unit increments.</li> </ul>
Insulin Comments	Enter any information about your insulin record in the Comments field.

## The Oral Meds Tab

The Oral Meds Tab lets you add Oral Medication Records to the OneTouch® DMS database (see [Figure 3.22](#)).

**Figure 3.22**

The screenshot displays the 'Manual Entry - A' window in the OneTouch DMS software. The window title bar includes 'File Patient Meter Reports Manual Entry Preferences Tools Help'. Below the title bar is a toolbar with various icons. The main content area is titled 'Manual Entry - A' and features a patient selection dropdown set to 'Conners, Carol'. A navigation bar at the top of the main area includes tabs for 'Glucose', 'Insulin', 'Oral Meds' (which is currently selected), 'Exercise', and 'Food', along with a 'Go to Health Records' link. The 'Oral Meds' form contains the following fields: 'Date' (12/04/2002), 'Time' (04:22 AM), 'Slot' (Night), 'Type' (dropdown), and 'Amount' (text input). Below these fields is an 'Add Oral Meds' button and a 'Comments' text area. At the bottom of the window are four buttons: 'Save', 'Cancel', 'Add New Record', and 'Done'.

When you see this:	You can do this:
Date and Time	<p>Set the Date and Time for the new Medications Record. Both are required data entry fields. You may modify the Date in the field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.</p> <p>You may modify the Time in the field by entering the desired time, or by clicking on the hour or minute values that appear. Then click on the up/down arrows until you find the desired hour and minute.</p>
Meal Slot	<p>Tag the new Medications Record to a Meal Slot. Click on the down arrow where you can select any of the 7 pre-defined Meal Slots.</p>
Type	<p>Specify the type and amount (number of pills) of medication. Click on the down arrow in the Type field. A drop-down menu of Medication types will appear. Select the desired one. The Types of Medications are set in the Medications Tab of the Patient Setup screen.</p> <p>If you do not see the desired Medication Type, you may click on the <b>Add Oral Meds</b> link. This will take you to the Medications Tab of the Patient Setup screen where you may add a new Medication Type.</p> <p>Specify the Medication amount (number of pills) by entering a value in the Amount field.</p>
<b>Medications Comments</b>	<p>You may create and assign a new text Comment to the Medications Record in the Comments field.</p>

## The Exercise Tab

The Exercise Tab lets you add Exercise Records to the OneTouch® DMS database (see [Figure 3.23](#)).

**Figure 3.23**

The screenshot shows the 'Manual Entry - A' window in the OneTouch DMS software. The window title is 'Manual Entry - A' and the patient name is 'Conners, Carol'. The interface has a menu bar with 'File', 'Patient', 'Meter', 'Reports', 'Manual Entry', 'Preferences', 'Tools', and 'Help'. Below the menu bar is a toolbar with various icons. The main content area is divided into tabs: 'Glucose', 'Insulin', 'Oral Meds', 'Exercise', and 'Food'. The 'Exercise' tab is selected. The 'Exercise' tab contains the following fields: 'Date' (12/04/2002), 'Time' (04:23 AM), 'Duration' (hrs and min), 'Intensity' (dropdown), and 'Comments' (text area). At the bottom of the window are four buttons: 'Save', 'Cancel', 'Add New Record', and 'Done'.

When you see this:	You can do this:
Date and Time	<p>Set the Date and Time for a new Exercise Record. Both are required entries for any new exercise record. There are no default values.</p> <ul style="list-style-type: none"> <li>■ Modify the Date in the field by directly entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.</li> <li>■ Modify the Time in the field directly by entering the desired time, or by selecting/clicking on the hour or minute values that appear. Then click on the up/down arrows until you find the desired hour and minute.</li> </ul>
Duration and Intensity	<p>Enter the Duration and Intensity of the exercise period. Both are required entry fields for any new exercise record. There are no default values.</p> <ul style="list-style-type: none"> <li>■ Specify exercise intensity by clicking on the down arrow in the Intensity field. Select the desired intensity level from the drop-down menu that will appear: <ul style="list-style-type: none"> <li>• Mild</li> <li>• Moderate</li> <li>• Hard</li> </ul> </li> <li>■ Specify exercise duration by entering the time period in hours and minutes in 5-minute increments.</li> </ul>
Exercise Comments	<p>You may create and assign a new text Comment to the Exercise Record in the Comments field.</p>

## The Food Tab

The Food Tab lets you add Food Records to the OneTouch® DMS database ([see Figure 3.24](#)).

**Figure 3.24**

The screenshot shows the 'Manual Entry - A' window in the OneTouch Diabetes Management Software. The window title is 'Manual Entry - A' and the patient is 'Conners, Carol'. The 'Food' tab is selected, and the following fields are visible:

- Date: 12/04/2002
- Time: 04:24 AM
- Meal: [Dropdown]
- Carbs: [Text Input]
- Fats: [Text Input]
- Calories: [Text Input]
- Proteins: [Text Input]
- Comments: [Text Input]

Buttons at the bottom include Save, Cancel, Add New Record, and Done.

When you see this:	You can do this:
Date and Time	<p>Set the Date and Time for the new Food Record. Both are required data entry fields for any new food record. There are no default values.</p> <ul style="list-style-type: none"> <li>■ Modify the Date in the field by directly entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.</li> <li>■ Modify the Time in the field by entering the desired time, or by clicking on the hour or minute values that appear. Then click on the up/down arrows until you find the desired hour and minute.</li> </ul>
Meal	<p>Tag the Food Record to a Meal Slot.</p> <ul style="list-style-type: none"> <li>■ Click on the down arrow where you can select Breakfast, Lunch, Dinner or Snack. You can also select Alcohol as a food type, to record the date and time when you had a drink.</li> </ul>
Carbs, Fats, Calories, and Proteins	<p>Specify values for the Carbohydrates, Fats, Calories, and Proteins boxes. At least one food type is required for any new food record.</p> <ul style="list-style-type: none"> <li>■ If you are entering Carbs, Proteins, or Fats, enter a value from 0 to 250 in 1-unit increments.</li> <li>■ If you are entering Calories, enter a value from 0 to 2500 in 5-unit increments.</li> </ul>
Food Comments	<p>Enter notes about your food record in the Comments field.</p>

## The Health Records Tab

The Manual Entry-B screen lets you add Health Records to the OneTouch® DMS database (see [Figure 3.25](#)).

**Figure 3.25**

The screenshot shows the 'Manual Entry - B' screen in the OneTouch DMS software. The patient name is 'Conners, Carol'. The form is titled 'Health Records' and includes a 'Go to Other Records' link. The form fields are: Type (HbA1c), Date (12/04/2002), Time (04:25 AM), Result (empty), and Comments (empty). The bottom of the screen has buttons for Save, Cancel, Add New Record, and Done.

You may add the following types of Health Records:

- HbA1c
- Dr. Visits
- Health Notes
- Ketones
- Microalbumin
- Cholesterol
- Blood Pressure (Systolic and Diastolic)
- Height and Weight

You must enter a Date, Time, and Result (value) for each Health Record that you add. Date and Time are specified in the same manner as with adding other data records.

When you see this:	You can do this:
HbA1c	Specify HbA1c (A1c) test result as follows: <ul style="list-style-type: none"> <li>■ Enter a value from 4 to 15 in 0.1-unit increments.</li> </ul>
Dr. Visits	Enter dates for any of the following types of Dr. Visits: <ul style="list-style-type: none"> <li>■ Office Visit</li> <li>■ Foot Exam</li> <li>■ Eye Exam</li> </ul>
Health Notes	Specify one or more of the following types of Health Notes: <ul style="list-style-type: none"> <li>■ Stress</li> <li>■ Feel Hypo</li> <li>■ Illness</li> <li>■ Menses</li> <li>■ Vacation</li> <li>■ Other</li> </ul>
Ketones	Specify one of the following Ketone test results: <ul style="list-style-type: none"> <li>■ Negative</li> <li>■ Trace</li> <li>■ Small</li> <li>■ Moderate</li> <li>■ Large</li> </ul>

When you see this:	You can do this:
Microalbumin	<p>Enter a Microalbumin test as either a result type or as an actual value.</p> <ul style="list-style-type: none"> <li>■ Result type choices are: <ul style="list-style-type: none"> <li>• Normal</li> <li>• Positive</li> <li>• &gt;1000</li> </ul> </li> <li>■ An actual value may be entered from 1 to 1000 in 1-unit increments.</li> </ul>
Blood Pressure	<p>Enter Blood Pressure test results. Both Systolic (the higher number) and Diastolic (the lower number) result are required entries for any new blood pressure record.</p> <ul style="list-style-type: none"> <li>■ Enter a Systolic result from 80 to 200 in 1-unit increments.</li> <li>■ Enter a Diastolic result from 40 to 150 in 1-unit increments.</li> </ul>
Height and Weight	<p>Enter height and weight values as follows:</p> <ul style="list-style-type: none"> <li>■ Heights must be from 0 to 255</li> <li>■ Weights must be from 0 to 600</li> <li>■ Both height and weight may be entered in 0.5 unit increments</li> <li>■ There are no units associated with height and weight records. You may enter values for height and weight in pounds and inches or kilograms and centimeters.</li> </ul>

When you see this:	You can do this:
Cholesterol	<p>Specify one or more of the following types of Cholesterol tests:</p> <ul style="list-style-type: none"> <li>■ Total</li> <li>■ LDL</li> <li>■ HDL</li> <li>■ Triglycerides</li> </ul> <p>You may enter Cholesterol test results as follows:</p> <ul style="list-style-type: none"> <li>■ For Total Cholesterol <ul style="list-style-type: none"> <li>• An actual value between 0 and 1000 mg/dL (0 to 25.9 mmol/L) or</li> <li>• Specify "&gt;1000" ("&gt;25.9" in mmol/L)</li> </ul> </li> <li>■ For LDL <ul style="list-style-type: none"> <li>• An actual value between 0 and 500 mg/dL (0 to 13.0 mmol/L) or</li> <li>• Specify "&gt;500" ("&gt;13.0" in mmol/L)</li> </ul> </li> <li>■ For HDL <ul style="list-style-type: none"> <li>• An actual value between 0 and 500 mg/dL (0 to 13.0 mmol/L) or</li> <li>• Specify "&gt;500" ("&gt;13.0" in mmol/L)</li> </ul> </li> <li>■ For Triglycerides <ul style="list-style-type: none"> <li>• An actual value between 0 and 3000 mg/dL (0 to 33.9 mmol/L) or</li> <li>• Specify "&gt;3000" ("&gt;33.9" in mmol/L)</li> </ul> </li> </ul> <p>Units are specified in the General tab of the Preferences screen.</p>

## Part 4: OneTouch® DMS Database Management

- Archiving/Retrieving Data Records
- Exporting/Importing Data Records
- Backing Up Data


You have two options for copying OneTouch® DMS database files to other file storage areas on your PC. This feature lets you store data records for safekeeping or use them with other PC applications. The first option is archiving data records and the second option is exporting data records. If you plan to review old data records, the archive option is preferred.

## Archiving/Retrieving Data Records

Archiving lets you transfer data records to a separate file for storage. Archiving also lets you manage the size of the OneTouch® DMS database by removing older data that you may no longer need to review (if you select the Delete Readings After Archive option). This way you can keep the database size at a minimum so the OneTouch® DMS works most efficiently.

You may restore archived data using the Retrieve feature ([see Figure 4.0](#)).

**Figure 4.0**

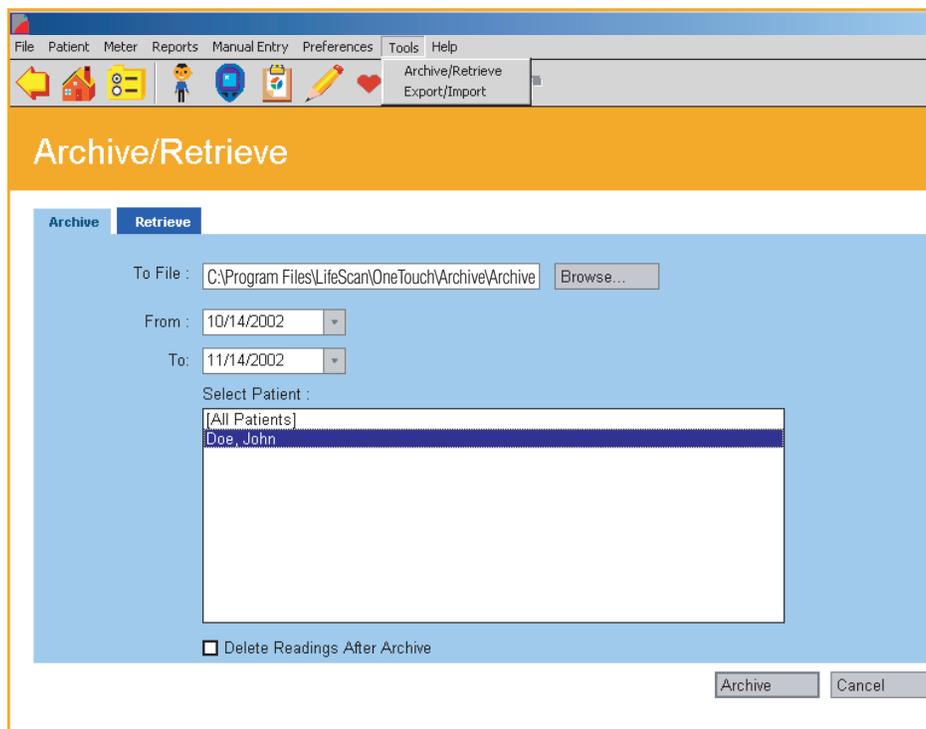

## The Archive Tab

Archiving lets you copy data records from one or more patients in the OneTouch® DMS database. Records are copied to a separate file on your PC.

When you see this:	You can do this:
To File	Specify the name of the file where data will be archived. The default directory for storing archived files is <b>C:\Program Files\LifeScan\OneTouch\Archive</b> . If you enter a new file name, you must include the directory path as the first part of the name. You may also use the Browse function to help locate a file or directory.
From and To	Specify the date range for the data records you want to archive. The <b>From</b> and <b>To</b> dates are required data entry fields. You may modify the dates in the field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.
Select Patient	Specify the patient data records you would like to archive. You may select <b>All Patients</b> , an individual patient, or a group of patients. To select a group of patients, hold the <b>CTRL</b> key down while you select a group of patient names.
Delete Readings After Archive	Specify if you want the archived data records to be deleted from the OneTouch® DMS database. Click on <b>Delete Readings After Archive</b> .


### Notes:

- When archiving data records, all sequential records in the Before-Breakfast through Night meal slot series will be tagged with the date of the Before-Breakfast start time. This includes records that may occur after midnight.
- Archived OneTouch® DMS data will include any “unsupported” results for a given patient and date range.

## The Retrieve Tab

Retrieving lets you place archived data back in the OneTouch® DMS database. You can also retrieve data from an InTouch Software database ([see Figure 4.1](#)).

**Figure 4.1**

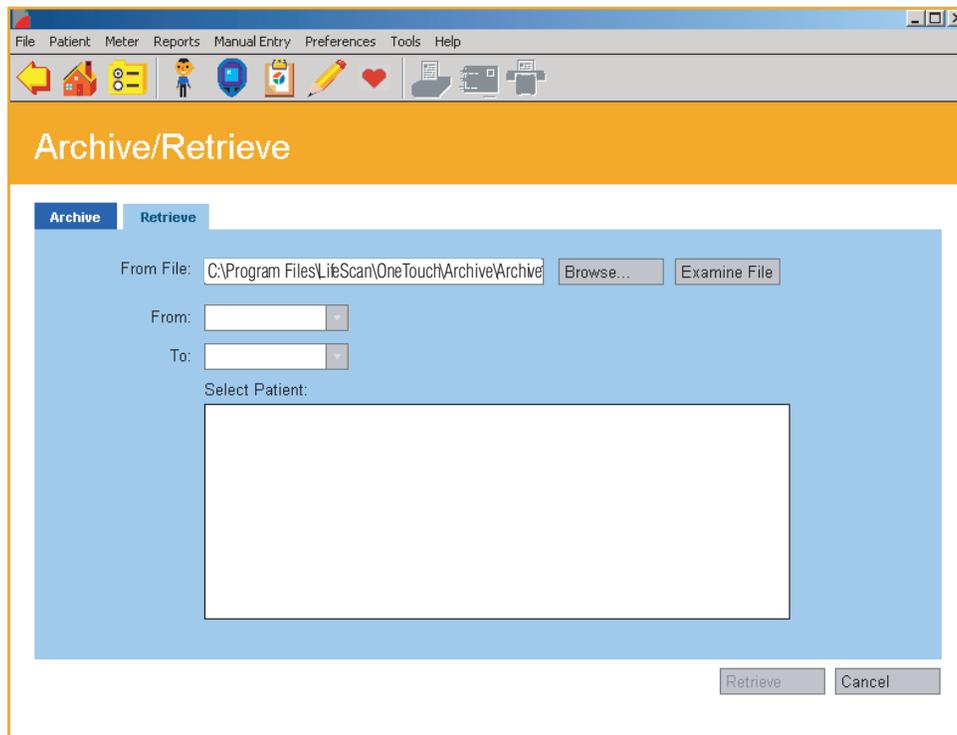

### Notes:

- If you are retrieving data records stored with an older version of OneTouch® DMS into this version of the Software, be aware of differences in the date tags that are assigned to meal slots that cross the midnight hour.
- Retrieved records for a given patient and date range will include any unsupported results.
- The Software does not allow you to retrieve data records from previously archived OneTouch® DMS versions in another language.

When you see this:	You can do this:
From File	<p>Specify the name of the archived file you would like to restore to the OneTouch® DMS database. You may enter the name of the file with the directory path as the first part of the name. You may also use the Browse function to help locate a file or directory.</p> <p>Click on the <b>Examine File</b> button to display the patient names in the archived file before you restore the records to the OneTouch® DMS database.</p>
From and To	<p>Specify the date range for the data records you want to restore. The <b>From</b> and <b>To</b> dates are required data entry fields. You may modify the dates in the field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.</p>
Select Patient	<p>Specify the patient data records you would like to retrieve. You may select <b>All Patients</b>, an individual patient, or a group of patients. To select a group of patients, hold the <b>Ctrl</b> key down while you select a group of patient names.</p>

## Exporting/Importing Data Records

Exporting lets you copy data records to a comma separated value (CSV) file, a common Excel file format, which is compatible with other PC applications. Exported files do not include headers for the various data fields.


You may restore exported data using the Import feature.

**Note:** “Unsupported” results cannot be exported. (See “unsupported results” in the *Glossary*).

### The Export Tab

Exporting lets you copy data records from one or more patients in the OneTouch® DMS database (see [Figure 4.2](#)). Records are copied to a comma separated value (CSV) file on your PC.

**Figure 4.2**

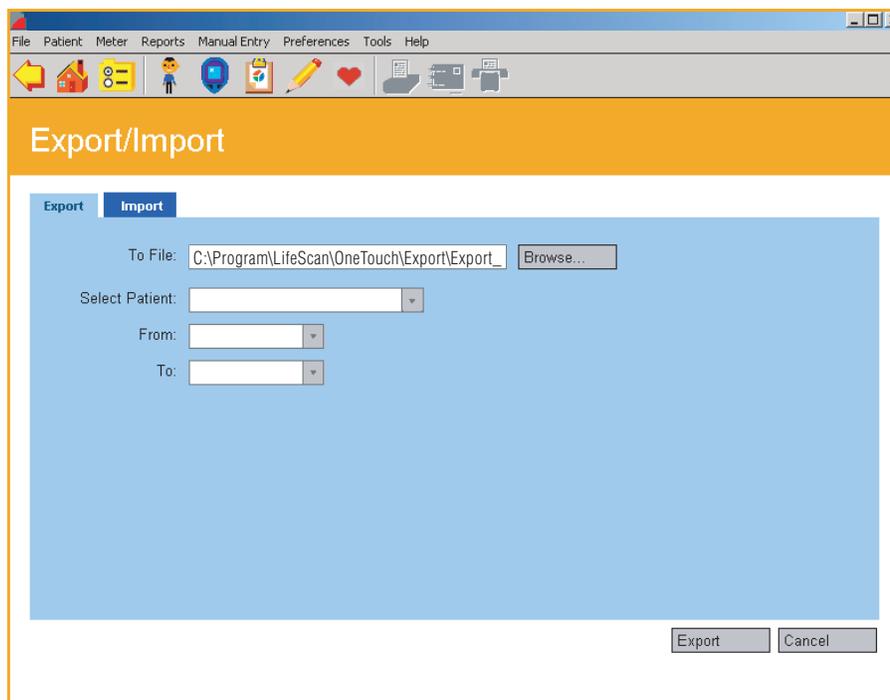

When you see this:	You can do this:
From File	Specify the name of the file where data will be exported. The default directory for storing exported files is <b>C:\Program Files\LifeScan\OneTouch\Export</b> . If you enter a new file name, you must include the directory path as the first part of the name. You may also use the Browse function to help locate a file or directory.
From and To	Specify the date range for the data records you want to archive. The <b>From</b> and <b>To</b> dates are required data entry fields. You may modify the dates in the field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.
Select Patient	Specify the patient whose data records you would like to export. You may export data records for only one patient at a time.

## The Import Tab


Importing lets you place exported data back in the OneTouch® DMS database (see [Figure 4.3](#)). You can import from OneTouch® DMS v2.0, v2.1, v.2.2 files or InTouch® Software export files. You may import data records for only one patient at a time.

If you are importing data records, you must keep the same file format. If you change the file format, you will not be able to put the data records back in the OneTouch® DMS database.

### Notes:

- If you are importing data records stored with an older version of OneTouch® DMS into this version of the Software, be aware of differences in the date tags that are assigned to meal slots that cross the midnight hour.
- The Software does not allow you to import previously exported data records from:
  - OneTouch® DMS versions in another language
  - OneTouch® DMS Pro

**Figure 4.3**

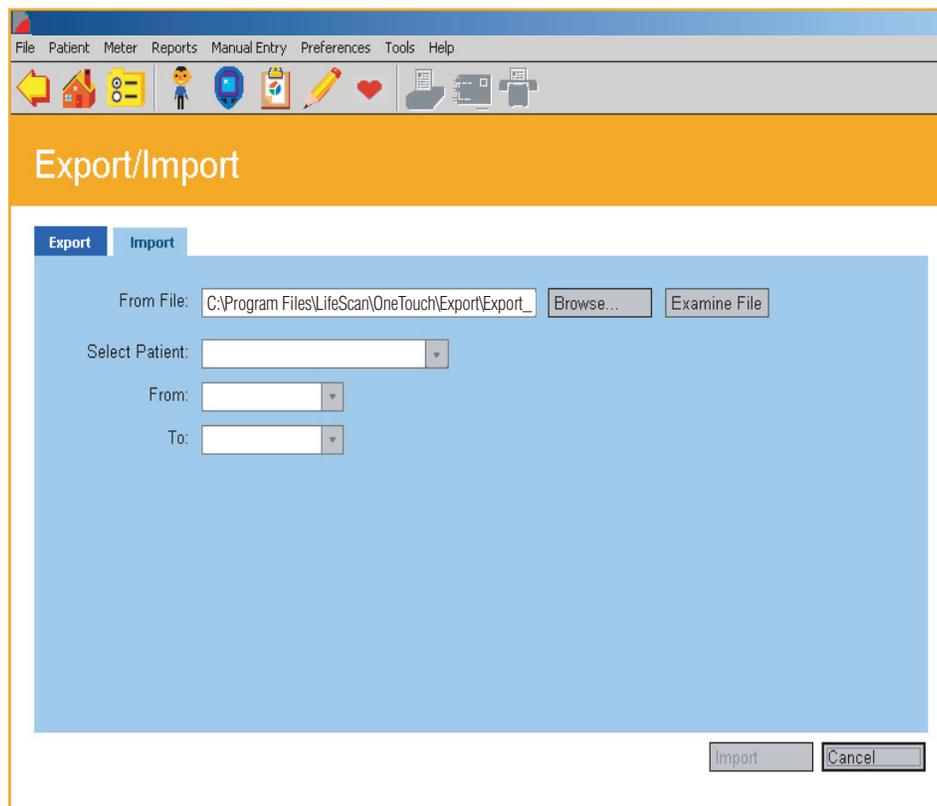

When you see this:	You can do this:
From File	<p>Specify the name of the exported file you would like to restore to the OneTouch® DMS database. You may enter the name of the file with the directory path as the first part of the name. You may also use the Browse function to help locate a file or directory.</p> <p>Click on the <b>Examine File</b> button to display the patient names in the exported file before you restore the records to the OneTouch® DMS database.</p>
From and To	<p>Specify the date range for the data records you want to restore. The <b>From</b> and <b>To</b> dates are required data entry fields. You may modify the dates in the field by entering the desired date. Or you can click on the down arrow. The pop-up calendar box will appear where you can change the day, month, and year.</p>
Select Patient	<p>Specify the patient whose data records you would like to import. To select a patient, click on the down arrow in the field. Select the desired patient. You may import data records for only one patient at a time.</p>

## Backing Up Data

Periodically copy OneTouch.mdb to a network drive, Zip™ disk, tape backup, or other storage media to guard against accidental data loss. Every three months OneTouch® DMS will generate a reminder to backup your database. See the Help File for more information about backing up data.

## Part 5: Appendices

### ■ Appendix A

- [Menu Bar Commands](#)
- [Tool Bar Commands](#)

### ■ Appendix B

- [Glossary](#)

### ■ Appendix C

- [Troubleshooting](#)

### ■ Appendix D

- [Technical Support](#)

### ■ Appendix E

- [Index](#)

## Appendix A: Menu Bar and Tool Bar Commands

Appendix A describes the Menu Bar and Tool Bar commands. Tool Bar and Menu Bar commands provide alternate ways to activate OneTouch® DMS functions.

### Menu Bar Commands

Menu Bar commands appear at the top of the OneTouch® DMS screens. When you click on each of the Menu Bar Commands, a drop-down menu of options will appear. Select the desired action as follows:

When you see this:	You can do this:
	<p><b>Print Options</b></p> <ul style="list-style-type: none"> <li>■ Color Printing - deselect this option for black and white printing.</li> <li>■ Printer Setup - display the Windows® Print Setup box. You can select the printer, paper size and orientation, etc. in this box.</li> </ul> <p><b>Print Report</b> - print the current selected Report.</p> <p><b>Fax Report</b> - fax the current selected Report.</p> <p><b>Email Report</b> - email the current selected Report.</p> <p><b>Exit</b> - close (exit) OneTouch® DMS.</p>

When you see this:	You can do this:
<p><b>Patient</b></p>	<p><b>Add</b> - display the General Tab of the Patient Setup screen. Data entry fields will be blank. You can add a new patient to the database in this Tab.</p> <p><b>Delete</b> - display the General Tab of the Patient Setup screen. Data entry fields will be filled with the current Patient Name. You can delete a patient from the database in this Tab.</p> <p><b>Find</b> - display the Find a Patient Tab of the Patient Setup screen. You can find a patient in the database in this Tab.</p> <p><b>Setup</b> - display the Patient Setup screen. You can view and change diabetes profile data for yourself or your patients in this screen.</p>
<p><b>Meter</b></p>	<p><b>Get Meter Readings</b> - transfer readings from a meter to the database.</p> <p><b>Clear Meter</b> - permanently clear readings from the meter memory.</p> <p><b>Meter Setup</b> - modify meter settings that can impact how data is entered and displayed in the meter.</p>

When you see this:	You can do this:
<p><u>Reports</u></p>	<p>Display any of the following screens:</p> <ul style="list-style-type: none"><li>■ Choose Your Report</li><li>■ Summary Report</li><li>■ Average Readings Report</li><li>■ Data List Report</li><li>■ Exception Report</li><li>■ Glucose Trend Report</li><li>■ Health Checks Report</li><li>■ Histogram Report</li><li>■ Insulin Report</li><li>■ Logbook Report</li><li>■ Pie Chart Report</li><li>■ Standard Day Report</li></ul>

When you see this:	You can do this:
<p data-bbox="240 1115 521 1161">Manual Entry</p>	<p data-bbox="586 369 1398 489"><b>Glucose</b> - display the Glucose Tab of the Manual Entry-A screen. You can add new Glucose Records to the database in this Tab.</p> <p data-bbox="586 527 1409 646"><b>Insulin</b> - display the Insulin Tab of the Manual Entry-A screen. You can add new Insulin Records to the database in this Tab.</p> <p data-bbox="586 684 1409 804"><b>Oral Meds</b>- display the Oral Meds Tab of the Manual Entry-A screen. You can add new Medications Records to the database in this Tab.</p> <p data-bbox="586 842 1398 961"><b>Exercise</b> - display the Exercise Tab of the Manual Entry-A screen. You can add new Exercise Records to the database in this Tab.</p> <p data-bbox="586 999 1365 1119"><b>Food</b> - display the Food Tab of the Manual Entry-A screen. You can add new Food Records to the database in this Tab.</p> <p data-bbox="586 1157 1409 1304"><b>Health Records</b> - display the Manual Entry-B screen. You can select from the cascading menu and add new Health Records to the database in this Tab. You can select from the following drop down menu:</p> <ul data-bbox="651 1341 951 1898" style="list-style-type: none"> <li data-bbox="651 1341 797 1377">■ HbA1c</li> <li data-bbox="651 1415 889 1451">■ Doctor Visits</li> <li data-bbox="651 1488 894 1524">■ Health Notes</li> <li data-bbox="651 1562 818 1598">■ Ketones</li> <li data-bbox="651 1635 911 1671">■ Microalbumin</li> <li data-bbox="651 1709 870 1745">■ Cholesterol</li> <li data-bbox="651 1782 922 1818">■ Blood Pressure</li> <li data-bbox="651 1856 951 1898">■ Height &amp; Weight</li> </ul>

When you see this:	You can do this:
<p><a href="#">Preferences</a></p>	<p><b>General</b> - display the General Tab of the Preferences screen. You can customize the OneTouch® DMS environment in this Tab, including:</p> <ul style="list-style-type: none"> <li>■ Locate the COM Port on your PC</li> <li>■ Display the Meter Communications Steps</li> <li>■ Set the Units and Display Viewing Options</li> </ul> <p><b>Report Options</b> - display the Report Options Tab of the Preferences screen. You can select the default Favorite Report and Date Range in this Tab.</p> <p><b>Global Targets</b> - display the Global Targets Tab of the Preferences screen. You can set glucose target ranges for different Diabetes Types in this Tab.</p> <p><b>Hypoglycemic</b> - display the Hypoglycemic Tab of the Preferences screen. You can set the Hypoglycemic Value in this Tab. This is the baseline at or below which readings are designated as “hypoglycemic” in Reports.</p>
<p><a href="#">Tools</a></p>	<p><b>Archive/Retrieve</b> - archive records from the database to a separate file for storage. Archiving lets you manage the size of the database so you can keep OneTouch® DMS running optimally.</p> <p>Archived data may be restored to the database.</p> <p><b>Export/Import</b> - export data from the database to a comma separated value (CSV) file, a common Excel file format, where it may be used with other PC applications.</p> <p>Exported data may be re-imported to the database.</p>

When you see this:	You can do this:
	<p><b>OneTouch Help</b> - Get help on the current screen.</p> <p><b>LifeScan Technical Support</b> - Learn when and how to contact LifeScan for Technical Support.</p> <p><b>About OneTouch</b> - display basic information about this version of Program Software.</p>

## Tool Bar Commands

Tool Bar icons appear underneath the Menu Bar commands. When you move the mouse cursor over any icon, a field will appear that describes its function. Click on any of the icons to activate the desired function. A description of each icon function follows:

When you click on this:	You can do this:	When you click on this:	You can do this:
	Back up to the previous screen.		Go to the Manual Entry-A screen.
	Go to the Home screen.		Go to the Manual Entry-B screen.
	Go to the Preferences screen.		Print the currently selected Report.
	Go to the Patient Setup screen.		E-mail the currently selected Report.
	Go to the Download Meter Readings Screen.		Fax the currently selected Report.
	Go to your Favorite Report screen		

## Appendix B: Glossary

This appendix describes most of the terms used in OneTouch® DMS and the User Manual.

**Autosearch.** The option whereby OneTouch® DMS will automatically search all PC COM ports until it finds the one that is connected to the meter. You may select Autosearch in the General Tab of the Preferences screen.

**Average Readings Report.** A Report that displays glucose averages by Meal Slot, Exercise period, and day of the week.

**bolus dose.** The additional units of insulin your body needs around meal times. Insulin dosing is usually called bolus when administered via an insulin pump. Bolus dosing data may be added to the database in the Manual Entry screens.

**button.** These are shaded boxes that appear on the bottom of screens. You can click on these boxes to accept or cancel data entries.

**calendar pop-up box.** A pop-up display screen that lets you change the calendar date. The screen is activated by clicking on the down arrow in any Date field.

**carbs.** One of the main nutrient groups, carbohydrates, also known as sugars and starches, are the preferred energy source for the body. Diabetes occurs when carbohydrates are eaten but not used properly by the body. Managing carbohydrate intake is key to maintaining good glucose control. OneTouch® DMS provides a way to track carbohydrates along with other diabetes health related information.

**cardiovascular complication.** A complication that involves the heart or the major blood vessels.

**cerebrovascular complication.** A complication that involves the blood vessels in and around the brain.

**check strip.** A special test strip used to verify that your meter and test strips are working properly. Check strips are used with the OneTouch® II, OneTouch® BASIC® and OneTouch® Profile® Meters. Look on the back of your meter to see the acceptable range of your check strip results.

**clear meter.** Permanently erases all readings from the meter memory.

**comment.** A text entry that is tagged to a Glucose, Insulin or Health Record.

**communications mode.** This setting defines the interface standard used for communications between your PC COM port and the meter. It applies only when connecting to a OneTouch® II Meter. In this case, the communications mode must be set to "RS-232."

**complication.** A health condition that develops from diabetes. OneTouch® DMS lets you keep track of complications for each patient.

**control solution.** A solution containing a measured amount of glucose that reacts with a OneTouch® Test Strip. It is used to test the accuracy of a new vial of OneTouch® Test Strips and the accuracy of the meter. Glucose readings using a control solution sample may be added to the database in the Manual Entry-A screen.

**Data List Report.** A Report that displays a list of all glucose, insulin, food, exercise, and other health data stored in the database.

**data entry field.** Any place on a screen where you can enter values such as a number, name, or text.

**data port.** The outlet on the meter that connects to the OneTouch™ Interface Cable. For meters that require an adapter, the data port is the strip port.

**data rate.** Refers to the speed of communications between your PC and the meter. It applies only when connecting to a OneTouch® II Meter. In this case, the data rate should be set at 9600.

**data record.** A unit of data about a patient in the database. A record may be a glucose reading, insulin dosing, or other health data.

**date range.** Represents the time frame for capturing data when viewing and printing Reports.

**Day View Screen.** This is the drill-down feature of the Logbook Report. The Day View screen lets you display additional information (including multiple readings) for any given day. Data records may also be edited from within the Day View screen.

**defaults.** Refers to options (data values or Program tasks) that are pre-set. Defaults are based on the most common user selections. You may modify the default settings.

**drill-down.** The feature that lets you access additional information from the Logbook and Health Checks Reports.

**drop-down menu.** The list of options that is displayed after clicking arrow buttons on screens or entries on the Menu Bar.

**Exception Report.** A OneTouch® DMS Report that ranks patients on key diabetes related information. Additional sorting capability is provided for patient groups, characterized by their insurance coverage. Designed in log format, each column of information can be sorted to provide flexibility in identifying those patients who may not be under good diabetes control.

**exercise record.** A data record that contains the duration and intensity of an exercise period.

**Favorite Report.** The OneTouch® DMS Report the Program will go directly to after readings are transferred from the meter.

**file.** A collection of data that is labeled and stored in your PC, or on an external storage device such as a floppy disk. Your software and database are stored as files on your PC.

**food record.** A data record that contains carbohydrate, fat, protein, or calorie data.

**footnotes.** Text messages that appear on the bottom of Reports whenever one or more glucose readings were outside the linear range of the meter. Footnotes also appear if readings have been edited or omitted from the Report.

**gestational.** A type of diabetes that occurs as a result of pregnancy.

**Get Meter Readings.** The function that transfers readings from the meter to the database.

**glucose average.** The arithmetic mean calculated on a set of glucose readings.

**Guest.** A permanent record in the database reserved for any patient who is not regularly served by OneTouch® DMS. Database records for a *Guest* are automatically deleted when you start OneTouch® Software. They are also deleted whenever new data for a *Guest* are transferred from a meter.

**Glucose Trend Report.** A OneTouch® DMS Report that displays an X-Y graph of glucose levels over a selected Date Range.

**HbA1c.** An abbreviated name for a glycated hemoglobin test/test result. Also commonly referred to as A1c. An HbA1c test result indicates your average glucose level over the past 3 to 4 months.

**Health Checks Report.** A OneTouch® DMS Report that displays home, lab, and doctor exam/test results.

**Health Record.** A data record that contains home, lab, or doctor exam/test results.

**HIGH glucose readings.** Glucose readings that were above the linear range of the meter. OneTouch® DMS Reports will display HIGH readings as either 501 mg/dL (27.8 mmol/L) or 601 mg/dL (33.4 mmol/L) depending on the meter from where the reading was taken. HIGH readings will also be converted to these values for calculation purposes.

**Histogram Report.** A OneTouch® DMS Report that displays glucose readings by Meal Slot over a set of pre-defined glucose ranges.

**hover-over.** The OneTouch® DMS feature that lets you display ToolTip information about a particular reading by moving the mouse cursor over the value.

**hypoglycemia.** A medical condition in which the glucose level is too low.

**injection.** An insulin dose administered by syringe or pen.

**Insulin Report.** A OneTouch® DMS Report that displays glucose readings, carbs, and insulin doses over the same Meal Slots.

**ketones.** A compound produced when not enough insulin is available to help the body break down glucose. The presence of ketones in the urine is a sign that glucose levels have been very high.

**Logbook Report.** This OneTouch® DMS Report displays daily glucose readings and other data in a logbook format. Data are sorted by Date and Meal Slot.

**LOW glucose readings.** Glucose readings that were below the linear range of the meter.

**Meal Slot.** How the OneTouch® DMS organizes the day into separate time periods. Many of the OneTouch® DMS Reports organize data by Meal Slot.

**Menu Bar.** An area above the Tool Bar that contains a series of text Headers that you can click on to display drop-down menus for OneTouch® DMS functions. You can select any menu option to perform that Software task.

**meter ID.** A unique serial number that is stored in the memory of each OneTouch® Meter. The Software retrieves the meter ID number when transferring meter readings so that it can keep track of the meter from which the data originated.

**microalbumin.** A measure of the amount of protein that is leaking into the urine. The presence of microalbumin in the urine may be a sign of kidney damage.

**nephropathic complication.** Any complication that involves the kidneys.

**neuropathic complication.** Any complication that involves the nervous system.

**omit.** Marks a data record for “omission” from calculations and Reports. The record remains in the database and may be re-included for future analyses using the “Restore” button.

**OneTouch® Interface Cable.** A cable used to connect a PC to your OneTouch® Meter.

**patient.** The standard data record in the OneTouch® DMS database.

**peripheral vascular complication.** Any complication that involves the capillaries or other small blood vessels.

**Preferences.** Settings that let you customize how you use and interact with the Software. Preferences are set in the Preferences screen.

**Pie Chart Report.** A OneTouch® DMS Report that displays pie charts of the percentage of within-target and out-of-target glucose readings over a particular Date Range.

**plasma reading.** A glucose measurement from only the plasma portion of a whole blood sample.

**readings.** One or more glucose level measurements recorded by the meter.

**regimen.** The daily schedule of insulin injections. OneTouch® DMS lets you define up to five insulin regimens for each patient.

**retinopathic complication.** Any complication that involves the retina and other areas of the eye.

**RS-232.** An interface standard for connecting serial devices to your PC. The data communication mode must be set to “RS-232,” when using a OneTouch® II Meter with the OneTouch® DMS. All other OneTouch® Meters are already pre-set to communicate using a RS-232 interface.

**schedule.** Seven pre-labeled Meal Slots that define how the Software divides the day. You can set the actual clock times for each Meal Slot on the Schedule Tab of the Patient Setup screen.

**selected patient.** The patient name that appears in the Patient field at the top of most OneTouch® DMS screens. Unless noted otherwise, most OneTouch® DMS functions will affect this particular patient record. You can select a different patient by clicking the arrow next to the box. This turns the box into a drop-down menu.

**serial port.** Refers to the communications port that connects your PC to a serial device. These may include your mouse controller, your printer, and the OneTouch™ Interface Cable (which connects to the meter or other serial devices).

**Standard Day Report.** A OneTouch® DMS Report that displays an X-Y graph of glucose readings over a 24-hour day.

**Statistics Panel.** A summary statistics panel of glucose readings compiled over the selected Date Range. The panel appears on several OneTouch® DMS Reports.

**Summary Report.** A OneTouch® DMS Report that includes portions of several Reports for a comprehensive overview of diabetes health status.

**target range.** The range of glucose readings that represent guidelines for “good” control.

**Tabs.** Refers to the way the OneTouch® DMS screens organize selection categories.

**Tool Bar.** An area below the Menu Bar that contains a series of icons that you can click on to perform many OneTouch® DMS functions.

**unit of measurement.** Refers to either mg/dL or mmol/L. Glucose values as well as other test results are reported in either mg/dL or mmol/L units of measure. Different units of measure are standard in different countries or regions. The glucose unit of measure appears on the meter display with a test result. OneTouch® DMS can automatically convert readings for display without affecting the setting in the meter.

**Unsuitable results.** Records transferred from a device that are invalid.

**Unsupported results.** Valid result types that are either not currently shown in OneTouch® DMS reports or include more information than expected.

**USB port.** Refers to **Universal Serial Bus**, an external bus standard that supports data transfer rates of 12 Mbps. A single USB port can be used to connect up to 127 peripheral devices, such as mice, modems, and keyboards.

**whole blood reading.** A glucose measurement from the entire whole blood sample.

## Appendix C: Troubleshooting

This appendix will help you solve some of the more common problems you may have when using OneTouch® DMS.

If you still require assistance, please call LifeScan Technical Support in your area or visit the Around the World section of the LifeScan website at LifeScan.com for the Technical Support telephone number in your area.

**Note:** The Software is not designed as a network application.

### Problems and Solutions

If you see these error messages on the screen:	You can do the following:
<p>“The meter is not responding. Check the cable/adapter connections.”</p> <p>or</p> <p>“Error-Could not find any meter on the serial port.”</p>	<ol style="list-style-type: none"> <li>1. Verify that the device is a OneTouch® Meter, or check OneTouch.com to see if it is supported by OneTouch® DMS.</li> <li>2. To begin data transfer, most meters need to be turned OFF; some meters need to be turned ON. If communication fails when you have the meter OFF, turn the meter ON.</li> <li>3. Make sure the cable is securely attached to the meter data port and the PC.</li> <li>4. Make sure you have selected the correct COM port (or the Autosearch) option on the Preferences screen.</li> <li>5. Make sure the cable is connected to a USB port.</li> <li>6. Make sure the port you are connecting to is not being used by an internal modem or any device software.</li> </ol>

**If you see these error messages on the screen:****You can do the following:**

I see data in my patient's meter that is not displayed in the OneTouch® DMS Reports.

The problem is most likely due to a meter date that was set incorrectly. As an example, if the year in the meter is incorrectly set to 2018, and the meter is subsequently used for testing, records in the meter memory will have dates in the future. When the meter is downloaded to OneTouch® DMS, the software may not “find” readings because the Software is looking for blood glucose data in the current year.

The following suggestions may help:

1) Use the Data List Report to see glucose readings since the last download. Select “Since last transfer” as the Date Range in the Data List Report. This will display all readings since the last download in the order in which the glucose tests were taken (as opposed to the calendar date order-which could have been affected by incorrect date settings in the meter). This feature is not available if you have transferred results from a OneTouch® UltraSmart® Meter.

Or

Select “Latest 14 days” when specifying the Date Range for viewing and printing Reports. This will show if there are any glucose results stamped with dates that are not the current year.

and

2) The data from the meter set to year 2018 are still in the database and in the meter. To prevent future problems, you should:

- Archive and delete the records with dates in the future
  - Clear the meter so that the data is not downloaded again
- and

3) Set the meter to the correct date and time.

## Appendix D: Technical Support

### OneTouch® DMS Support

If you have difficulty using OneTouch® DMS please consult the appropriate section of the user manual and help file. If that does not resolve the problem, please contact LifeScan for OneTouch® DMS technical support.

Contact your local LifeScan Office or visit the Around the World section of the LifeScan website at [Lifescan.com](http://Lifescan.com) for the Technical Support telephone number in your area.

## Appendix E: Index

Topic	Page Number
<b>A</b>	
<a href="#">Add A Patient Link</a>	28
<a href="#">Add Comments</a>	87
<a href="#">Add Data Records</a>	113
<a href="#">Add Exercise Records</a>	124
<a href="#">Add Food Records</a>	126
<a href="#">Add Glucose Records</a>	117
<a href="#">Add Health Records</a>	128
<a href="#">Add Insulin Link</a>	120
<a href="#">Add Insulin Records</a>	120
<a href="#">Add New Patient Button</a>	73
<a href="#">Add New Record Button</a>	14
<a href="#">Add Oral Medication Records</a>	122
<a href="#">Add Oral Meds Link</a>	123
<a href="#">Apply Button</a>	14
<a href="#">Archive Button</a>	14
<a href="#">Archiving</a>	133
<a href="#">Assigning Transferred Readings</a>	73
<a href="#">Average Readings Report</a>	100
<b>B</b>	
<a href="#">Back Arrow Icon</a>	147
<a href="#">Backup Data Records</a>	140
<a href="#">Beep</a>	61
<a href="#">Blood Pressure Results</a>	130

---

<a href="#">Button Behavior</a>	<a href="#">14</a>
---------------------------------	--------------------

## C

---

<a href="#">Cancel Button</a>	<a href="#">15</a>
<a href="#">Carbohydrate Intake</a>	<a href="#">96</a>
<a href="#">Cholesterol Results</a>	<a href="#">98</a>
<a href="#">Cholesterol Units</a>	<a href="#">46</a>
<a href="#">Clear Meter</a>	<a href="#">75</a>
<a href="#">Clipboard Icon</a>	<a href="#">147</a>
<a href="#">Clock, Meter</a>	<a href="#">59</a>
<a href="#">Clock, PC</a>	<a href="#">59</a>
<a href="#">Colors in Reports</a>	<a href="#">84</a>
<a href="#">COM Ports</a>	<a href="#">45</a>
<a href="#">Commands</a>	<a href="#">14</a>
<a href="#">Conversion from Whole Blood to Plasma Values</a>	<a href="#">48</a>

## D

---

<a href="#">Data List Report</a>	<a href="#">104</a>
<a href="#">Data Record Status, Added</a>	<a href="#">113</a>
<a href="#">Data Record Status, Edited</a>	<a href="#">111</a>
<a href="#">Data Record Status, Omitted</a>	<a href="#">104</a>
<a href="#">Database Backup</a>	<a href="#">140</a>
<a href="#">Date Format</a>	<a href="#">61</a>
<a href="#">Date Range</a>	<a href="#">78</a>
<a href="#">Day View Screen</a>	<a href="#">90</a>
<a href="#">Default Date Range</a>	<a href="#">50</a>
<a href="#">Delete Button</a>	<a href="#">15</a>
<a href="#">Delete Data Records</a>	<a href="#">15</a>

<a href="#">Delete Patient</a>	<a href="#">15</a>
<a href="#">Diabetes Type Target Settings</a>	<a href="#">52</a>
<a href="#">Done Button</a>	<a href="#">15</a>
<a href="#">Download Meter Readings</a>	<a href="#">69</a>
<a href="#">Dr. Visits Records</a>	<a href="#">129</a>
<a href="#">Drill-Down Feature</a>	<a href="#">149</a>

## **E**

<a href="#">Edit Data Records</a>	<a href="#">111</a>
<a href="#">E-Mail Reports</a>	<a href="#">109</a>
<a href="#">Exception Report</a>	<a href="#">102</a>
<a href="#">Exercise Records</a>	<a href="#">124</a>
<a href="#">Export Button</a>	<a href="#">14</a>
<a href="#">Exporting</a>	<a href="#">137</a>

## **F**

<a href="#">Favorite Report</a>	<a href="#">50</a>
<a href="#">Fax Reports</a>	<a href="#">108</a>
<a href="#">File Menu</a>	<a href="#">142</a>
<a href="#">Find A Patient Link</a>	<a href="#">43</a>
<a href="#">Folder Icon</a>	<a href="#">147</a>
<a href="#">Food Records</a>	<a href="#">126</a>
<a href="#">Footnotes in Reports</a>	<a href="#">83</a>

## **G**

<a href="#">Global Target Settings</a>	<a href="#">52</a>
<a href="#">Glucose Records</a>	<a href="#">117</a>
<a href="#">Glucose Trend Report</a>	<a href="#">92</a>

<a href="#">Glucose Units</a>	<a href="#">46</a>
<a href="#">Go Button</a>	<a href="#">15</a>
<a href="#">Guest Patient</a>	<a href="#">73</a>

## H

<a href="#">HbA1c Results</a>	<a href="#">129</a>
<a href="#">Health Checks Report</a>	<a href="#">98</a>
<a href="#">Health Notes</a>	<a href="#">89</a>
<a href="#">Health Records</a>	<a href="#">128</a>
<a href="#">Health Records, Blood Pressure</a>	<a href="#">130</a>
<a href="#">Health Records, Cholesterol</a>	<a href="#">131</a>
<a href="#">Health Records, Dr. Visits</a>	<a href="#">129</a>
<a href="#">Health Records, HbA1c</a>	<a href="#">129</a>
<a href="#">Health Records, Height</a>	<a href="#">130</a>
<a href="#">Health Records, Ketones</a>	<a href="#">129</a>
<a href="#">Health Records, Microalbumin</a>	<a href="#">130</a>
<a href="#">Health Records, Weight</a>	<a href="#">130</a>
<a href="#">Heart Icon</a>	<a href="#">147</a>
<a href="#">Height Results</a>	<a href="#">130</a>
<a href="#">Help Menu</a>	<a href="#">147</a>
<a href="#">HIGH Glucose Readings</a>	<a href="#">83</a>
<a href="#">Histogram Report</a>	<a href="#">106</a>
<a href="#">Home Screen</a>	<a href="#">24</a>
<a href="#">House Icon</a>	<a href="#">147</a>
<a href="#">Hover-Over Feature</a>	<a href="#">80</a>
<a href="#">Hypoglycemic Threshold</a>	<a href="#">55</a>
<a href="#">Hypoglycemic Values</a>	<a href="#">55</a>

**I**

<a href="#">Icons</a>	147
<a href="#">Import Button</a>	15
<a href="#">Importing</a>	139
<a href="#">Include Button</a>	15
<a href="#">Insulin</a>	121
<a href="#">Insulin Pump</a>	38
<a href="#">Insulin Records</a>	120
<a href="#">Insulin Regimen</a>	40
<a href="#">Insulin Report</a>	96
<a href="#">Insulin Types</a>	39
<a href="#">Insulin Units</a>	96
<a href="#">InTouch® Software</a>	19

**K**

<a href="#">Ketones Records</a>	129
<a href="#">Keyboard Conventions</a>	13

**L**

<a href="#">Letter Icon</a>	147
<a href="#">Linking to a Database</a>	20
<a href="#">Logbook Report</a>	85
<a href="#">LOW Glucose Readings</a>	88

**M**

<a href="#">Manual Entry Menu</a>	145
<a href="#">Manual Entry Screen, Exercise Records</a>	124

<a href="#">Manual Entry Screen, Food Records</a>	<a href="#">126</a>
<a href="#">Manual Entry Screen, Glucose Records</a>	<a href="#">117</a>
<a href="#">Manual Entry Screen, Health Records</a>	<a href="#">128</a>
<a href="#">Manual Entry Screen, Insulin Records</a>	<a href="#">120</a>
<a href="#">Manual Entry Screen, Oral Medication Records</a>	<a href="#">122</a>
<a href="#">Manually Entry Data</a>	<a href="#">113</a>
<a href="#">Meal Slots</a>	<a href="#">151</a>
<a href="#">Menu Bar Commands</a>	<a href="#">142</a>
<a href="#">Meter Display Languages</a>	<a href="#">61</a>
<a href="#">Meter Icon</a>	<a href="#">147</a>
<a href="#">Meter Menu</a>	<a href="#">143</a>
<a href="#">Meter Settings, Clock</a>	<a href="#">58</a>
<a href="#">Meter Settings, General</a>	<a href="#">60</a>
<a href="#">Meter Settings, Insulin</a>	<a href="#">64</a>
<a href="#">Meter Settings, Oral Meds</a>	<a href="#">65</a>
<a href="#">Meter Settings, Schedule</a>	<a href="#">62</a>
<a href="#">Meter Settings, Targets &amp; Hypoglycemic</a>	<a href="#">67</a>
<a href="#">Meter Setup</a>	<a href="#">143</a>
<a href="#">Microalbumin Results</a>	<a href="#">130</a>
<a href="#">Mouse Conventions</a>	<a href="#">13</a>

## O

<a href="#">OK Button</a>	<a href="#">15</a>
<a href="#">Omit Button</a>	<a href="#">16</a>
<a href="#">Omit Data Records</a>	<a href="#">91</a>
<a href="#">OneTouch® Meters</a>	<a href="#">9</a>
<a href="#">OneTouch™ Interface Cable</a>	<a href="#">152</a>
<a href="#">Oral Medication Records</a>	<a href="#">65</a>

<a href="#">Oral Medication Types</a>	<a href="#">66</a>
<a href="#">Outside-Target Readings</a>	<a href="#">85</a>

## P

<a href="#">Patient Icon</a>	<a href="#">147</a>
<a href="#">Patient Menu</a>	<a href="#">143</a>
<a href="#">Patient Setup Screen, Complications</a>	<a href="#">42</a>
<a href="#">Patient Setup Screen, Find a Patient</a>	<a href="#">43</a>
<a href="#">Patient Setup Screen, General Patient Profile Data</a>	<a href="#">30</a>
<a href="#">Patient Setup Screen, Insulin Regimen</a>	<a href="#">40</a>
<a href="#">Patient Setup Screen, Medications</a>	<a href="#">38</a>
<a href="#">Patient Setup Screen, Schedule</a>	<a href="#">33</a>
<a href="#">Pencil Icon</a>	<a href="#">147</a>
<a href="#">Personal Target Settings</a>	<a href="#">32</a>
<a href="#">Pie Chart Report</a>	<a href="#">93</a>
<a href="#">Plasma Values</a>	<a href="#">48</a>
<a href="#">Preferences Menu</a>	<a href="#">146, 147</a>
<a href="#">Preferences Screen, General Programming Environment</a>	<a href="#">45</a>
<a href="#">Preferences Screen, Global Target Settings</a>	<a href="#">52</a>
<a href="#">Preferences Screen, Hypoglycemic Value Setting</a>	<a href="#">55</a>
<a href="#">Preferences Screen, Report Options</a>	<a href="#">49</a>
<a href="#">Print Reports</a>	<a href="#">84</a>
<a href="#">Print Selected Reports Button</a>	<a href="#">107</a>
<a href="#">Printer Icon</a>	<a href="#">147</a>

## R

<a href="#">Record Details Screen</a>	<a href="#">87</a>
<a href="#">Report Colors</a>	<a href="#">84</a>

<a href="#">Report Headers</a>	<a href="#">81</a>
<a href="#">Report, Average Readings</a>	<a href="#">100</a>
<a href="#">Report, Data List</a>	<a href="#">104</a>
<a href="#">Report, Exception</a>	<a href="#">102</a>
<a href="#">Report, Favorite</a>	<a href="#">50</a>
<a href="#">Report, Glucose Trend</a>	<a href="#">92</a>
<a href="#">Report, Health Checks</a>	<a href="#">98</a>
<a href="#">Report, Histogram</a>	<a href="#">106</a>
<a href="#">Report, Insulin</a>	<a href="#">96</a>
<a href="#">Report, Logbook</a>	<a href="#">85</a>
<a href="#">Report, Pie Chart</a>	<a href="#">93</a>
<a href="#">Report, Standard Day</a>	<a href="#">95</a>
<a href="#">Report, Summary</a>	<a href="#">94</a>
<a href="#">Reports Menu</a>	<a href="#">144</a>
<a href="#">Reset to Defaults Button</a>	<a href="#">16</a>
<a href="#">Retrieve Button</a>	<a href="#">16</a>
<a href="#">Retrieve Data</a>	<a href="#">135</a>
<a href="#">RS-232</a>	<a href="#">152</a>

## S

<a href="#">Save Button</a>	<a href="#">16</a>
<a href="#">Schedule</a>	<a href="#">33</a>
<a href="#">Select Button</a>	<a href="#">16</a>
<a href="#">Standard Day Report</a>	<a href="#">95</a>
<a href="#">Start Up Software</a>	<a href="#">24</a>
<a href="#">Statistics Panel</a>	<a href="#">82</a>
<a href="#">Summary Report</a>	<a href="#">94</a>

## T

<a href="#">Target Ranges</a>	52
<a href="#">Technical Support</a>	156
<a href="#">Telephone Icon</a>	147
<a href="#">Test Strips</a>	48
<a href="#">Time Format</a>	61
<a href="#">Tool Bar Commands</a>	146, 147
<a href="#">ToolTips</a>	80
<a href="#">Transfer Meter Readings</a>	25
<a href="#">Troubleshooting</a>	154

## U

<a href="#">Uninstall Software</a>	23
<a href="#">Units</a>	46
<a href="#">Units of Measure</a>	47

## V

<a href="#">View Footnotes Link</a>	83
<a href="#">View Reports</a>	78
<a href="#">View Statistics and Targets Link</a>	94
<a href="#">View/Edit Targets Link</a>	32

## W

<a href="#">Weight Records</a>	130
<a href="#">Whole Blood Values</a>	48
<a href="#">Within-Target Readings</a>	93



©2007-2014 LifeScan, Inc.  
Rev. date: 10/2014 AW 06435001D



LifeScan, Inc.,  
Wayne, PA  
19087 USA